A&D A&D Medical

Wrist Digital Blood Pressure Monitor

Model UB-543

Instruction Manual
Original

Manual de Instrucciones
Traduction

ENGLISH

ESPAÑOL

Contents

Dear Customers.....	2
Preliminary Remarks	2
Precautions.....	2
Parts Identification	4
Symbols	5
Using the Monitor.....	7
1. Installing / Changing the Batteries.....	7
2. Selecting a User	8
3. Adjusting the Built-in Clock Before Use.....	8
4. Applying the Cuff	9
5. How to Take Proper Measurements.....	9
6. The C.P.G. Function Switch	10
7. Selecting the C.P.G. and my C.P.G.	10
8. Measurement	11
9. After Measurement.....	11
Measurements	12
Recalling the Memory Data	13
Deleting Data Stored in Memory.....	14
What is an Irregular Heartbeat?.....	15
Pressure Bar Indicator	15
The C.P.G. Indicator	16
About Blood Pressure.....	17
What is Blood Pressure?.....	17
What is Hypertension and How is it Controlled?	17
Why Measure Blood Pressure at Home?	17
Blood Pressure Variations.....	18
Maintenance	18
Troubleshooting	19
Technical Data.....	20

Dear Customers

Congratulations on purchasing a state-of-the-art A&D blood pressure monitor, one of the most advanced monitors available today. Designed for ease of use and accuracy, this device will facilitate your daily blood pressure regimen. **We recommend that you read through this manual carefully before using the device for the first time.**

Preliminary Remarks

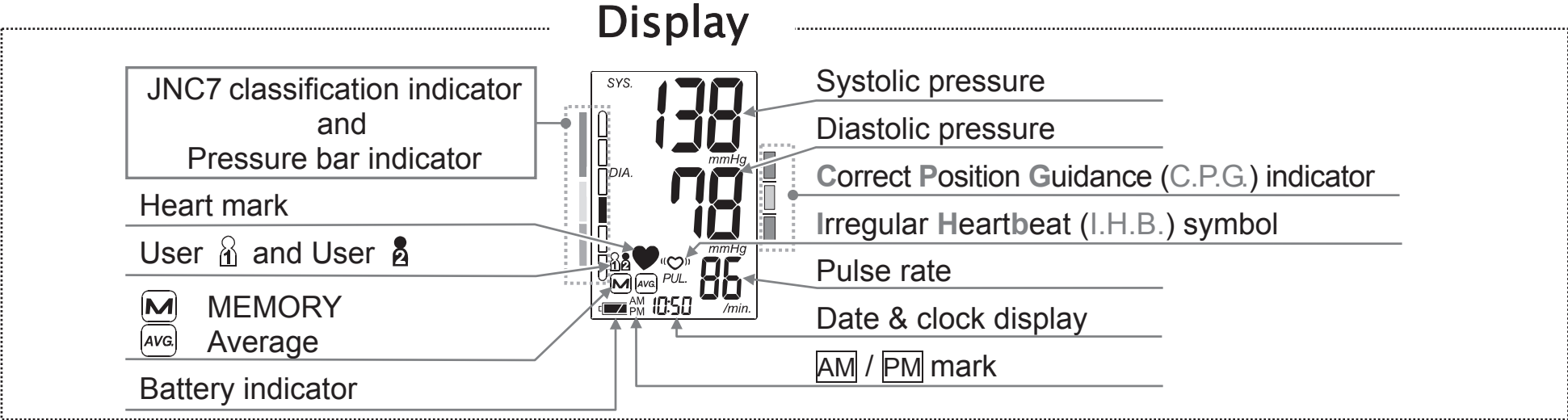
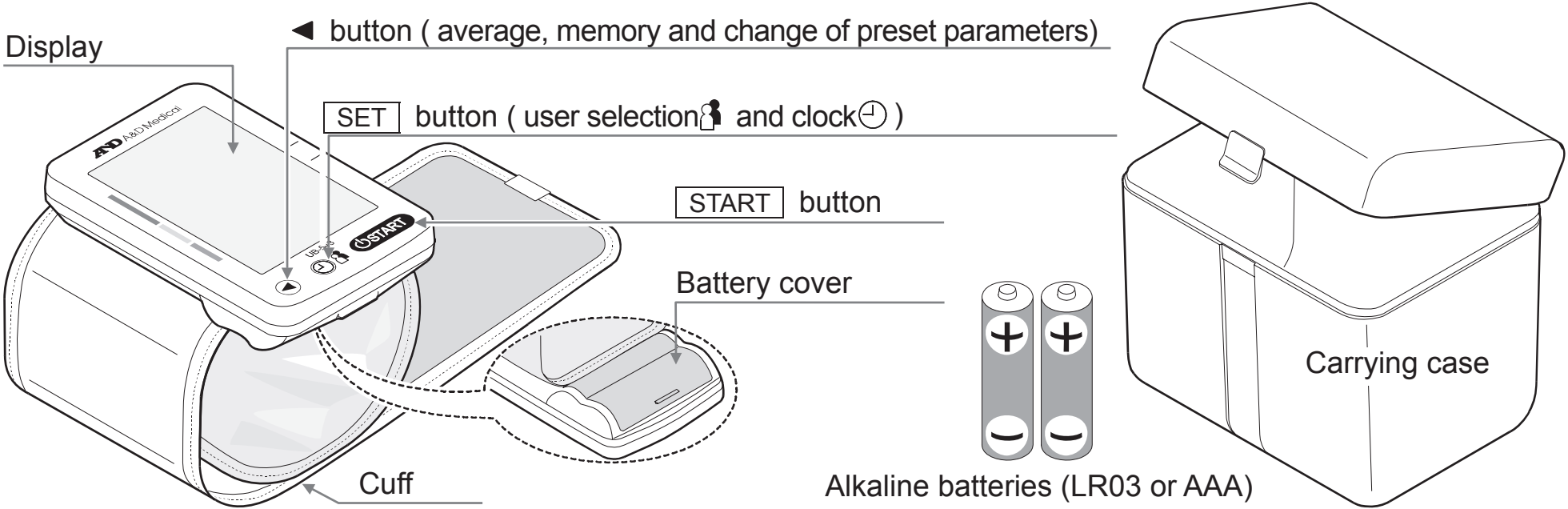
- The device is designed for use on adults only, not newborns or infants.
- Environment for use. The device is for use in the home healthcare environment.
- This device is designed to measure blood pressure and pulse rate of people for diagnosis.

Precautions

- Precision components are used in the construction of this device. Extremes in temperature, humidity, direct sunlight, shock or dust should be avoided.
- Clean the device with a dry, soft cloth or a cloth dampened with water and a neutral detergent. Never use alcohol, benzene, thinner or other harsh chemicals to clean the device.
- Avoid tightly folding the cuff for long periods, as such treatment may shorten the life of the components.
- The device is not water resistant. Prevent rain, sweat and water from soiling the device.
- Measurements may be distorted if the device is used close to televisions, microwave ovens, cellular telephones, X-ray or other devices with strong electrical fields.
- Used equipment, parts and batteries are not treated as ordinary household waste, and must be disposed of according to the applicable local regulations.
- When reusing the device, confirm that the device is clean.

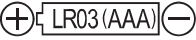
- ❑ Do not modify the device. It may cause accidents or damage to the device.
- ❑ To measure blood pressure, the wrist must be squeezed by the cuff hard enough to temporarily stop blood flow through the artery. This may cause pain, numbness or a temporary red mark to the wrist. This condition will appear especially when measurement is repeated successively. Any pain, numbness, or red marks will disappear with time.
- ❑ Do not apply the device to a wrist with other medical electrical equipment attached. The equipment may not function properly.
- ❑ People who have a severe circulatory deficit in the arm must consult a doctor before using the device to avoid medical problems.
- ❑ Do not self-diagnose the measurement results and start treatment by yourself. Always consult your doctor for evaluation of the results and treatment.
- ❑ Do not apply the device on a wrist with an unhealed wound.
- ❑ Do not apply the device on an arm receiving an intravenous drip or blood transfusion. It may cause injury or accidents.
- ❑ Do not use the device where flammable gases such as anesthetic gases are present. It may cause an explosion.
- ❑ Do not use the device in highly concentrated oxygen environments, such as a high-pressure oxygen chamber or an oxygen tent. It may cause a fire or explosion.

Parts Identification

Symbols

Symbols that are printed on the device case

Symbols	Function / Meaning
	Standby and Turn the device on
	Battery installation guide
	Direct current
SN	Serial number
	Type BF: Device and cuff are designed to provide special protection against electrical shocks.
	Refer to instruction manual / booklet
	User selection
IP	International protection symbol
	Keep dry

Symbols that appear on the display

Symbols	Function / Meaning / Recommended Action
	The heart mark appears while measurement is in progress. It blinks when the pulse is detected. Remain as still as possible.
	Irregular Heartbeat (I.H.B.) symbol appears when an irregular heartbeat is detected. It may light when a very slight vibration like shivering or shaking is detected.
	User 1 and user 2
	Previous measurements stored in MEMORY
	Average data

Symbols

Symbols that appear on the display (continued)

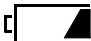
Symbols	Function / Meaning	Recommended Action
 Full Battery	The battery power indicator during measurement	————
 Low Battery	The battery is low when it blinks	Replace all batteries with new ones when the indicator blinks.
Err ₁ or Err ₂	Unstable blood pressure due to movement during measurement	Take another measurement. Remain still during measurement.
	The systolic and diastolic values are within 10 mmHg of each other.	
Err ₃	The pressure value did not increase during inflation.	Apply the cuff correctly, and take another measurement.
	The cuff is not applied correctly.	
E PUL. DISPLAY ERROR	The pulse is not detected correctly.	
Err _E	Blood pressure monitor internal error	Remove the batteries and press the START button, and then install the batteries again. If the error still appears, contact A&D Medical.
Err _g		
SYS.	Systolic blood pressure in mmHg	————
DIA.	Diastolic blood pressure in mmHg	————
PUL./min.	Pulse per minute	————
AM	Data taken between 4:00 and 9:59	————
PM	Data taken between 18:00 and 1:59	————


Using the Monitor

1. Installing / Changing the Batteries

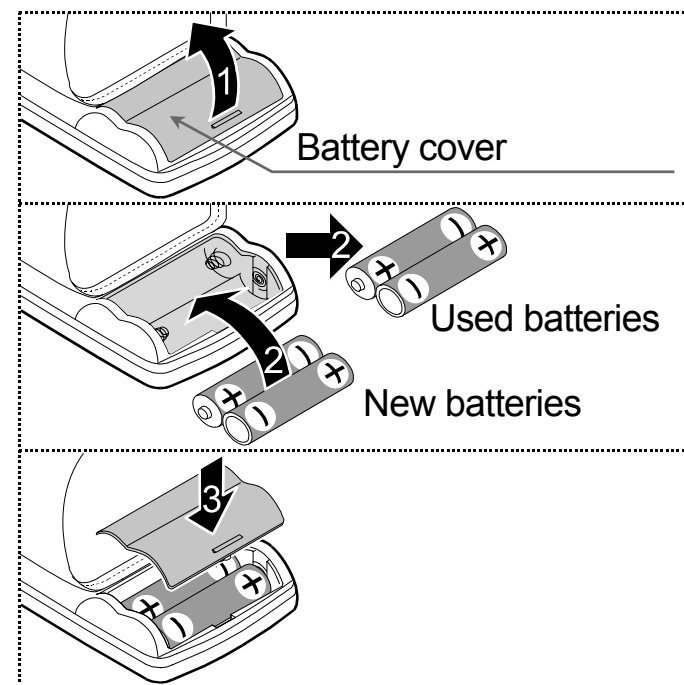1. Remove the battery cover.
2. Remove the used batteries and insert new batteries into the battery compartment as shown, taking care that the polarities (+ and -) are correct. Use only LR03 or AAA batteries.
3. Attach the battery cover.

⚠ CAUTIONS

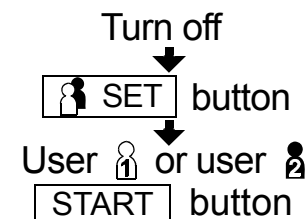
- ❑ Insert the batteries as shown in the battery compartment. If installed incorrectly, the device will not work.
- ❑ When  (LOW BATTERY mark) appears on the display, replace all batteries with new ones. Do not mix old and new batteries. It may shorten the battery life, or cause the device to malfunction.
- ❑  (LOW BATTERY mark) does not appear when the batteries are drained.
- ❑ The battery life varies with the ambient temperature and may be shorter at low temperatures.
- ❑ Use the specified batteries only. The batteries provided with the device are for testing monitor performance and may have a limited life.
- ❑ Remove the batteries if the device is not to be used for a long period of time. The batteries may leak and cause a malfunction.
- ❑ When removing the batteries, preset parameters (of clock, user and my C.P.G.) are reset.



Using the Monitor

2. Selecting a User

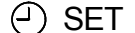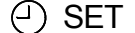1. Press the  SET button when turning off the device. The indicator  or  is blinking.
2. Select a user from user  and user  using the  SET button.
Press the  button to turn off the device.
After three minutes of non-operation, the device will turn off automatically.


3. Adjusting the Built-in Clock Before Use


1. Press and hold the  SET button until the year starts blinking.
2. Select the year using the ◀ button.
Press the  SET button to set the current year and move to month/day selection.
The date can be set anywhere between the years 2010 and 2059.
3. Select the month using the ◀ button.
Press the  SET button to set the current month and move to day selection.
4. Select the day using the ◀ button.
Press the  SET button to set the current day and move to hour/minute selection.
5. Select the hour using the ◀ button.
Press the  SET button to set the current hour and move to minute selection.
6. Select the minute using the ◀ button.
Press the  SET button to proceed to "4. Applying the Cuff".

- Holding down the ◀ button will change the value continuously.
- Pressing the  button will turn the device off anytime.

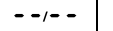Notes: After three minutes of non-operation, the device will turn off automatically.

When the clock has not been set,  is indicated for the clock display.

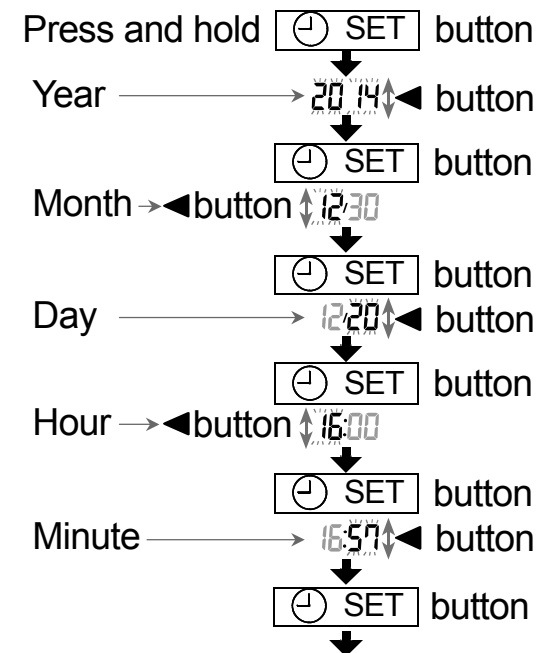When removing the batteries, preset parameters (of clock, user and my C.P.G.) are reset.


The operation proceeds to "6. The C.P.G. Function Switch".

Using the Monitor

4. Applying the Cuff

1. Wrap the cuff around your wrist about 1 cm above your hand as shown in the figure at the right.
2. Fasten the cuff tightly using the Velcro strip.

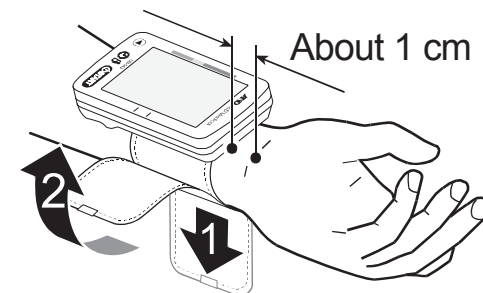Note: For proper measurements, fasten the cuff tightly and measure on a bare wrist.

5. How to Take Proper Measurements

For the most accurate blood pressure measurement:

- Remain still and keep quiet during measurement.
- Sit down in a comfortable position. Place your elbow on a table. Raise your hand so that the cuff is at the same level as your heart.
- Relax for several minutes before measurement. If you are excited or depressed by emotional stress, the measurement will reflect this stress as a higher (or lower) than normal blood pressure reading and the pulse reading will usually be faster than normal.
- Try to measure your blood pressure at about the same time every day.
- An individual's blood pressure varies constantly, depending on what they are doing, what they have eaten and what they drink can have a very strong and rapid effect on your blood pressure.
- Do not measure immediately after physical exercise or a bath. Rest for twenty or thirty minutes before taking the measurement.
- Do not cross your legs. Keep your feet on the floor and straighten your back.
- This device bases its measurements on the heartbeat. If you have a very weak or irregular heartbeat, the device may have difficulty determining your blood pressure.
- Should the device detect a condition that is abnormal, it will stop the measurement and display an error symbol. See page 6 for the description of symbols.
- This blood pressure monitor is intended for use by adults only. Consult with your physician before using this device on a child. A child should not use this device unattended.

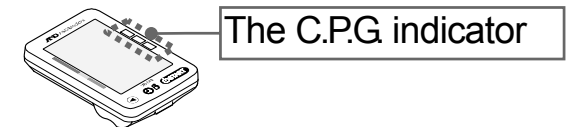
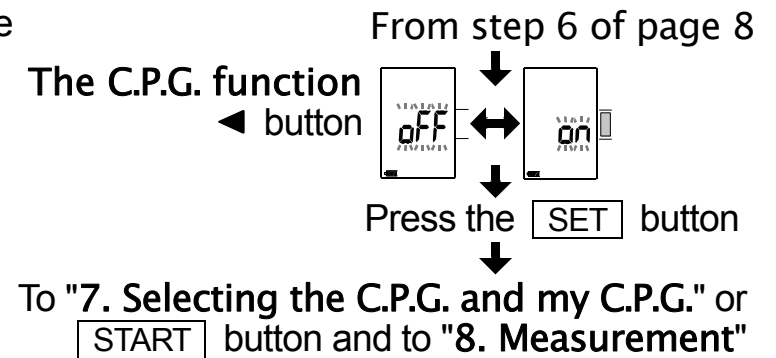
6. The C.P.G. Function Switch

- Refer to page 16 for the C.P.G. function that will indicate the proper angle so that the height of the cuff is the same level as your heart.
- 1. After step 6 in page 8, press the ◀ button to select either "on" or "off" concerning the C.P.G. function.
- 2. Press the **SET** button to store the selection.
- 3. □ If you do not use my C.P.G. function, press **START** button to turn off. Proceed to "8. Measurement".
 - If you use my C.P.G. function, proceed to "7. Selecting the C.P.G. and my C.P.G."


7. Selecting the C.P.G. and my C.P.G.

- You can select an indicator either the C.P.G. function or my C.P.G. .
- Preset a proper posture (wrist angle) in memory if you use the my C.P.G. .
- 1. Adjust and keep the height of the blood pressure monitor to the same level as your heart using your wrist angle.
- 2. Select an indicator using the ◀ button.
 - Indicator The C.P.G. function is used. (my C.P.G. function : **OFF**)
Data of my C.P.G. is deleted. Proceed to step 3.
 - Indicator my C.P.G. function is **ON** and the current angle is stored when switched to the indicator . Proceed to step 3.
- 3. Press the **START** button to turn off the device.


Note: When removing the batteries, preset parameters (of clock, user and my C.P.G.) are reset.

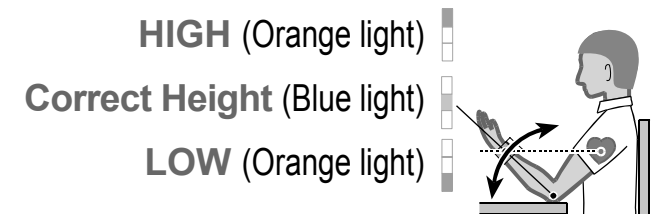Step 1 [Adjust and keep the height of the device (wrist angle)]

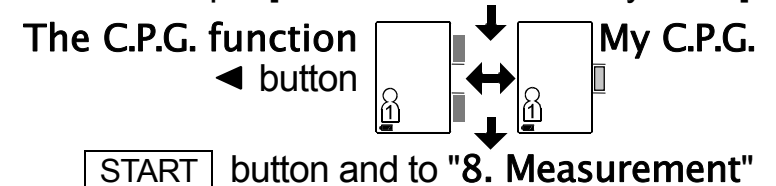Step 2 [Select The C.P.G. or my C.P.G.]

8. Measurement

During measurement, it is normal for the cuff to feel very tight.

9. After Measurement

While readings are displayed, if you press the START button to turn the device off, new readings are stored in memory.

While readings are displayed, if you press the ◀ button to turn the device off, new readings are not stored.

Remove the cuff and record your data.

Notes: The device is provided with an automatic power shut-off function which stores the current data in the memory and turns the device off automatically one minute after measurement. Allow at least three minutes between measurements on the same person.

Measurements

Note: The UB-543, once used, will provide an inflation appropriate to the user.

1. Wrap the cuff around your wrist. Sit comfortably with the cuff at the same level as your heart and relax.

2. Press the **START** button. All of the display segments are displayed.

3. Select a user from user 1 and user 2 using the **SET** button immediately. Adjust and keep the height of the cuff (with blue light) to the same level as your heart using the C.P.G. indicator.

Note: If you do not use user selection, wait for the inflation for several seconds. If you do not use the C.P.G. function, the C.P.G. indicator is not displayed.

4. Zero (0) is displayed blinking briefly. Then the display changes, as measurement begins. The cuff starts to inflate. It is normal for the cuff to feel very tight. The measurement starts automatically when inflation starts, and the ♥ (heart mark) blinks.

Note: If you wish to stop inflation at any time, press the **START** button again.

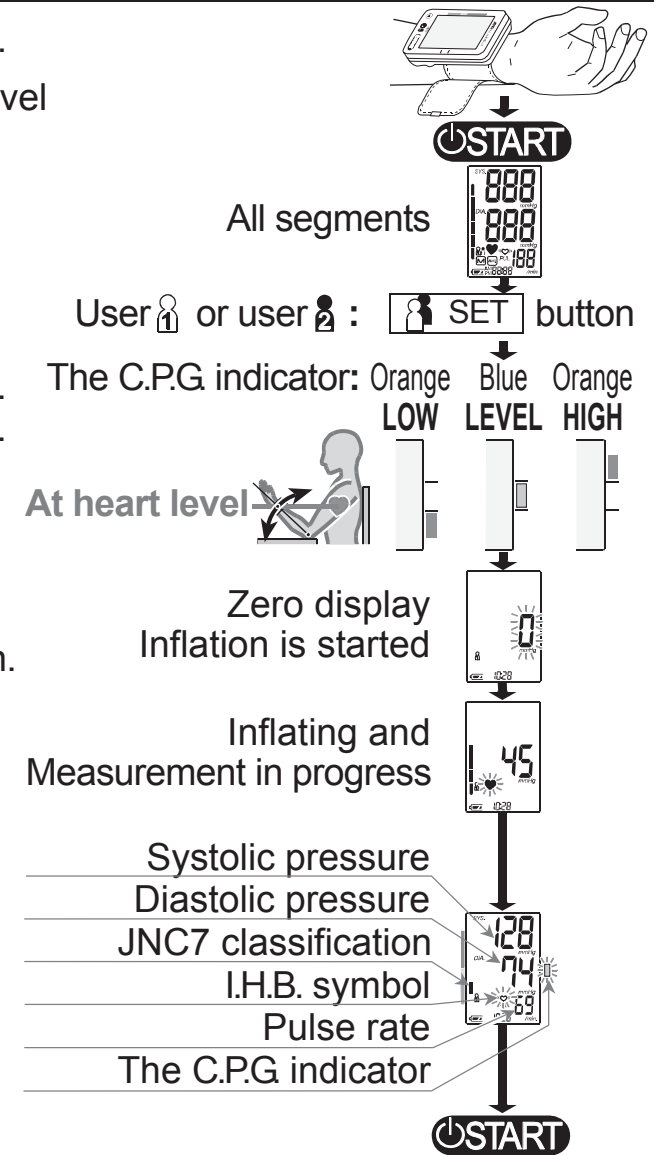
5. When the measurement is complete, the device displays readings (of the systolic and diastolic pressure readings, pulse rate, WHO classification, I.H.B. symbol and C.P.G. indicator). While readings are displayed, the date and time are displayed alternately. The cuff exhausts the remaining air and completely deflates automatically.

Note: If you do not want to store new readings in memory, press the ◀ button while readings are displayed.

6. Press the **START** button again to turn the device off. Remove the cuff.

Notes: The device is provided with an automatic power shut-off function.


Allow at least one minute between measurements on the same person.

Recalling the Memory Data

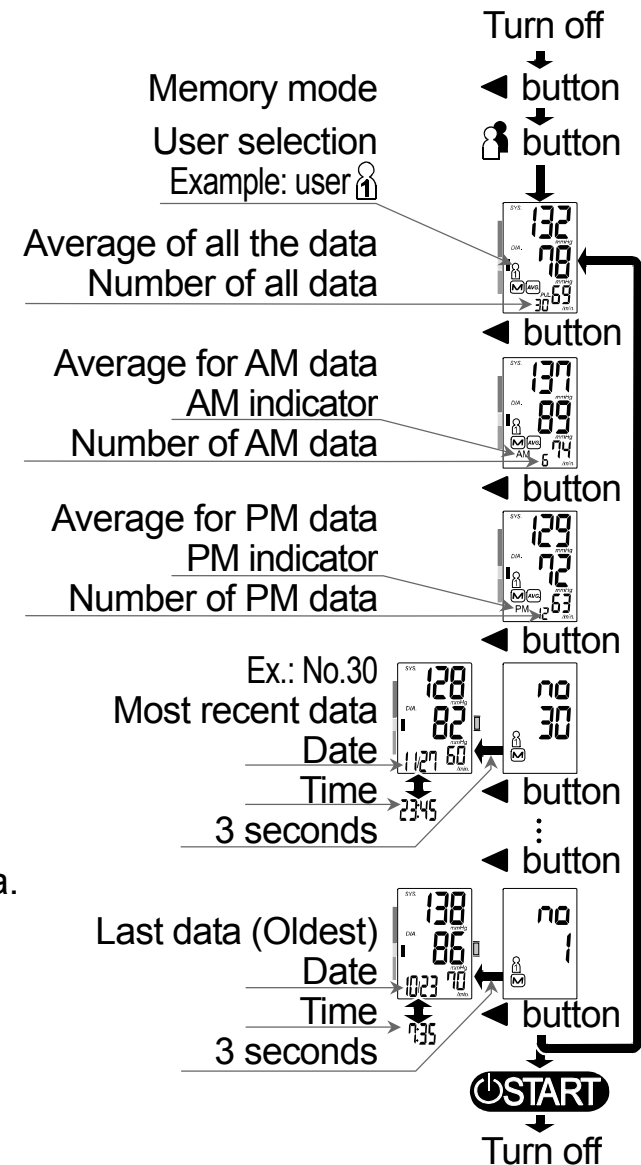Note: This device stores the last 60 measurements in memory.

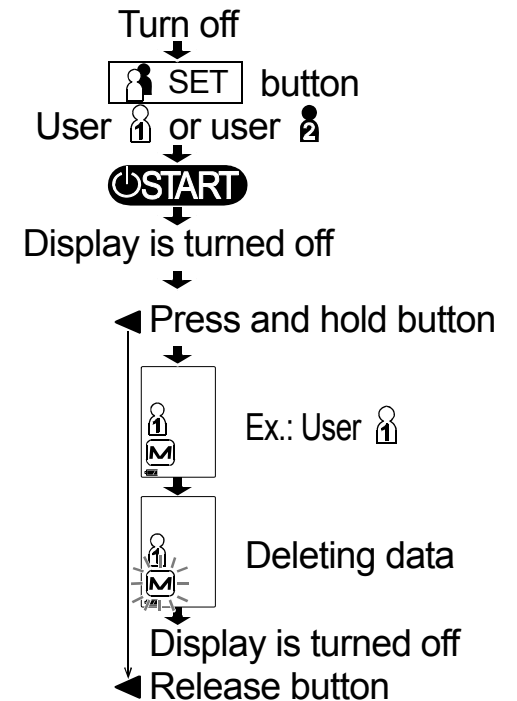
1. Press the ◀ button when turning the device off.
The average of all measurements and the number of data are displayed.
If no data, "0" is displayed. Press the ◀ or [START] button to turn the device off.
2. Use the following buttons to display data (of number and measurement data).
 - Select a user from user 1 and user 2 using the [SET] button.
The device displays the average of all measurements and the number of data are displayed.
 - Pressing the ◀ button each time, the device displays as follows:
 - Average data of all AM (morning) measurements taken between 4:00 and 9:59. In the example, If no data, "--" is displayed.
 - Average data of all PM (evening) measurements taken between 18:00 and 1:59.
 - Data (of number and measurement data).
The device displays in order from most recent data. The date and time are displayed alternately while displaying the measurement data.
In the example: No.30 & data → No.29 & data → ... → No.01 & data.
3. If you press the ◀ button after oldest data is displayed, the device proceeds to step 1, the average of all measurements and the number of data are displayed.
4. Press the [START] button to turn the device off.
After one minute of non-operation, the device will turn off automatically.

Deleting Data Stored in Memory

1. Select a user from user 1 and user 2 using the  SET button. Turn off the device using the  button.
2. Press and hold the  button until turning off the device automatically. The device displays a user icon and the  mark, deletes data stored in memory while blinking the  mark and turns off automatically.

Note: This operation will delete specified user data stored in memory. You cannot select which data to delete.

What is Irregular Heartbeat?

Model UB-543 blood pressure monitor provides a blood pressure and pulse rate measurement even when an irregular heartbeat occurs. An irregular heartbeat is defined as a heartbeat that varies by 25% from the average of all heartbeats during the blood pressure measurement. It is important that you be relaxed, remain still and do not talk during measurements.

Note: We recommend contacting your physician if you see this «♥» symbol frequently.

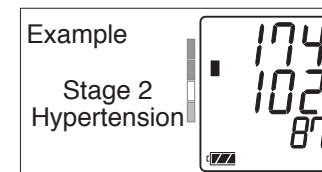
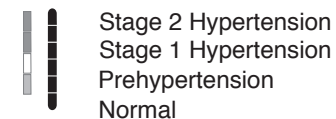
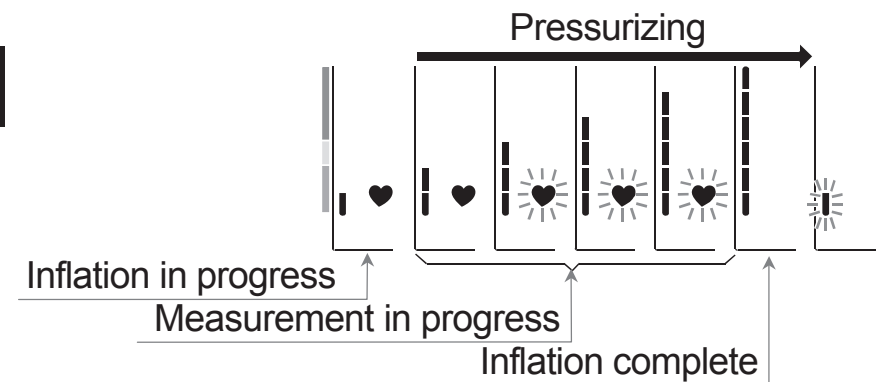
Pressure Bar Indicator

The Pressure Rating Indicator is a feature which provides a snapshot of your blood pressure classification based on your measurements. This will let you know what your blood pressure readings mean.

Each segment of the bar indicator corresponds to the Seventh Report of the Joint National Committee (JNC7) on Prevention, Evaluation and Treatment of High Blood Pressure from the National Heart, Lung and Blood Institute - May 2003.

■ : The indicator displays a segment, based on the current measurement, corresponding to the JNC7 Classification.



NOTE: Due to other risk factors (e.g. diabetes, obesity, smoking, etc.) in addition to your blood pressure measurement, the Pressure Rating Indicator is approximate. Please consult with your physician for interpretation and diagnosis of your blood pressure measurements.

The C.P.G Indicator

❑ The C.P.G. Indicator

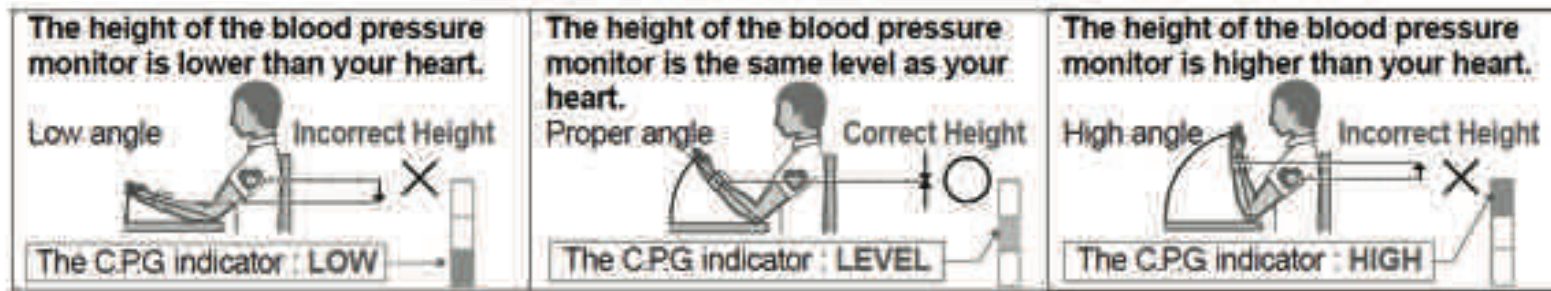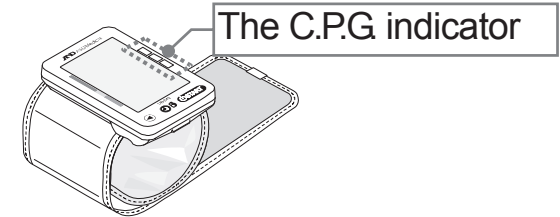The C.P.G. (**C**orrect **P**osition **G**uidance) indicator is the function to inform a difference between the height (wrist angle) of the blood pressure monitor and your cardiac height in the correct posture (Example: sitting posture, height of table and chair, etc.) during the measurement. The indicator can be used to get more stable measurement condition.

The position of the device is checked both before and after measurement.

If both checks show a correct measurement position the LEVEL indicator is lit (blue)

For all other measurements an indicator for LOW or HIGH measurement position will be lit (orange)

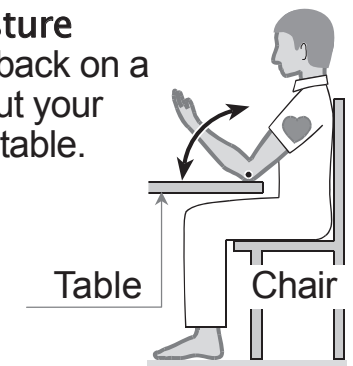
❑ How To Use My C.P.G.

The C.P.G. function can be used with proper posture (wrist angle) in the majority of measurements. If you need to change the posture to adjust the height so that the height of the blood pressure monitor is the same level as your cardiac height, you can use my C.P.G. function to store a personal posture. Preset your angle to my C.P.G. function before measurement.

❑ The Indicator During Measurement And Recalling Memory

The C.P.G. indicator can be displayed at the beginning of measurement and is included in data stored in the memory. Adjust and keep the angle during measurement.

- ❑ **Proper Posture**
Sit straight back on a chair and put your elbow on a table.

About Blood Pressure

What is Blood Pressure?

Blood pressure is the force exerted by blood against the walls of the arteries. Systolic pressure occurs when the heart contracts. Diastolic pressure occurs when the heart expands. Blood pressure is measured in millimeters of mercury (mmHg). One's natural blood pressure is represented by the fundamental pressure, which is measured first thing in the morning while one is still at rest and before eating.

What is Hypertension and How is it Controlled?

Hypertension, an abnormally high arterial blood pressure, if left unattended can cause many health problems including stroke and heart attack. Hypertension can be controlled by altering lifestyle, avoiding stress, and with medication under a doctor's supervision.

To prevent hypertension or keep it under control:

- Do not smoke
- Exercise regularly
- Reduce salt and fat intake
- Have regular physical checkups
- Maintain proper weight

Why Measure Blood Pressure at Home?

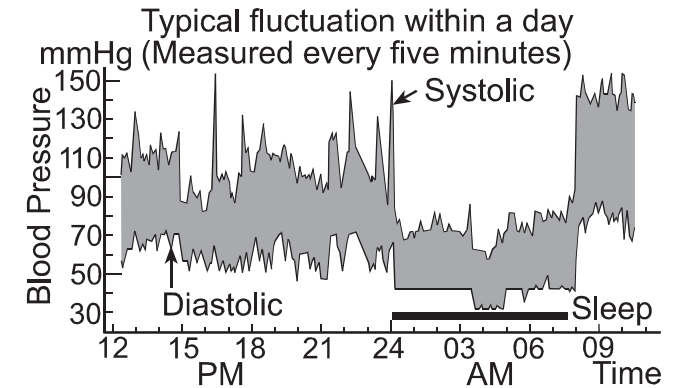
It is now well known that, for many individuals, blood pressure readings taken in a doctor's office or hospital setting might be elevated as a result of apprehension and anxiety. This response is commonly called "white coat hypertension."

In any case, self-measurement at home supplements your doctor's readings and provides a more accurate, complete blood pressure history. In addition, clinical studies have shown that the detection and treatment of hypertension is improved when patients both consult their physicians and monitor their own blood pressure at home.

Blood Pressure Variations

An individual's blood pressure varies greatly on a daily and seasonal basis. It may vary by 30 to 50 mmHg due to various conditions during the day. In hypertensive individuals variations are even more pronounced. Normally, the blood pressure rises while at work or play and falls to its lowest levels during sleep. So, do not be overly concerned by the results of one measurement.

Take measurements at the same time every day using the procedure described in this manual to get to know your normal blood pressure. Regular readings give a more comprehensive blood pressure history. Be sure to note the date and time when recording your blood pressure. Consult your doctor to interpret your blood pressure data.

Maintenance

Do not open the device. It uses delicate electrical components and an intricate air unit that could be damaged. If you cannot fix the problem using the troubleshooting instructions, contact the authorized dealer in your area or our customer service department. The A&D customer service will provide technical information, spare parts and units to authorized dealers.

Troubleshooting

Problem	Possible Reason	Recommended Action
Nothing appears in the display, even when the device is turned on.	Batteries are drained.	Replace all batteries with new ones.
	Battery terminals are not in the correct position.	Reinstall the batteries with negative and positive terminals matching those indicated in the battery compartment.
The cuff does not inflate.	Battery voltage is too low.  (LOW BATTERY mark) blinks. If the batteries are drained completely, the mark does not appear.	Replace all batteries with new ones.
The device does not measure. Readings are too high or too low.	The cuff is not applied properly.	Apply the cuff correctly.
	You moved your wrist or body during the measurement.	Make sure you remain very still and quiet during the measurement.
	The cuff position is not correct.	Sit comfortably and still. Raise your hand so that the cuff is at the same level as your heart.
	_____	If you have a very weak or irregular heart beat, the device may have difficulty in determining your blood pressure.
Other	The value is different from that measured at a clinic or doctor's office.	See "Why Measure Blood Pressure at Home?".
	_____	Remove the batteries. Place them back properly and try measurement again.

Note: If the actions described above do not solve the problem, contact the dealer. Do not attempt to open or repair this product yourself, as any attempt to do so will make your warranty invalid.

Technical Data

Type	UB-543
Measurement method	Oscillometric measurement
Measurement range	Pressure: 0 – 299 mmHg Pulse: 40 – 180 beats / minute
Measurement accuracy	Pressure: ± 3 mmHg Pulse: $\pm 5\%$
Power supply	2 x 1.5V alkaline batteries (LR03 or AAA)
Number of measurements	Approx. 180 measurements, when AAA alkaline batteries are used, with pressure value of 170 mmHg at room temperature of 23°C.
Wrist circumference	5.3" – 8.5" (13.5 – 21.5 cm)
Classification	Internally powered ME equipment (Continuous operation mode)
Applied part	Cuff Type BF 
Useful life	Device: 5 years (when used six times a day)
Clinical test	According to ANSI / AAMI SP-10 1992
EMC	IEC 60601-1-2: 2007
Memory	Last 60 measurements each for user 1 and user 2.
Operating conditions	+50°F to +104°F (10°C to 40°C) / 15% RH to 85 %RH/ 800 hPa to 1060 hPa
Transport / storage conditions	-4°F to +140°F (-20°C to 60°C) / 10%RH to 95 %RH
Dimensions	Approx. 2.2" (5.6 cm) [W] x 3.5" (8.8 cm) [H] x 0.7" (1.8 cm) [D]
Weight	Approx. 3.6 oz (103 g), not including batteries
Ingress protection	IP20

Note: Specifications are subject to change for improvement without prior notice.

WARRANTY INFORMATION

LIMITED 5 YEAR WARRANTY A&D Medical Blood Pressure Monitor

Only for Retail Purchasers within the US. [For retail purchasers outside the US, NO warranty is provided by A&D – please contact your local distributor or dealer.]

A&D Medical (“A&D”) warrants to the first retail purchaser (“You”) that this blood pressure monitor (the “Product”) will be free from defects in material and workmanship for 5 years under normal use. If the Product contains a blood pressure cuff, the warranty period for such cuff is limited to two years from the date You purchased the Product. This Limited Warranty is personal to You and is not transferable. If a Product is defective, You can return the Product to the retailer You purchased it from (if within such retailer’s return time frame) or return it to A&D in accordance with the procedure set forth below. A&D’s warranty obligation is limited to the repair or replacement, at A&D’s option, of the defective Product that has been returned by You within the warranty period. Such repair or replacement will be at no charge to You. The repaired or replacement Product is warranted hereunder for the longer of the remainder of the original warranty period or 90 days from the date of shipment of the repaired or replacement Product. If you return the Product for warranty service to A&D, You must return the Product, freight and insurance prepaid, within the warranty period to the address set forth below, together with satisfactory proof of the date of Your purchase (such as a sales receipt or statement of online warranty registration). Also, please enclose a check for return shipping and insurance of the Product back to You.

In the United States, to:
A&D Medical
Attn.: Warranty Department
1756 Automation Parkway
San Jose, CA 95131
1-888-726-9966

Please contact in the US: A&D Medical, at the numbers set forth above for the appropriate shipping costs.

This Limited Warranty does not cover, and A&D will not be liable for (i) any shipment damage, (ii) any damage or defect due to misuse, abuse, failure to use reasonable care, failure to follow written instructions enclosed with the Product, accident, subjecting the Product to any voltage other than the specified voltage, improper environmental conditions, or modification, alteration or repair by anyone other than A&D or persons authorized by A&D, or (iii) expendable or consumable components.

THIS LIMITED WARRANTY IS THE ONLY WARRANTY FROM A&D; THERE ARE NO OTHER EXPRESS WARRANTIES. If A&D cannot reasonably repair or replace the Product, A&D will refund the amount You paid for the Product (not including taxes), less a reasonable charge for usage. To receive a refund you must have returned the Product and all associated materials to A&D. The above remedy of repair, replacement or refund is your only and exclusive remedy. IN NO EVENT SHALL A&D BE LIABLE FOR ANY DAMAGES, INCLUDING, WITHOUT LIMITATION, LOST PROFITS, LOST INFORMATION OR REPLACEMENT COSTS, ARISING OUT OF YOUR USE OF OR INABILITY TO USE THE PRODUCT, INCLUDING, WITHOUT LIMITATION, ANY SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES, EVEN IF A&D HAS BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. Some states do not allow the exclusion of incidental or consequential damages, so that the above exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that may vary from state to state.

No distributor, dealer or other party is authorized to make any warranty on behalf of A&D or to modify this warranty, or to assume for A&D any liability with respect to its products.

CONTACT INFORMATION

For more information regarding use, care or servicing of your blood pressure monitor, contact:

A&D Medical

A division of A&D Engineering, Inc.

1756 Automation Parkway

San Jose, CA 95131

A&D Medical Customer Service

Toll-Free: 1-888-726-9966

Visit our web site at www.andmedical.com for warranty information and registration.

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A&D Company, Limited
1756 Automation Parkway, San Jose, California 95131 U.S.A.