AEG

BMG 4907

Bedienungsanleitung/Garantie

Gebruiksaanwijzing/Garantie • Istruzioni per l'uso/Garanzia
Instrucciones de servicio/Garantía • Manual de instruções/Garantia
Instruction Manual/Guarantee • Bruksanvisning/Garanti
Εγχειρήδιο λειτουργίας / Εγγύηση • Instrukcja obsługi/Gwarancja
Návod k použití/Záruka • A használati utasítás/garancia
Руководство по эксплуатации/Гарантия

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

Bloeddrukmeter • Sfigmomanometro • Tensiómetro arterial • Tonómetro
Blood pressure gauge • Blodtryckmålapparat
Συσκευή μέτρησης της πίεσης • Urządzenie do pomiaru ciśnienia krwi
Vérnyomásmérő készülék • Přístroj měření krevního tlaku • Тонометр

Inhalt

Inhoud · Indice · Índice · Indice · Contents · Innhold
Περιεχόμενο · Spis treści · Obsah · Tartalom · Содержание

D	Übersicht Bedienelemente	Seite	3
	Bedienungsanleitung	Seite	4
	Garantie	Seite	11
NL	Overzicht bedieningselementen	Pagina	3
	Gebruiksaanwijzing	Pagina	13
	Garantie	Pagina	20
I	Descrizione dei singoli pezzi	Pagina	3
	Istruzioni per l'uso	Pagina	21
	Garanzia	Pagina	28
E	Vista de conjunto Elementos de mando	Página	3
	Instrucciones de servicio	Página	30
	Garantía	Página	37
P	Vista geral dos elemntos de comando	Página	3
	Manual de instruções	Página	39
	Garantia	Página	46
GB	Control element overview	Page	3
	Instruction Manual	Page	48
	Guarantee	Page	55
N	Oversikt over betjeningselementer	Side	3
	Bruksanvisning	Side	57
	Garanti	Side	64
GR	Περιεχόμενο αντικείμενα χειρισμού	Σελίδα	3
	Οδηγίες λειτουργίας	Σελίδα	65
	Εγγύηση	Σελίδα	72
PL	Przegląd elementów obsługi	Strona	3
	Instrukcja obsługi	Strona	74
	Gwarancja	Strona	81
CZ	Přehled obsluhovacích prvků	Strana	3
	Návod k použití	Strana	83
	Záruka	Strana	90
H	A használt elemek megtekintése	Oldal	3
	A hasznalati utasítás	Oldal	91
	Garancia	Oldal	98
RUS	Обзор управляющих элементов	стр.	3
	Руководство по эксплуатации	стр.	99
	Гарантия	стр.	106

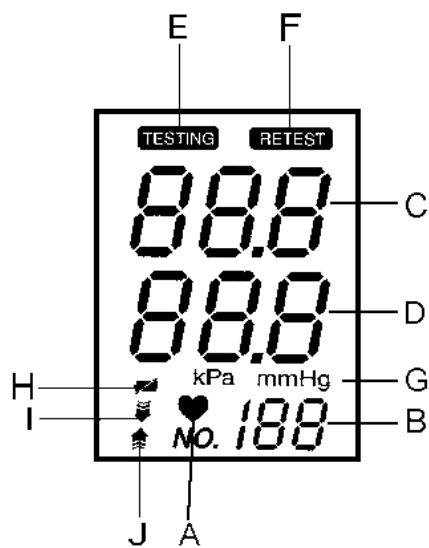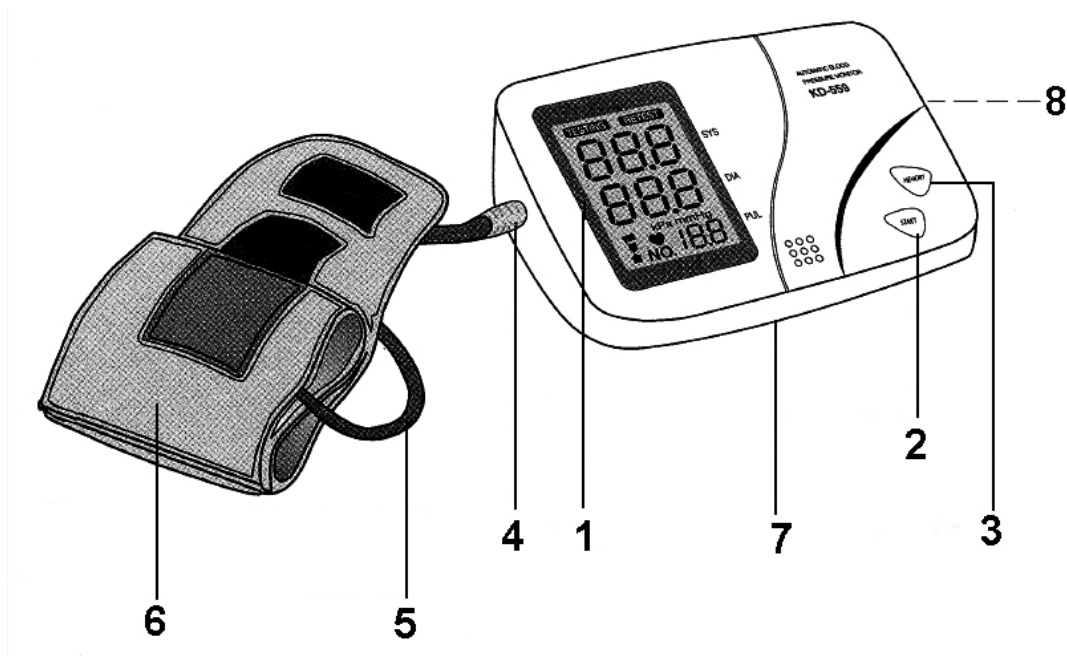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General Safety Instructions

- Read the operating instructions carefully before putting the appliance into operation and keep the instructions including the warranty, the receipt and, if possible, the box with the internal packing.
- The appliance is designed exclusively for private use and for the envisaged purpose. This appliance is not fit for commercial use. Do not use it outdoors (except if it is designed to be used outdoors). Keep it away from sources of heat, direct sunlight, humidity (never dip it into any liquid) and sharp edges. Do not use the appliance with wet hands. If the appliance is humid or wet, unplug it immediately. Do not put it in water.
- When cleaning or putting it away, switch off the appliance and always pull out the plug from the socket (pull on the plug, not the cable) if the appliance is not being used and remove the attached accessories.
- Do not operate the machine without supervision. If you should leave the workplace, always switch the machine off or remove the plug from the socket (pull the plug itself, not the lead).
- To protect children from the dangers posed by electrical appliances, make sure that the cable is hanging low and that children do not have access to the appliance.
- Check the appliance and the cable for damage on a regular basis. Do not use the appliance if it is damaged.
- Do not try to repair the appliance on your own. Always contact an authorized technician. To avoid the exposure to danger, always have a faulty cable be replaced only by the manufacturer, by our customer service or by a qualified person and with a cable of the same type.
- Use only original spare parts.
- Pay careful attention to the following "Special Safety Instructions".

Safety symbols on the type plate

Carefully read the Operating Instructions before use and keep them!

Application of Type BF

What is blood pressure?

The blood pressure is the pressure the blood is exerting on the walls of the arteries. The blood pressure in the arteries is subject to continuous variation during a heart cycle. The highest pressure within this cycle is called 'systolic pressure' (SYS), the lowest pressure is called 'diastolic pressure' (DIA). With these two blood pressure values, a doctor can determine a patient's blood pressure. Your blood pressure may be influenced by many different factors


as physical effort, anxiety or the time of the day etc. Generally, blood pressure is low in the morning and raises in the afternoon towards the evening. In summer, blood pressure is lower than in winter.

Why is it a good thing to measure blood pressure at home

Having one's blood pressure measured by a doctor in a hospital or clinic environment tends to stimulate nervousness in a patient and may lead to a higher blood pressure. Blood pressure varies in accordance with a variety of conditions and so judgment is not always possible on the basis of a single measurement. Blood pressure measured first thing in the morning after getting up, before eating, and keeping still, is known as the fundamental blood pressure. In practice it is rather difficult to record the fundamental blood pressure but measurement at home in the right environment aids this.


A. Blood pressure classification of the WHO

The standards for the determination of high or low blood pressure independently from the patient's age – as shown in the table – were determined by the World Health Organization (WHO).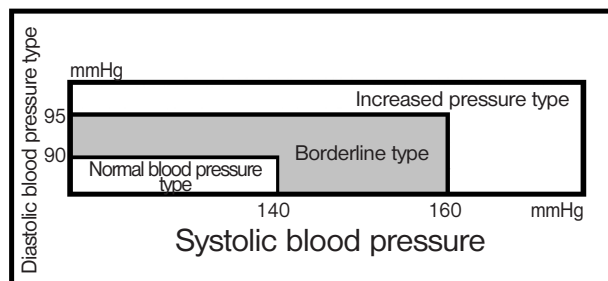

B. Variations of blood pressure

Individual blood pressure is subject to variations on a day-to-day basis or in relation to the time of the year. These variations are more perceptible in patients with high blood pressure. Usually, blood pressure is higher during work and is lowest in the night when sleeping. The figure to the right shows the variations of the blood pressure in the course of a day with measurements taken every 5 minutes. The bold line represents the sleep phase. The increase in blood pressure at 4 pm (shown as letter 'A' in the figure) and at midnight (shown as letter 'B' in the figure) mark pain events of the patient.

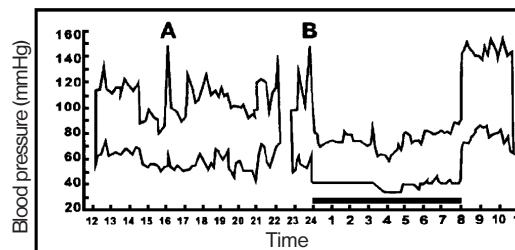Please note the following before using the device:

- Blood pressure should be judged by a medical doctor or other medical specialist who is familiar with your case history. Through the regular use of the blood pressure gauge you can continuously record your pressure values. These records can be used for therapeutic purposes.
- Tightly wrap the cuff around your upper arm. The cuff must be positioned on the level of the heart.
- The monitor must not be moved or shaken during measuring as this would lead to inaccurate results.
- Carry out measuring in a still and relaxed position.
- Do not wrap the cuff around the sleeve of a jacket, shirt or the like as this would make measuring impossible.
- Keep in mind that blood pressure naturally varies in the course of the day and is influenced by many different factors as, e.g., smoking, alcohol, drugs and physical effort. We recommend not to smoke, eat or do physical exercises for about 30 minutes before measuring.
- In people with a condition leading to circulatory disorders (diabetes, kidney diseases, hardening of the arteries), the values measured with this device may be lower than those measured with a gauge taking the blood pressure at the upper arm. Consult your doctor to make sure that measuring blood pressure at your upper arm leads to accurate measuring values.
- The measuring values received with this gauge correspond to the values received by a skilled expert taking measurements with a cuff or through the use of a stethoscope and are within the limits defined by the national standards for electronic or automated sphygmomanometers of the United States of America.
- The measuring device is designed for the measuring of blood pressure in adults only.
- Do not use the device if your upper arm is injured.
- If the automatic inflation function of the cuff does not stop automatically it must be opened and taken off immediately!

Note

User group: This device is not suitable for invasive blood pressure measuring in adults or children. It may only be used for non-invasive blood pressure monitoring and only in adults!

The following table shows the standard blood pressure values in relation to the age and sex:

Sex Age	Men		Women	
	Systole	Diastole	Systole	Diastole
11-15	114	72	109	70
16-20	115	73	110	70
21-25	115	73	110	71
26-30	115	75	112	73
31-35	117	76	114	74
36-40	120	80	116	77
41-45	124	81	122	78
46-50	128	81	122	78
51-55	134	84	134	80
56-60	137	84	139	82
61-65	148	86	145	83

Description of the device

A. Denomination of the components

1. Display
2. START button
3. MEMORY button
4. Connection
5. Rubber hose
6. Cuff
7. Battery compartment
8. 6V DC jack for mains adaptor

B. Description of displayed symbols

- A. Pulse symbol
- B. Pulse/measured
- C. SYS systole
- D. DIA diastole
- E. TESTING symbol
- F. RETEST symbol
- G. Blood pressure unit
- H. Battery symbol
- I. Pressure off
- J. Pressure on

Putting in the batteries

Remove the cover of the battery compartment (7) on the lower side of the measuring device and put the batteries into the battery compartment as shown. Put in 4 batteries of type 1.5V, type AA. Pay particular attention to the correct positions of the + and - poles of the batteries. Close the battery compartment.

A battery symbol (H) on the display (1) indicates that the batteries should be replaced. Always replace the whole battery set, never replace only one battery.

Attention: Batteries do not belong in your usual household rubbish. Please dispose of old batteries at proper collecting points or at the competent dealer. Never throw batteries into a fire.


Operation with mains adaptor

You can optionally operate the device over a mains adaptor (not included).

1. Put the plug of the mains adaptor into a properly installed shockproof socket (230V, 50Hz).
2. Connect the device to the mains adaptor by putting the plug of the mains adaptor into the 6V DC jack (8) of the device.
3. When using a mains adaptor (6V DC) take care to apply the correct polarisation according to the labels on the device $\ominus$ (•) $\oplus$.


A. Wrap the cuff around the upper arm.

1. Open the Velcro fastener and pull to open the cuff (6).
2. Lead your arm through the cuff and pull the cuff up to a position 1-2 cm above your elbow.
Suitable for a perimeter of the upper arm of about 22-34 cm.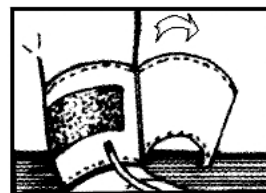

B. Pull the cuff tight around the upper arm.

1. Do not pull on the cuff with exaggerated force.
2. Do not pull the cuff too tight, leave a gap of about the width of a finger.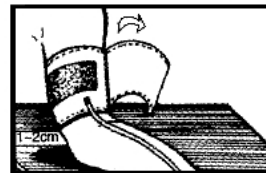


Correct body position for measuring

To receive best possible measuring results make sure that the cuff is held on the same level as your heart. If the cuff is held in a position above or below the heart this may lead to deviations from the exact value.

Measuring blood pressure in sitting position

1. Rest your elbow on a table or other surface (as, e.g., a bag).
2. Position your arm on the rest in a way that makes sure that the cuff is on the same level with your heart.
3. Relax your arm; the palm of your hand must point upwards.

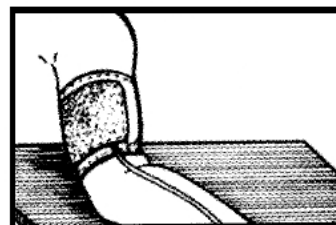Description of the symbols on the display

The ♥ symbol is shown on the display during the measuring cycle and flashes as soon as the pulse has been detected. Do not move or talk during the measuring cycle.




The word **TESTING** is displayed and the ♥ flashes over the whole measuring cycle until measurement is completed.

Measuring error: EE and **RETEST** will be displayed if the blood pressure could not be measured exactly. Shortly push and hold the START button (2) two times and measure again. The cuff must be wrapped around the upper arm as described in the Operating Instructions. Relax your hand. Do not move or talk during measuring. Pay attention to the correct posture.

Measuring instructions

A. Measuring blood pressure

- a. Put the device on a solid and level surface.
- b. Connect the cuff, i.e., the rubber hose (5) to the connection (4).
- c. Wrap the cuff around your upper arm (see section 'Putting the cuff around the upper arm').
- d. Sit down on a chair and take up an upright position (see section 'Correct body position for measuring').
- e. Push the START button and hold it down for about 1 second. The number '888' will be shown on the display for 3 seconds. Then, the display will show the value '0' (Fig. A). Air pressure is automatically increased to 190 mmHg. In case of a tendency to high blood pressure, the air pressure is automatically increased to the required level (up to 300 mmHg). Do not move or talk during the measuring cycle.
- f. After the air pressure has been increased it is slowly let out. As soon as the pulse has been detected, the ♥ symbol on the display starts to flash (Fig. B).
- g. After the measuring cycle has been completed, the values of the systolic, diastolic pressures and the pulse rate are shown on the display (Fig. C). After about 3 minutes or pushing the START button, the device switches off automatically.
- h. If you want to repeat the measurement push the START button two times for about 1 second.

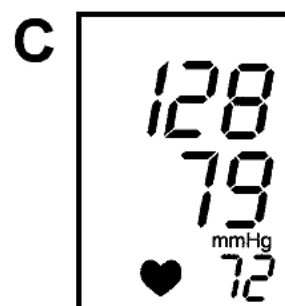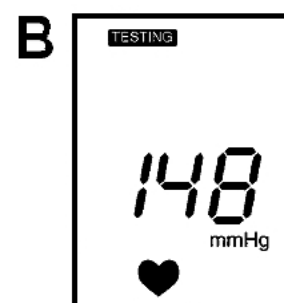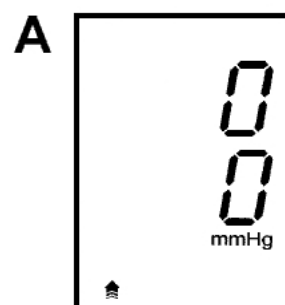B. Save, retrieve and delete measuring data

a. Save data

After each measurement of the blood pressure, the data of the systolic/diastolic pressures and heart rate of the respective measuring cycle are automatically saved. The last 90 measuring results can be retrieved from the device's memory. If the number of measuring cycles exceeds 90, the oldest data in the memory are automatically deleted.

b. Retrieve data

- (1) **Only** after a measuring cycle, you can use the – or + buttons to retrieve measuring data saved at an earlier date (if there are any) using the MEMORY button (3). When retrieving data, the display will first show the number (NO.) of the measurement, and after about 3 seconds the measuring data. The most recent measurement is measurement NO. 1.
- (2) To select the next line of saved measuring data repeatedly push the MEMORY button. The retrieved measuring data shown on the display are numbered accordingly. After reading the stored measuring data push the START button to switch off the device.

c. Delete data

- (1) To delete data (**only** after a measuring cycle) hold the MEMORY button pushed down. The display will show '—'.
- (2) To verify whether the data has been deleted push the MEMORY buttons after a measuring cycle. The display will only show the result of the most current measurement.

Maintenance

- To clean the housing of the device use a cloth moistened with water or a soft cleaning agent, then rub dry with a dry cloth. Use a dry cloth to clean the cuff if it should be dirty.
- Do not immerse the device in water.
- Do not use gas or strong cleaning agents.
- If the measuring device is not used for a longer time, the batteries should be removed. (Leaking batteries may damage the device.)

Storing in the bag

- Always store the cuff in the provided bag after use.
- Protect the measuring device with its accessories from direct sunlight, high temperatures, humidity and dust.
- Do not expose the device to extremely low (below –20 °C (–4.0 °F)) or high (above 65°C (122.0 °F)) temperatures.

Prevention of malfunction

- The cuff must not be twisted in any way.
- Do not pump air into the cuff if it is not wrapped around the upper arm.
- Do not attempt to disassemble the measuring device or to modify components of the monitor and cuff.
- Do not let the device fall down and avoid heavy impacts.

Guarantee

The device supplied by our Company is covered by a 24 month guarantee starting on the date of purchase (receipt).

During the life of the guarantee any fault of the device or its accessories *) ascribable to material or manufacturing defects will be eliminated free of charge by repairing or, at our discretion, by replacing it. The guarantee services do not entail an extension of the life of the guarantee nor do they give rise to any right to a new guarantee!

Proof of the guarantee is provided by the proof of purchase. Without proof of purchase no free replacement or repair will be carried out.

If you wish to make a claim under the guarantee please return the entire machine in the original packaging to your dealer together with the receipt.

*) Damage to accessories does not mean automatic free replacement of the whole machine. In such cases please contact our hotline. Broken glass or breakage of plastic parts are always subject to a charge.

Defects to consumables or parts subject to wearing (e.g. motor pistons, kneading blades, drive shafts, spare remote control, spare toothbrushes, saw blades, etc.) as well as cleaning, maintenance or the replacement of said parts are not covered by the guarantee and hence are to be paid!

The guarantee lapses in case of unauthorized tampering.

After the expiry of the guarantee

After the expiry of the guarantee repairs can be carried by the competent dealer or repair service against the payment of the ensuing costs.

Applied standards

The CE marking **CE 0197** of the measuring device corresponds to EU Directive 93/42/EEC (Medical Device Directive). The following standards apply for the design and/or production of the product:

EN 1060: Margin of error for the measurement readings of the cuff pressure
EN 55022:1998, EN 55024:1998, EN 60601-4-2/-3: Medical electrical equipment – electromagnetic compatibility

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Barbara Alt, Bredmed Italia S.r.l., Via Alviano no 4/1, 34170 GORIZIA (ITALY)