beurer BC 20

(D) Gebrauchsanleitung	
Blutdruckmessgerät	(2-9)
(GB) Instruction for Use	
Blood pressure monitor	(10-16)
(F) Mode d'emploi	
Tensiomètre	(17-24)
(E) Instrucciones para el uso	
Tensiómetro	(25-32)
(I) Istruzioni per l'uso	
Misuratore di pressione	(33-40)
(TR) Kullanma Talimatı	
Tansiyon Ölçme Aleti	(41-47)
(RUS) Инструкция по применению	
Прибор для измерения артериального давления на запястье	(48-56)
(PL) Instrukcja obsługi	
Ciśnieniomierz	(57-64)

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Dear Customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for the applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air.

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Best regards,
Your Beurer Team

1. Familiarization

The wrist blood pressure monitor is used for non-invasive measurement and monitoring of adults' arterial blood pressure.

You can use it to measure your blood pressure quickly and easily, storing the results and displaying the progression of readings together with the average.

2. Important information

Advice on use

- Always measure your blood pressure at the same time of day, in order to ensure that values are comparable.
- Relax for approx. 5 minutes before each measurement.
- You should wait five minutes before measuring a second time.
- The readings that you take may only be used for information purposes – they are not a substitute for a medical examination! Discuss your results with the doctor. Never use them to make medical decisions independently (e.g. regarding medication and dosage thereof).
- If circulation on one arm is restricted as a result of chronic or acute vascular disease (including vasoconstriction), the precision of any wrist measurement is impaired. In this case, use a blood pressure monitor for the upper arm.
- There may be incorrect measurements where there is cardiocirculatory disease, as well as with very low blood pressure, circulatory disorders, dysrhythmia and other preexisting diseases.

- Only use the instrument on persons whose wrist has the right circumference for the instrument.
- You may only operate the blood pressure monitor with batteries. Note that it is only possible to save data when your blood pressure monitor is receiving power. As soon as the batteries wear out, the blood-pressure instrument loses the date, time and the values that have been stored.
- The automatic switch-off function switches off the blood pressure monitor in order to preserve the batteries, if no button is pressed within one minute.

Storage and Care

- The blood pressure monitor is made up of precision electronic components. Accuracy of readings and the instrument's service life depend on careful handling.
 - You should protect the device from impact, moisture, dirt, major temperature fluctuations and direct exposure to the sun's rays.
 - Never drop the device.
 - Do not use near strong electromagnetic fields, i.e. keep it away from any radio systems and mobile phones.
 - Only ever use the cuffs provided with the monitor or original replacement cuffs. Otherwise erroneous results will be recorded.
- Do not press any buttons until the cuff is in position.

- If the instrument is not used for any length of time, we recommend removing the batteries.

Advice on batteries

- Batteries can be fatal if swallowed. You should therefore store the batteries and products where they are inaccessible to small children. If a battery has been swallowed, call a doctor immediately.
- Batteries should not be charged or reactivated with any other means, nor should they be taken apart, thrown in the fire or short-circuited.
- Remove the batteries from the instrument if they are worn out or if you are not going to use the instrument for any length of time. This prevents any damage as a result of leakage. Always replace all the batteries at the same time.
- Never use different types of battery, battery brands or batteries with different capacities. You should preferably use alkaline batteries.

Repair and disposal

- Batteries do not belong in domestic refuse. Used batteries should be disposed of at the collection points provided.
- Never open the instrument. If these instructions are not heeded, the warranty will be null and void.

- Never attempt to repair the instrument or adjust it yourself. We can no longer guarantee perfect functioning if you do.
- Repairs may only be performed by Beurer Customer Service or authorized dealers. However, always check the batteries and replace them if necessary prior to making any complaint.
- The appliance should be disposed of according to Regulation 2002/96/EC-WEEE (Waste Electrical and Electronic Equipment). In case of queries, please contact the municipal authorities responsible for waste disposal in your area.


3. Unit description

- ① Display
- ② Battery compartment cover
- ③ Memory button **M**
- ④ Start/Stop button

Icons in the display:

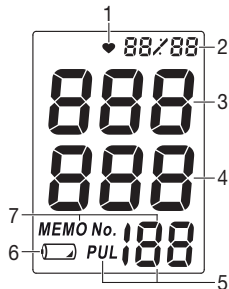- 1. Pulse symbol
- 2. Time and date
- 3. Systolic pressure
- 4. Diastolic pressure
- 5. Measured pulse
- 6. „Change battery“ icon 
- 7. Memory space number

4. Inserting/changing batteries

- Remove the battery compartment cover ②.
- Use exclusively brand-name batteries of the type: 2x 1.5V Micro (Alkaline Type LR03). Making absolutely sure that you insert them with the correct polarity as marked. Do not use rechargeable batteries.
- Then close the battery compartment by fitting the cover and clicking it in position.

When the batteries are almost exhausted, an empty battery symbol  appears at bottom left in the display ① and the batteries must then be exchanged.

Once the batteries have been removed from the device, the date and time must be set again. The saved measurements are lost.

They are considered to be toxic waste and, as such, should be disposed of in special containers, toxic waste collection points or brought to electrical goods dealers. You are legally obliged to dispose of the batteries.

Note: You will find these markings on batteries containing harmful substances: Pb = battery containing lead, Cd = battery containing cadmium, Hg = battery containing mercury.

5. Setting date/time

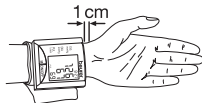
The date/time setting is necessary, so that you can save your readings correctly with the date and time and call them up again later.

To set the date and time, proceed as follows:

- After inserting the batteries, press the Start/Stop button **4**. The year display will start to flash. Use the memory button **M** **3** to set the year and confirm using the Start/Stop button **4**.
- Then set the month, day, hour and minutes and confirm each setting with the Start/Stop button **4**.

6. Applying the blood pressure computer

- Apply the blood pressure computer to the inside of the left wrist.
- Ensure that the wrist is unclothed.
- Close the cuff with the Velcro fastener, so that the top margin of the computer is about 10 mm below the ball of the thumb (see illustration).
- The cuff must be placed tightly around the wrist but must not constrict.

7. Measuring blood pressure

- Sit in a relaxed position.
- Ensure that your arm is supported and bend it so that it is at the level of your heart (see illustration).
- Before measurement allow an adequate relaxation time of 3–5 minutes.
- Now press the Start/Stop button **4** to start the measurement.
- Following the display check, during which all numbers light up, the cuff automatically pumps up to 180 mmHg.

- When the 180 mmHg pressure is reached, the pressure is automatically released from the cuff and a beating heart appears in the display.
- Your blood pressure readings and pulse rate will then appear in the display.
- To switch off and release the pressure, press the Start/Stop button **4** again.
- If you forget to switch off the computer, it will switch off automatically after 3 minutes.
- You can interrupt measurement at any time by pressing the Start/Stop button **4**.

8. Display of the readings saved

The computer automatically saves the blood pressure readings of the last 60 measurements.

When the 60 memory locations are exceeded, the current oldest value is deleted.

You can display the memory as follows:

- Press the memory button **M 3** with the computer switched off. The last reading measured appears.
- You can call up all individual readings saved by pressing the memory button **M** repeatedly.
- The memory location number is shown in the display at bottom right. The date and time of the individual measurement are shown in the top part of the display.

- The last result obtained is stored in the first memory location. As soon as a new result is obtained, this appears in the first memory location and the older result moves down to the next location.
- After the last memory location occupied, the computer shows the average of all readings saved on pressing the memory button **M 3** again.
- To switch off, press the Start/Stop button **4** again.
- If you forget to switch off the computer, it will switch off automatically after 3 minutes.

9. Rectifying faults

Error messages can occur when

- cuff inflation takes longer than 25 seconds (**Err2** appears in the display),
- you move during the measurement instead of relaxing quietly (**Err1** appears in the display),
- insufficient pressure is reached despite extra inflation (**Err2** appears in the display),
- the blood pressure readings are extremely high or low (**Err1** appears in the display),
- the inflation pressure is higher than 300 mmHg (error message **Err330**),
- the batteries are almost exhausted, the batteries must then be exchanged.

In the about cases, you must repeat the measurement. Put the batteries back in if necessary, or else replace them.

10. Cleaning and Care

- Clean your blood pressure monitor carefully using a slightly damp cloth only.

- Do not use any detergents or solvents.
- Never hold the instrument under water as otherwise liquid can penetrate and damage the instrument.
- Never place any heavy objects on the instrument.

11. Table for classification of blood pressure values (unit of measurement mmHg) for adults:

Range	Systolic	Diastolic	Action
Hypotonia (degraded blood pressure)	lower than 105	lower than 60	Check with doctor
Normal range	between 105 and 120	between 60 and 80	Self check
Pre-hypertensive blood pressure range*	between 120 and 140	between 80 and 90	Check with doctor
High blood pressure – degree I	between 140 and 160	between 90 and 100	Consult your doctor
High blood pressure – degree II	higher than 160	higher than 100	Consult your doctor

(*) Blood pressure range that might transfer to high blood pressure Adapted from JNC 2003

12. Technical data

Type:	BC 20
Measuring method:	non-invasive, oscillometric blood pressure measurement at the wrist, for wrist circumferences of 12.5–20.5 cm
Display range:	0–300 mmHg
Measuring range:	systolic: 50–250 mmHg/ diastolic: 40–180 mmHg/ pulse: 40–160/min
Accuracy of the pressure reading:	systolic: ± 3 mmHg/ diastolic: ± 3 mmHg/ pulse: $\pm 5\%$ of the displayed value
Measuring uncertainty:	(max. permissible standard deviation according to clinical testing in accordance): systolic: 8 mmHg/ diastolic: 8 mmHg
Power supply:	2 x 1.5V Micro (Alkaline Type LR03)
Classification	Application part Type BF
Ambient conditions:	+10°C – +40°C max. 85% atmospheric humidity
Storage/transport:	-5°C – +50°C max. 85% atmospheric humidity

Key to symbols: Application part Type BF 

Important!

Read the operating instructions! 

- This unit is in line with European Standard EN60601-1-2 and is subject to particular precautions with regard to electromagnetic compatibility (EMC). Please note that portable and mobile HF communication systems may interfere with this instrument. More details can be requested from the stated Customer Service address or found at the end of the instructions for use.
- The instrument is in line with the EU Medical Products Directive 93/42/EC, the German medical products act and European standards EN1060-1 (non-invasive blood pressure monitors, Part 1: General Requirements) and EN1060-3 (non-invasive blood pressure monitors, Part 3: Supplementary Requirements for Electromechanical Blood Pressure Measurement Systems).

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Electromagnetic Compatibility Information

Guidance and manufacturer's declaration – electromagnetic emissions – for all EQUIPMENT and SYSTEMS

Guidance and manufacturer's declaration – electromagnetic emission

The BC 20 Blood Pressure Monitor is intended for use in the electromagnetic environment specified below. The customer or the user of the BC 20 Blood Pressure Monitor should assure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The BC 20 Blood Pressure Monitor uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emission CISPR 11	Class B	The BC 20 Blood Pressure Monitor is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	N/A	
Voltage fluctuations/flicker emissions IEC 61000-3-3	N/A	

Guidance and manufacture's declaration – electromagnetic immunity – for all EQUIPMENT and SYSTEMS

Guidance and manufacture's declaration – electromagnetic immunity

The BC 20 Blood Pressure Monitor is intended for use in the electromagnetic environment specified below. The customer or the user of BC 20 Blood Pressure Monitor should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floor are covered with synthetic material, the relative humidity should be at least 30%.
Power frequency (50/60Hz) magnetic field IEC61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields Should be at levels characteristic of a typical location in a typical commercial or hospital environment.

NOTE: U_T is the a.c. mains voltage prior to application of the test level.

Guidance and manufacturer's declaration – electromagnetic immunity – for EQUIPMENT and SYSTEMS that are not LIFE-SUPPORTING

Guidance and manufacturer's declaration – electromagnetic immunity

The BC 20 Blood Pressure Monitor is intended for use in the electromagnetic environment specified below. The customer or the user of BC 20 Blood Pressure Monitor should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	Portable and mobile RF communications equipment should be used no closer to any part of the BC 20 Blood Pressure Monitor, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: $d = \left[\frac{3.5}{V_1} \right] \sqrt{P}$ $d = \left[\frac{3.5}{E_1} \right] \sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = \left[\frac{7}{E_1} \right] \sqrt{P} \quad 800 \text{ MHz to } 2.5 \text{ GHz}$ Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, a should be less than the compliance level in each frequency range. b Interference may occur in the vicinity of equipment marked with the following symbol: 

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the BC 20 Blood Pressure Monitor is used exceeds the applicable RF compliance level above, the BC 20 Blood Pressure Monitor should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the BC 20 Blood Pressure Monitor.

Recommended separation distances between portable and mobile RF communications equipment and the EQUIPMENT or SYSTEM – for EQUIPMENT or SYSTEM that are not LIFE-SUPPORTING

Recommended separation distances between portable and mobile RF communications equipment and the CMS50I PULSE OXIMETER

The BC 20 Blood Pressure Monitor is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the BC 20 Blood Pressure Monitor can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the BC 20 Blood Pressure Monitor as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)		
		80 MHz to 800 MHz $d = \left[\frac{3.5}{E_1} \right] \sqrt{P}$	800 MHz to 2.5 GHz $d = \left[\frac{7}{E_1} \right] \sqrt{P}$
0.01		0.12	0.23
0.1		0.37	0.74
1		1.17	2.33
10		3.69	7.38
100		11.67	23.33

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.