- (D) Herzfrequenzmessung mit Smartphones**  
Gebrauchsanweisung .....2-11
- (GB) Heart rate measurement with smartphones**  
Instructions for use .....12-20

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### 1 Sendeeinheit

inkl. Lithium-Knopfbatterie (3V, CR2032)

### 1 Brustgurt

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*Runtastic / Runtastic Road Bike / Runtastic Mountain Bike / Runtastic Wintersports / Endomodo Sports Tracker / Wahoo Fitness / RunKeeper / MapMyFitness / MapMyRide / MapMyHike / MapMyRun / MapMyWalk / Runalyzer Utility / Strava Cycling & Run / 60 beat / 3,2,1 Run Free / Wahoo Beats / Bit of Exercise / Kinomap Trainer / Ritmo Time Free / rubiTrack / Biky Coach CycleOps / SPLIT: multisport GPS / CoolRunner / Kinomap Maker / Seconds / Joggy Coach Free*

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Incl. lithium button cell battery (3V, CR2032)

### **1 chest strap**

(consisting of elastic tension belt and removable transmitter unit)

## IMPORTANT NOTES

Read these instructions for use carefully, store them in a safe place and make them available to other users.

### **Training**

- This product is not a medical device. It is a training instrument that has been designed for measuring and displaying the human heart rate.
- If in doubt or in the case of illness, consult your doctor on your own personal upper and lower training heart rate values, and on the length and frequency of training. This will ensure that you achieve optimum training results.
- **IMPORTANT:** People with heart and circulatory problems or wearers of pacemakers should consult their doctor before using this heart rate measurement device.
- The chest strap can contain latex components. If you experience skin irritations or redness after wearing the chest strap, stop using it and contact your dermatologist.
- This device is not intended for use by people (including children) with restricted physical, sensory or mental skills or a lack of experience and/or a lack of knowledge, unless they are supervised by a person who has responsibility for their safety or they receive instructions from this person on how to use the device.

- Children should be supervised around the device to ensure they do not play with it.

### **Intended use**

- The device is intended for private use only.
- This device may only be used for the purpose for which it is designed and in the manner specified in the instructions for use. Any improper use can be dangerous. The manufacturer is not liable for damage resulting from improper or careless use.

### **Functions and possible application**

#### **NOTE**

The heart rate is transmitted from the chest strap to the smartphone using the latest *Bluetooth*<sup>®</sup> technology. This means that the Beurer PM 235 is compatible with *Bluetooth*<sup>®</sup> smart devices and *Bluetooth*<sup>®</sup> smart ready devices.

The chest strap included in the delivery is a “combo chest strap”, which means that it emits two signals. A *Bluetooth*<sup>®</sup> signal and a further signal at a transmission frequency of 5.3 kHz. Due to the additional 5.3 kHz analogue signal, your chest strap is also compatible with most training devices in the fitness studio and heart rate monitors with analogue signal transmission (e.g. PM 62).

### **Electromagnetic interference**

- Electromagnetic interference may occur. Strong magnetic fields (e.g. transformers) must always be avoided as they may give rise to deviations in transmission. Failure to observe these notes may give rise to faults or damage to the transmitter unit.

### **Cleaning, maintenance and storage**

- Before beginning the cleaning process, disconnect the transmitter unit from the elastic chest strap.
- Clean the elastic chest strap occasionally under running water. Dry the chest strap carefully using a soft cloth.
- The elastic chest strap can be washed in the washing machine at 40 °C without fabric condi-

tioner. Then hang up the elastic chest strap to dry. Do NOT dry the elastic chest strap in the tumble dryer!

- Clean the transmitter unit occasionally using a damp cloth. Never immerse the transmitter unit in water!
- Store the chest strap and the transmitter unit separately in a clean, dry place away from direct light. Dirt can impair the elasticity of the elastic chest strap and the function of the transmitter unit. Sweat and moisture may cause the sensors on the transmitter unit to remain active, which can reduce the service life of the battery.
- Do not use any corrosive, gaseous cleaning agents or solvents.

### **Repair, accessories and disposal**

- Before use, ensure that there is no visible damage to the device or accessories. If you have any doubts, do not use the device and contact your retailer or the specified Customer Services address.
- Repairs must only be carried out by Customer Services or authorised suppliers. Never attempt to repair the device yourself!
- Only use the device with the supplied accessories.
- Avoid contact with suncream and similar substances as these can damage the printing or plastic parts.
- Never expose the device to direct sunlight.

### **Notes on handling batteries**

- If your skin or eyes come into contact with battery fluid, flush out the affected areas with water and seek medical assistance.
- Choking hazard! Small children may swallow and choke on batteries. Store the batteries out of the reach of small children.
- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Risk of explosion! Never throw batteries into a fire.

- If the device is not to be used for a long period, take the batteries out of the battery compartment.
- Do not disassemble, split or crush the batteries.

### **Battery disposal**

- The empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.
- The codes below are printed on batteries containing harmful substances:

PB = Battery contains lead,

Cd = Battery contains cadmium,

Hg = Battery contains mercury.

### **General disposal**

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the unit at a suitable local collection or recycling point.

Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

## **INSTALLATION – INITIAL USE**

### **1. Installing a fitness app**



- Open the relevant App Store on your smartphone (e.g. Apple App Store, Market or Google Play Store).
- Download the desired fitness app on to your smartphone.

### **i NOTE**

The chest strap is compatible with the most common iOS and Android fitness apps. Below is a list of fitness apps that have been tested with the chest strap and deemed to be compatible (this is not an exhaustive list).

*Runtastic / Runtastic Road Bike / Runtastic Mountain Bike / Runtastic Wintersports / Endomondo Sports Tracker / Wahoo Fitness / RunKeeper / MapMyFitness / MapMyRide / MapMyHike / MapMyRun / MapMyWalk / Runalyzer Utility / Strava Cycling & Run / 60 beat / 3,2,1 Run Free / Wahoo Beats / Bit of Exercise / Kinomap Trainer / Ritmo Time Free / rubiTrack / Biky Coach CycleOps / SPLIT: multisport GPS / CoolRunner / Kinomap Maker / Seconds / Joggy Coach Free*

## **2. Putting on the chest strap**



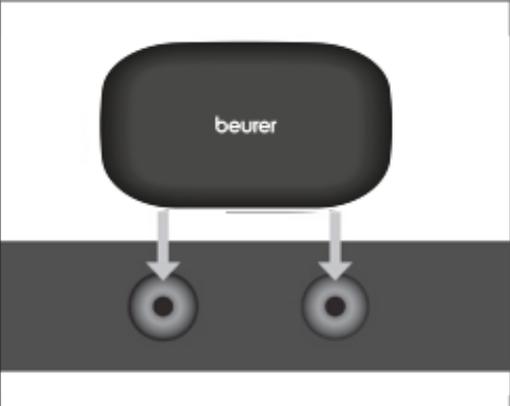
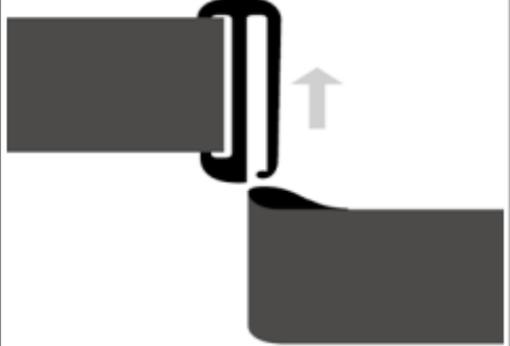
<p>Attach the transmitter unit to the chest strap using the two poppers.</p>	 A diagram showing a dark, rounded rectangular transmitter unit with the word "beurer" printed on it. Two grey arrows point downwards from the bottom of the unit to two circular poppers on a dark grey chest strap.
<p>Remove the hook from the loop on the chest strap.</p>	 A diagram showing a dark grey chest strap with a black hook and loop fastener. A grey arrow points upwards from the hook, indicating it should be lifted to separate from the loop.

Put on the chest strap so that it is positioned immediately below the pectoral muscle. Ensure that the chest strap is not too loose or too tight.

Place the hook back into the loop on the chest strap.

Position the chest strap so that the transmitter unit is facing outwards and the writing on the transmitter unit can be read from the front.

On men, the chest strap should lie immediately below the pectoral muscle as shown, on women, immediately below the bust.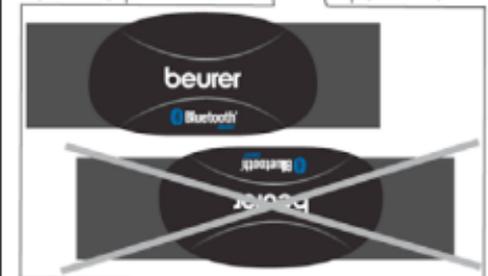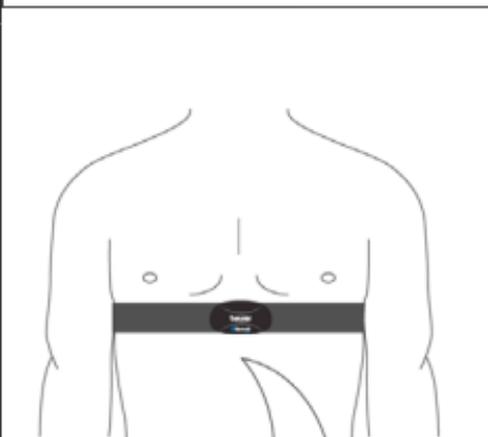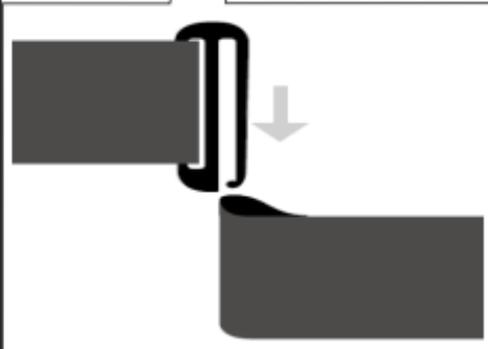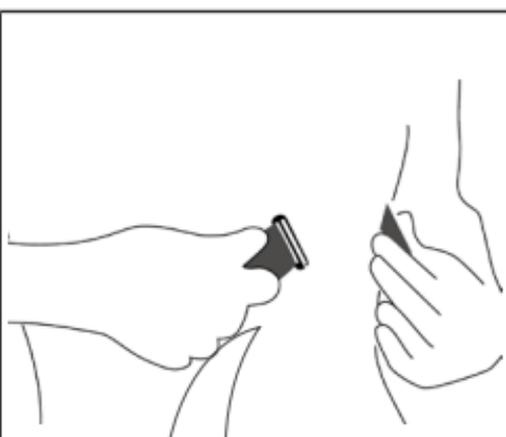

## **i** NOTE

As optimum contact between the skin and pulse sensors is not always possible immediately, it may be some time before the pulse is measured and displayed. Adjust the position of the transmitter to achieve optimum contact. The formation of sweat under the chest strap usually helps to ensure good contact. You can also moisten the contacts on the inside of the chest strap. To do this, lift the chest strap away from the skin slightly

and carefully moisten both sensors with saliva, water or ECG gel (available from pharmacies). Contact between the skin and the chest strap must not be interrupted when there is significant movement of the ribcage, such as when taking a deep breath. Chest hair may interrupt or even prevent contact.

Put on the chest strap a few minutes before commencing training to allow it to warm to body temperature and to establish optimum contact.

### **3. Activating pulse measurement**

#### **3.1 Pulse measurement with a smartphone**

- Activate the *Bluetooth*<sup>®</sup> function on your smartphone.
- Open the fitness app you would like to use to measure your heart rate.
- Activate “heart rate measurement” in the app settings.
- Now wait until the connection to the chest strap is established.

#### **NOTE**

The service life of the transmitter unit battery is approx. 1 year when used for 1 hour per day. If transmission between the transmitter unit and the chest strap ceases to function, replace the battery as detailed under “Replacing the transmitter unit battery”.

#### **3.2 Pulse measurement with training devices (e.g. in the fitness studio) and heart rate monitors with analogue signal transmission**


The pulse signal is sent via *Bluetooth*<sup>®</sup> smart technology as well as via analogue signal transmission.

#### **NOTE**


Pulse measurement only works with training devices and analogue heart rate monitors that have an uncoded transmission frequency of 5.3 kHz.

## REPLACING THE TRANSMITTER UNIT BATTERY


Open the battery compartment cover with a coin. To do so, rotate the coin in an anticlockwise direction.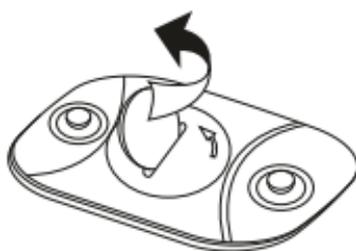

Remove the depleted button cell battery and insert a new button cell battery of the type CR2032. Ensure that the positive terminal (+) is facing upwards and the battery is positioned below the contact.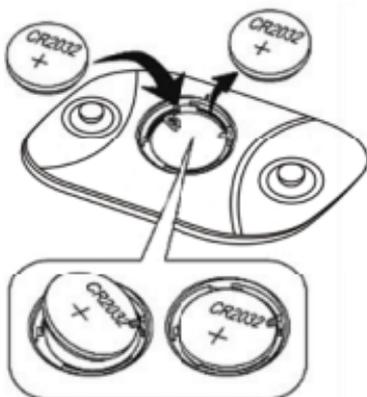

Close the battery compartment cover again. To do so, rotate the coin in a clockwise direction. Ensure that the battery compartment cover is closed properly so that no liquid can penetrate the transmitter unit.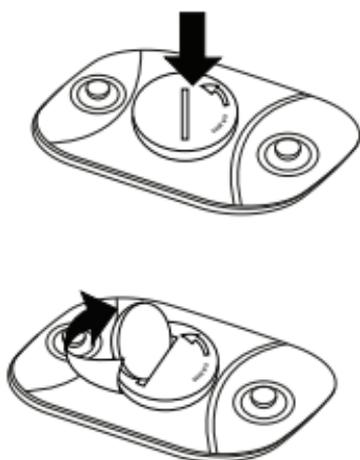

## TECHNICAL DATA

<b>Transmitter unit</b> Power supply:	1 lithium button cell battery (3V, CR2032)
Transmission frequency from the chest strap to the transmitter unit:	<i>Bluetooth</i> <sup>®</sup> 4.0 and 5.3 kHz <i>Bluetooth</i> <sup>®</sup> QD ID: B019950

### **Watertightness**

The transmitter unit and the chest strap are splash-proof. The following table indicates the conditions and activities for which the receiver and chest strap are suitable:

Drops of sweat	Showers	Baths, swimming pools	Water sports
Yes	No	No	No

We hereby guarantee that this product complies with the European EMC Directive 2004/108/EC. Please contact the specified service address to obtain further information, such as the CE Declaration of Conformity.