BRAUN

BP 2550

SensorControl

Type 6053

Braun Infoline

English 4, 13, 55

Français 14, 23, 55

Nederlands 24, 33, 55

Español 34, 43, 55

Português 44, 53, 55

(GB) Helpline
Should you require any further assistance please call Braun (UK) Consumer Relations on 0800 783 70 10

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
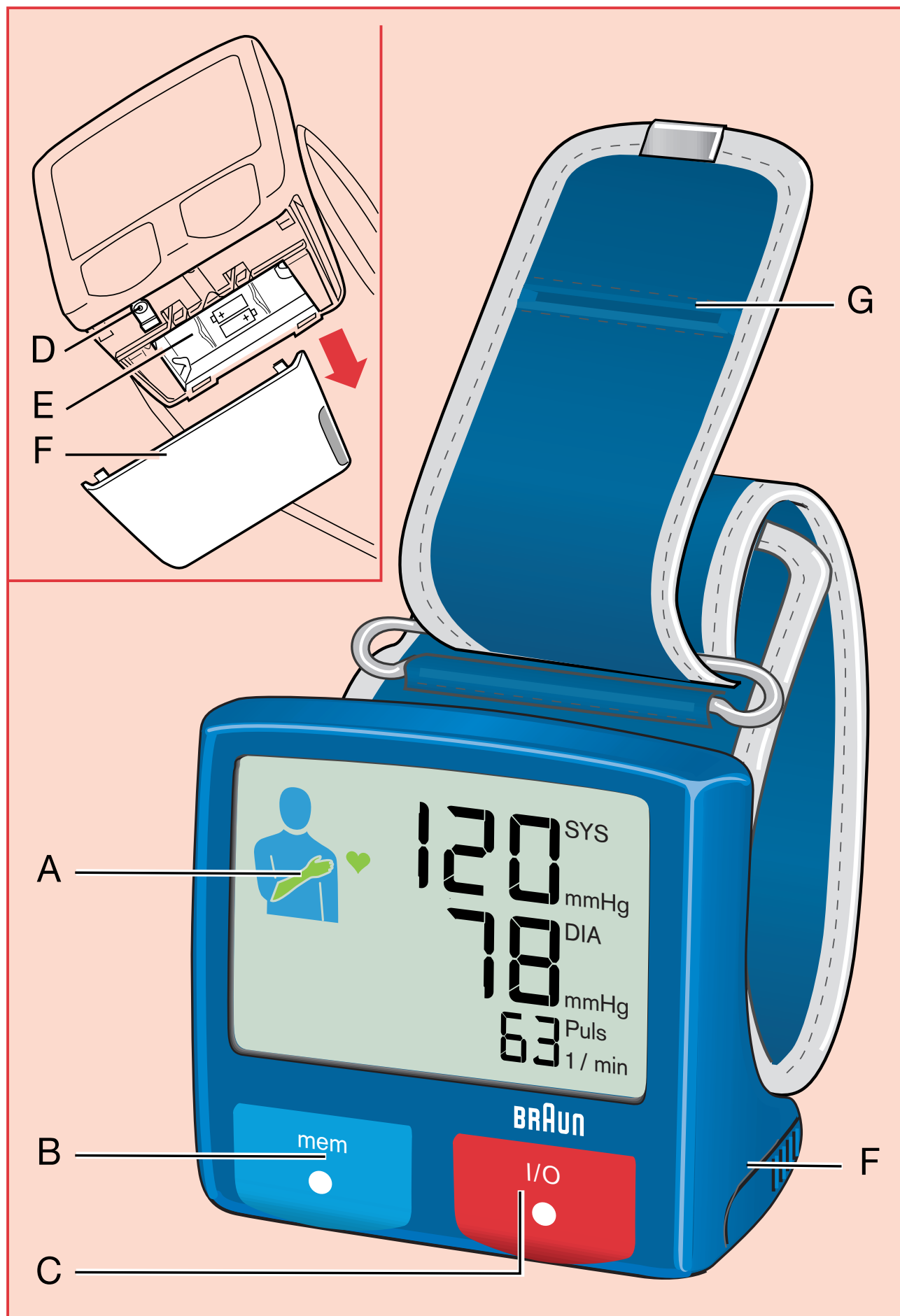
Internet:
www.braun.com

English

What you should know about blood pressure

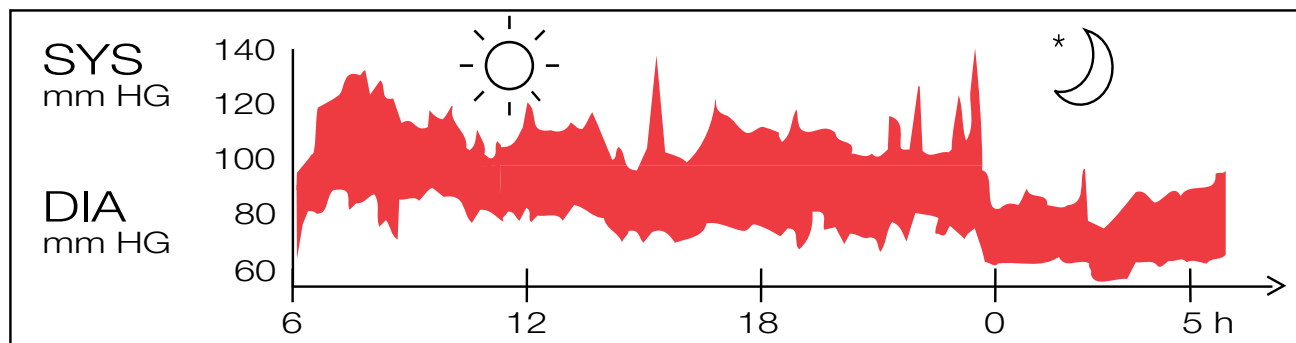Blood pressure constantly changes throughout the day. It rises sharply in the early morning and declines during the late morning. Blood pressure rises again in the afternoon and finally drops to a low level at night. Also, it may vary in a short period of time. Therefore, readings from successive measurements can fluctuate.

Blood pressure readings from a healthy 31 year old male, measured at 5 minute intervals


Blood pressure measured in a doctor's office only provides a momentary value. Repeated measurements at home better reflect one's actual blood pressure values under everyday conditions.

Moreover, many people have a different blood pressure when they measure at home, because they tend to be more relaxed than when in the doctor's office. Regular blood pressure measurements taken at home can provide your doctor with valuable information on your normal blood pressure values under actual «everyday» conditions.

The World Health Organisation (WHO) has set up the following standard blood pressure values when measured at resting pulse:

Blood pressure (mmHg)	Normal values	Mild hypertension	Severe hypertension
SYS = Systole = upper value	up to 140	140 – 180	over 180
DIA = Diastole = lower value	up to 90	90 – 110	over 110

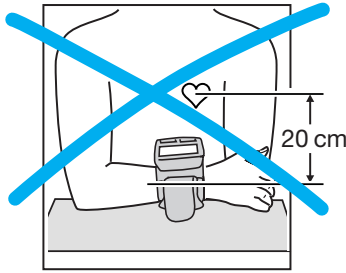What you should know about this blood pressure monitor

For accurate measurements, all wrist blood pressure monitors must be positioned at heart level.

Example: If the monitor is positioned 20 cm below heart level as shown on the left (e.g. by putting the arm on a table), the reading you will obtain is approximately 15 mmHg too high.

Therefore, the Braun SensorControl has a unique positioning system: The position sensor icon (A) on the display guides you to hold your measuring arm in the right position at heart level.

Additionally, your blood pressure monitor offers the following benefits:

- Easy handling due to large buttons and the large display.
- Automatic storage of the last 60 readings consisting of blood pressure values, pulse, date and time of measurement.
- Comfort cuff.

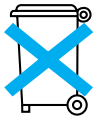
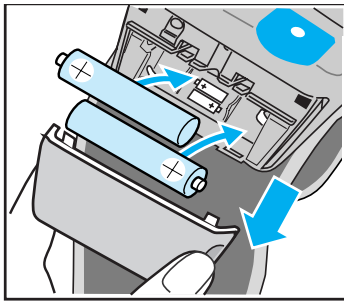
- To ensure accurate measuring results, carefully read these instructions.
- This product is intended for household use only. Keep product and batteries away from children.
- People suffering from cardiac arrhythmia, vascular constriction, arteriosclerosis in extremities, diabetes or users of cardiac pacemakers should consult their doctor before measuring their blood pressure themselves, since deviations in blood pressure values may occur in such cases.
- Please consult your doctor in case of any doubt, especially if you are under medical treatment or receiving medication.
- The use of this blood pressure monitor is not intended as a substitute for consultation with your doctor.

Product description

- A Position sensor icon
- B Memory button («mem»)
- C «I/O» (on/off) button
- D «Select» button
- E Battery compartment
- F Battery compartment cover
- G Cuff

Inserting batteries

- Use alkaline batteries, type LR 03 (AAA) (e.g. Duracell Ultra) as supplied with this product or rechargeable batteries.
- Remove the battery compartment cover (F) and insert two batteries with correct polarity (see symbol in the battery compartment (E)).
- New batteries (Duracell Ultra) will provide approximately 300 measurements.
- Only discard empty batteries. They should not be disposed of in the household waste, but at appropriate collection sites or at your dealer.

Use of rechargeable batteries

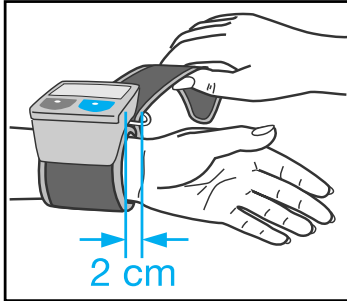
In case rechargeable batteries are used, please note that the «battery low» symbol may appear earlier on the display than with alkaline batteries. However, this will not affect the measuring accuracy.

Key rules for accurate blood pressure measurements

- Always take readings at the same time of day, ideally in the morning, under the same conditions.
- Do not measure within 30 minutes after smoking, coffee or tea consumption.
- Take off wrist watch and jewellery before fitting the appliance on the measuring arm.
- While taking a measurement, sit down, relax, keep still, especially the hand of the measuring arm, do not move or speak.
- Hold the appliance at heart level.
- Wait approximately 3 minutes before repeating a measurement.


Measuring

For a reliable blood pressure measurement, make sure that you hold the appliance at heart level. The position sensor icon (A) will guide you to hold your arm in the right position at heart level.


1. Fasten the appliance on the slim side of your bare forearm.

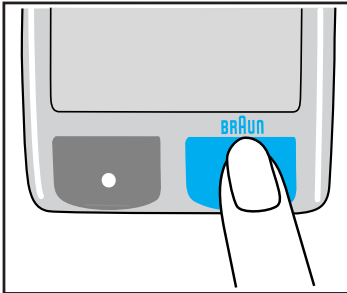Make sure that the cuff fits tight.

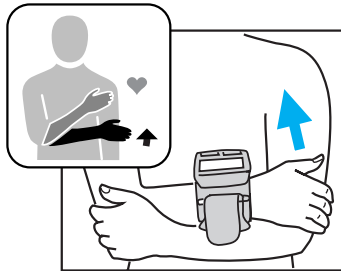
2. Press the red «I/O» (on/off) button (C).


3. Put the hand with the appliance on the other upper arm as shown. Slowly move the hand upward along the upper arm.

Watch the position sensor icon and move your hand up or down according to the direction of the arrow(s) shown.

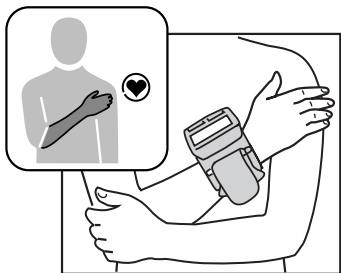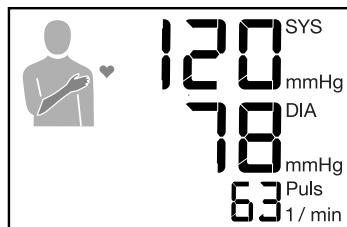4. The position sensor icon shown on the left and a beep confirm that you have reached the right measuring position. The measurement will start automatically.

When the measurement is complete, a beep will sound again, and the reading is displayed.

Upper (systolic) value
Lower (diastolic) value
Pulse

Please note:

15 seconds after switching on the appliance, the measuring process will start automatically, even if the appropriate measuring position has not been reached.

If at any time, you want to discontinue the measurement, press the «I/O» button.

Turning off the product

Press the I/O button to turn the product off. Otherwise the product will turn off automatically after 2 minutes.

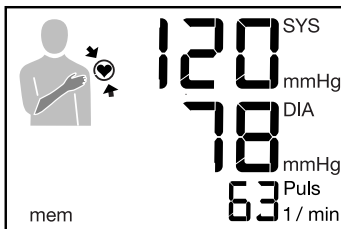
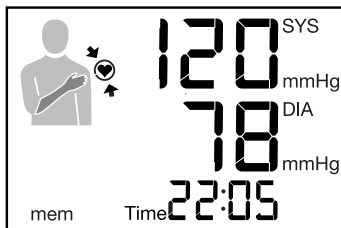
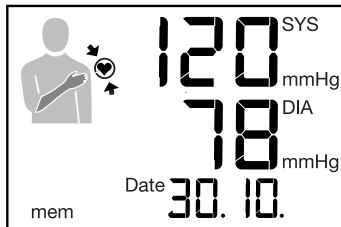
Memory function

Your blood pressure monitor can store the readings of 60 measurements. Storing is done automatically without having to press the memory button after a measurement.

The memory cannot be cleared. Once the memory is full, the oldest values will be overwritten.

Recalling stored readings

To recall stored readings, first turn the product off, then press the memory button («mem») (B).

The latest reading will be displayed. The lower display line shows successively date, time and pulse of the respective measurement.

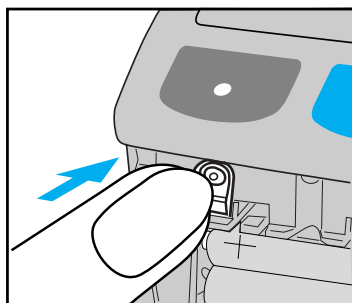
The position sensor symbol () shows that the respective measurement was taken at heart level.

Repeatedly pressing the memory button will call up one by one all stored readings (with date and time).

Scrolling

Keep the memory button pressed for scrolling through the memory. For your orientation, refer to the date in the lower display line.

Settings

By means of the «select» button (D) placed underneath the battery compartment cover, you can adjust the settings described below.

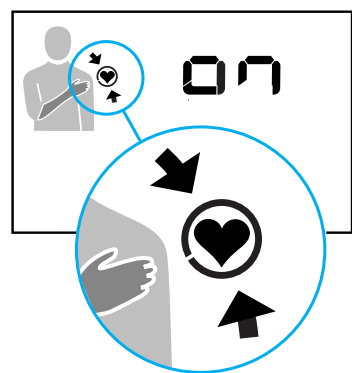
Time and date

First turn off the product. After pressing the «select» button, the hour position of the time display flashes. Using the memory button (B), you can change the hours.

Press the «select» button again to make the minute (accordingly day / month) position flash and change these positions using the memory button.

After having changed the time and date settings, press the «I/O» button to confirm. The device will then turn off.

Position sensor

Should you wish to deactivate the position sensor (e.g. because you want to measure while lying down), proceed as follows:

First turn off the product. Briefly press the «select» button. The display shows the time and date settings. Then repeatedly press the «select» button until the position sensor symbol will be displayed next to a flashing «on».

Press the memory button to change the setting from «on» (position sensor activated) to «--» (position sensor deactivated).

To confirm the change of setting, press the «I/O» button.

Please note:

Unless you confirm a modified setting by pressing the «I/O» button, the appliance will turn off automatically after 2 minutes without adapting the modified setting.

Storage and cleaning

Clean the product with a soft, slightly damp cloth. Stains on the cuff (G) can be carefully removed with damp cloth. Do not wash or dry clean the cuff. Do not store the product in a place exposed to direct sunlight or high humidity.

Calibration

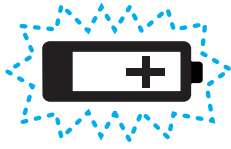
This product has been calibrated at the time of manufacture. If used according to the use instructions, periodic re-calibration is not required.

If at any time, you question the measuring accuracy, please contact a Braun Service Centre.

Manufacturing date is given by the LOT number on the back of the product. The first number after LOT represents the last digit of the year of manufacture. The next three digits give the day of the year of manufacture.

An example: LOT 2116 xx xx – this product has been manufactured on the 116th day of the year 2002.

What to do if ...

Problem	Reason	Solution
	Batteries are low. Measurements still possible. With rechargeable batteries, the «battery low» symbol may appear earlier than with alkaline batteries.	Battery replacement required soon.
 (flashing)	Batteries are flat. Memory readings can be called up. Measurements are not possible.	Insert new batteries.
	Cuff is too loose or cannot be inflated.	Refit cuff tightly, repeat measurement as shown and described under «Measuring».
e.g. 	Error message higher than 64: Cuff is not properly fitted.	Refit cuff tightly, repeat measurement as shown and described under «Measuring».

Problem	Reason	Solution
e.g. 	Error message lower than 64: System error	Have the product checked at a Braun Service Centre.
Measurement or inflation does not start instantly.	Right measurement position was not reached.	Repeat measurement as shown and described under «Measuring» or deactivate the position sensor.
Readings from repeated measurements differ considerably.	Blood pressure is a fluctuating value. For healthy adults, deviations of 10 to 20 mmHg are possible.	none
	Cuff is not properly fitted.	Ensure that the appliance is fitted as follows: • on top of the slim side of the forearm • with the display on top • with the battery compartment facing the user's body • approx. 2 cm away from the thumb root
	Readings were not consistently taken at heart level.	For each measurement, proceed exactly as shown and described under «Measuring».
	Talking, coughing, laughing, moving etc. when measuring will influence the reading.	While measuring, relax, keep still, do not move, or speak.
Prior to a measurement, the position sensor icon on the display does not work.	Position sensor is deactivated.	Refer to the chapter «Settings» to activate the position sensor.

Specifications

Model:	BP 2550
Operation principle:	Oscillometric method
Display:	Liquid Crystal Display
Measuring range:	0 – 300 mmHg (cuff pressure) 40 – 160 / min (pulse rate)
Blood pressure measurement:	50 mmHg (minimum diastolic value) 250 mmHg (maximum systolic value)
Laboratory accuracy:	± 3 mmHg (cuff pressure) ± 5 % of reading (pulse rate)
Clinical accuracy:	According to AAMI-SP 10 with auscultatory reference: < 5 mmHg systematic offset < 8 mmHg standard deviation
Inflation:	Automatic, adapted to blood pressure
Deflation:	Electronic control valve adapted to heart frequency
Power supply:	2 batteries, type LR 03 (AAA) (e.g. Duracell Ultra)
Operating temperature:	+ 10 °C to + 40 °C
Storage temperature:	– 20 °C to + 60 °C
Humidity:	Up to 85 % RH
Cuff:	Suitable for wrist circumference of 13.5 to 20 cm
Clinical validation:	According to AAMI-SP10 with auscultatory reference. (Please note that a validation against intra-arterial measurements can lead to different results for devices which are calibrated against auscultatory reference.)

Type BF equipment

Subject to change without notice.

This appliance conforms to the following standards:

DIN EN 60601-1: 3/96 «Medical electrical equipment» –

Part 1: General requirements for safety

DIN EN 1060-1: 12/95, AMD 1 09/02 «Non-invasive sphygmomanometers» –

Part 1: General requirements

DIN EN 1060-3: 9/97 «Non-invasive sphygmomanometers» –

Part 3: Supplementary requirements for electro-mechanical blood pressure measuring systems

This product conforms to the provisions of the EC directive 93 /42 / EEC
(Medical Device Directive).

Guarantee

We grant 2 years guarantee on the product commencing on the date of purchase. Within the guarantee period we will eliminate, free of charge, any defects in the appliance resulting from faults in materials or workmanship, either by repairing or replacing the complete appliance as we may choose.

This guarantee extends to every country where this appliance is supplied by Braun or its appointed distributor.

This guarantee does not cover: damage due to improper use, normal wear or use as well as defects that have a negligible effect on the value or operation of the appliance. The guarantee becomes void if repairs are undertaken by unauthorised persons and if original Braun parts are not used.

To obtain service within the guarantee period, hand in or send the complete appliance with your sales receipt to an authorised Braun Customer Service Centre.

For UK only:

This guarantee in no way affects your rights under statutory law.

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Australia

Gillette Australia Pty. Ltd.,
Private Bag 10, Scoresby,
Melbourne,
Victoria 3179,
☎ 1 800 641 820

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Bermuda

Gibbons Company
21 Reid Street
P.O. Box HM 11, Hamilton
☎ 295 00 22

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Hong Kong

Audio Supplies Company,
Room 506, St. George's Building,
2 ICE House Street, Hong Kong,
☎ 25 24-93 77

Hungary

Kisgep KFT,
Pannonia u. 23, 1136 Budapest,
☎ 13 494 955

Iceland

Verzlunin Pfaff hf.,
Grensasvegur 13,
Box 714, 121 Reykjavik,
☎ 5 33 22 22

India

Braun Division,
c/o Gillette Div. Op. Pvt. Ltd.,
34, Okhla Industrial Estate,
New Delhi 110 020,
☎ 11 68 30 218

Iran

Tehran Bouran Company
Irtuc Building,
No 874 Enghelab Ave.,
P.O. Box 15815-1391,
Tehran 11318,
☎ 02 16 70 23 50

Ireland (Republic of)

Gavin's Electronics,
83/84, Lower Camden Street,
Dublin 2,
☎ 1800 509 448

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Taiwan

Audio & Electr. Supplies Ltd.,
Brothers Bldg., 10th Floor,
85 Chung Shan N Rd., Sec. 1,
Taipei (104),
☎ 0 80 221 630

Thailand

Gillette Thailand Ltd.,
175 South Sathorn Road,
Tungmahamek,
Sathorn,
11/1 Floor,
Sathorn City Tower
Bangkok 10520
☎ 02-344 91 91

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Yemen (Republic of)

Saba Stores for Trading,
26th September Street,
P.O. Box 5278, Taiz,
☎ 967 4-25 23 80

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