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pressure sense

comfort KS 452


Instruction Manual

KS 452

- GB** Instruction Manual
- GR** Οδηγίες χρήσης
- D** Gebrauchsanweisung
- F** Manuel d'Instructions
- E** Manual de Instrucciones
- P** Manual de instrucoes
- PL** Instrukcja Obsługi
- I** Manuale di Istruzioni
- NL** Gebruiksaanwijzing
- TR** Kullanım Kilavuzu
- AR** دليل التعليمات
- R** Инструкция по эксплуатации

GB Important

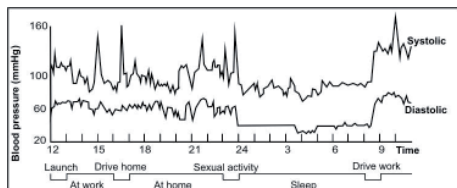1. Read these directions for use carefully in order to obtain accurate measuring results.
2. This blood pressure monitor is only suitable for measuring the blood pressure of adults.
3. This blood pressure monitor measures blood pressure on the basis of pulse rate. If you have a very weak or irregular pulse, the blood pressure monitor may not be able to determine the right pressure value.
4. Eating, smoking, drinking (especially alcoholic beverages) and the use of medicines will influence the measurement.
5. If in doubt, consult a doctor.
6. Only a doctor is capable of interpreting your blood pressure values. Never change your medication without consulting a doctor.
7. Your blood pressure may fluctuate over time and during day. It may even vary more than 10mmHg within minutes. This means that successive measurements may show different blood pressure values. (See Shape 1)


Shape 1: Alteration diagram of blood pressure towards 12 hours of a 35 years old

GB Conditions for accurate blood pressure measurement

1. Sit down and relax for 10 minutes before you use the blood pressure monitor. Breath deeply in and out five times before you start measuring.
2. Wait at least 3 minutes between two successive measurements to avoid incorrect readings due to the temporary congestion of the arteries caused by the cuff.

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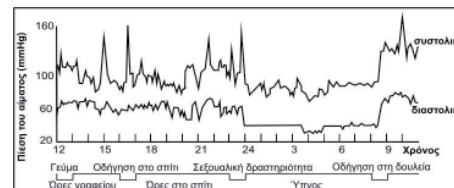3. Perform the measurement every day at the same time, preferably early in the morning on empty stomach, since your blood pressure fluctuates in the course of the day.
4. Do not eat, smoke or drink before measuring your blood pressure. Alcoholic beverages in particular will influence the measurement.
5. Do not talk, laugh, cough or move your arm during the measurement, as this may influence the measurement results.
6. Do not cross your legs during measurement.
7. Do not attach too much importance to the results of a single measurement. Performing measurements regularly over a longer period of time and recording the results will give you a better insight into your blood pressure than a single measurement.

GB General information about blood pressure

Blood pressure is measured in mmHg (mm mercury pressure). Blood pressure consists of two values. Systolic pressure (SYS) is created by the contractions of the heart muscle, which pumps the blood into the arteries. Diastolic pressure (DIA) is created when the heart relaxes and fills with blood again. The World Health Organization (WHO) has laid down the following standard values with regards to the blood pressure of adults.

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GB Introduction

Blood pressure measurements determined with **KS 452** are equivalent to those obtained by a trained observer using cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard, Electronic or Automated Sphygmomanometers.

KS 452 is protected against manufacturing defects by an established International Warranty Program. For warranty information, you can contact Kessler or your local distributors.

Attention: Consult the accompanying documents.

Please read this manual carefully before use. For specific information on your own blood pressure, contact your physician. Please be sure to keep this manual.

GB Preliminary Remarks

This Blood Pressure Monitor complies with the European regulations underlying the German Law governing medical products (MPG) and bears the CE mark "CE 0120". The quality of the device has been verified and conforms to the provisions of the EC council directive 93/42/EEC dated 14 June 1993 concerning medical devices, as well as the EMC directive 89/336/EEC: EN 1060-1 (Dec. 1995) Non-invasive blood pressure measuring equipment General requirements and EN 1060-3 (Feb. 1997) Noninvasive blood pressure measuring equipment Supplementary requirements for electromechanical blood pressure measuring systems.

GB How "KS 452" Works

KS 452 uses oscillometric method to detect your blood pressure. Before the cuff starts inflating, the device will establish a baseline cuff pressure, which is equivalent to the air pressure. The measurement of your blood pressure is based on this baseline pressure. After the cuff inflates to block your blood in the artery, the deflation process starts. During the deflation of the wrist cuff, **KS 452** is detecting the pressure oscillations generated by the beat-to-beat pulsatile. Any muscle movement during this period of time will cause measurement error. After detecting the amplitude and the slope of the pressure oscillations during the deflation process, **KS 452** will determine the systolic and diastolic pressures for you, and your pulse rate is detected at the same time.

Systolic (SYS.)	< 120	120 – 139	140 – 159	≥ 160
Diastolic (DIA.)	< 80	80 – 89	90 – 99	≥ 100
GB	Normal	Prehypertension	Stage 1 Hypertension	Stage 2 Hypertension
D	Normal	Pröhypertonie	Hypertension Bluthochdruck, Stufe 1	Hypertension Bluthochdruck, Stufe 2
F	Normale	Pnhypertension	Hypertension stade 1	Hypertension stade 2
E	Normal	Prehipertension	Fase 1 Hipertension	Fase 2 Hipertension
P	Normal	Pn-hipertensyo	Hipertensyo de estagio 1	Hipertensyo de estagio 2
PL	Ciśnienie normalne	Ciśnienie podwyższone	Nadciśnienie pierwszego stopnia	Nadciśnienie drugiego stopnia
I	Normal	Pre-ipertensione	Stadio 1 Ipertensione	Stadio 2 Ipertensione
NL	Normaal	Prehypertensie	Fase 1 Hypertensie	Fase 2 Hypertensie
TR	Normal	Prehipertansiyon	1. Evre Hipertansiyon	2. Evre Hipertansiyon
AR	المرحلة الطبيعية	ما قبل زيادة ضغط الدم	المرحلة الثانية من زيادة ضغط الدم	المرحلة الأولى من زيادة ضغط الدم
R	Норма	Предгипертоническое состояние	1 стадия гипертонии	2 стадия гипертонии

GB

For reliable monitoring and reference of blood pressure, keeping long-term records is recommended.

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GB About Blood Pressure

The National High Blood Pressure Education Program Coordinating Committee has developed a blood pressure standard, according to which areas of low- and high-risk blood pressure are identified. This standard, however, is a general guideline as individual's blood pressure varies among different people and different age groups...etc. It is important that you consult with your physician regularly. Your physician will tell you your normal blood pressure range as well as the point at which you will be considered at risk.

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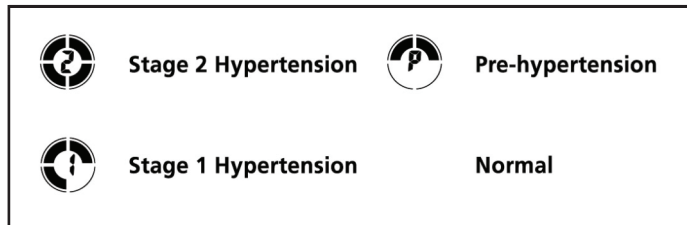
GB Blood Pressure Classification Calculator

The National High Blood Pressure Education Program Coordinating Committee has developed a blood pressure standard according to which areas of low- and high-risk blood pressure are identified, classifying blood pressure ranges into 4 stages. This unit is equipped with innovative Blood Pressure Classification Indicator, which visually indicates the assumed risk level (prehypertension / stage 1 hypertension / stage 2 hypertension) of the result after each measurement. (Please note above.)

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Device Messages

Message	Root Causes	Corrections
EE	Measurement Error	Exceeding measurement range. Measure again quietly. Wrap the cuff correctly and keep wrist steady during measurement. If error keeps occurring, return the device to the local distributor or service center.
E1	Air Circuit Abnormality	Measure again quietly. If error keeps occurring contact, local Kessler service.
E2	Pressure Exceeding 300 mmHg	Switch the unit off and measure again quietly. If error keeps occurring, contact local Kessler service.
E3	Data Error	Remove and reload the batteries. If error keeps occurring, contact local Kessler service.
CA	Recalibration	The device has completed a measurement number after which it is recommended to proceed on recalibration in order to secure device correct function. Please contact local Kessler service to arrange recalibration.

GB Cautionary Notes

1. The unit contains high-precision assemblies. Therefore, avoid extreme temperatures, humidity, and direct sunlight. Avoid dropping or strongly shocking the main unit, and protect it from dust.
2. Leaky batteries can damage the unit. Remove the batteries when the unit is not used for a long time.
3. The unit should not be operated by children so to avoid hazardous situations.
4. **KS 452** is not filed serviceable. You should not use any tool to open the device nor should you attempt to adjust anything inside the device. If you have any problems, please contact the Kessler Local Service.
5. If the unit is stored near freezing, allow it to acclimate at room temperature before use.
6. For users diagnosed with common arrhythmia (atrial or ventricular premature beats or atrial fibrillation), diabetes, poor circulation of blood, kidney problems, or for users suffered from stroke, or for unconscious users, the device may have difficulty in determining the proper blood pressure.
7. Clean the blood pressure monitor body and the cuff carefully with a slightly damp, soft cloth. Do not press. Do not wash the cuff or use chemical cleaner on it. Never use thinner, alcohol or petrol (gasoline) as cleaner.
8. To stop inflation at any time, press the ⏸ key, and the air in the cuff will be rapidly exhausted.
9. Once the inflation reaches 300 mmHg, the unit will start deflating rapidly for safety reason.

Notes:

Batteries are hazardous waste.
Do not dispose them together with the household garbage.

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GB Specifications

Power source	: 3V DC Two LR03 (AAA) Alkaline batteries
Measurement method	: Oscillometric
Measurement range	: Pressure: 40~250 mmHg Pulse: 40~199 beats/minute
Accuracy	: Pressure: ± 3 mmHg Pulse: within $\pm 5\%$ of reading
Pressure sensor	: Semi-conductor
Inflation	: Pump driven
Deflation	: Automatic Pressure Release Valve
Memory capacity	: 99 memories
Auto-shut-off	: 1 minute after last key operation
Operation environment	: 10°C~40°C(50°F~104°F) 40%~85% RH max.
Storage environment	: -10°C~60°C(14°F~140°F) 10%~90% RH max.
Weight	: 125g (w/o batteries)
Limited user	: Adult users
Wrist circumference	: 13.5~22cm (5.3"~8.7")
Dimensions	: 73(L) x 69(W) x 82(H) mm
	: Type BF Device and cuff are designed to provide special protection against electrical shocks.

*Specifications are subject to change without notice.

Blood Pressure Log

Name: _____ Age: _____ Weight: _____

Date 																				
Time 																				
mmHg																				
220																				
200																				
180																				
160																				
140																				
120																				
100																				
80																				
60																				
Pulse 																				
Body Condition 																				