MEDISANA®

CardioDock®

DE Blutdruck-Messmodul

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info@bluepoint-service.ch

www.bluepoint-service.ch

- ➊ Connection slot on your iPhone or iPod touch
- ➋ Upper arm pressure cuff
- ➌ CardioDock blood pressure module
- ➍ Connector for your iPhone or iPod touch
- ➎ Air hose
- ➏ 3MAM button (for 3 readings in a row to calculate a weighted average value)
- ➐ CardioDock (rear view)
- ➑ Connector for the hose
- ➒ Connection for power unit's micro USB plug
- ➓ Start button

Description of the system	2
Contents	3
1 Explanation of symbols	5
2 Safety Information	
2.1 Tips for your well being	6
2.2 Important considerations	8
2.3 Information about the software	10
3 Useful Information	
3.1 Items supplied and packaging	11
3.2 What is blood pressure?	12
3.3 How do you measure with CardioDock?	12
3.4 Why measure at home with CardioDock?	13
3.5 Blood pressure classification for adults	14
3.6 Influencing and evaluating readings	14
3.7 CardioDock as a charger for iPhone/iPod	15
4 Getting started	
4.1 Charging the blood pressure module's battery	16
4.2 Installing the VitaDock® (app) software	17
4.3 Settings	18
5 Operation	
5.1 Putting on the upper arm pressure cuff	21
5.2 Correct position for taking readings	21
5.3 Taking blood pressure readings	22
5.4 The measurement	24
5.5 3MAM The measurement	25
5.6 Displaying and managing the measurements ...	26
5.6.1 Statistics	26

5.6.2 Cardio diary	27
5.6.3 Sending a report	27
5.7 Adding further user profiles	28
5.8 Deleting user profiles	29

6 Miscellaneous

6.1 Error messages	30
6.2 Care and maintenance	31
6.3 Guidelines / Standards	32
6.4 Disposal	32
6.5 Technical specifications	33

7 Warranty

7.1 Warranty and repair terms	35
-------------------------------------	----

This instruction manual belongs to this device.

It contains important information about starting up and operation. Read the instruction manual thoroughly.

Non-observance of these instructions can result in serious injury or damage to the device.

WARNING

These warning notes must be observed to prevent any injury to the user.

CAUTION

These notes must be observed to prevent any damage to the device.

NOTE

These notes give you useful additional information on the installation and operation.

Classification: Type BF

Lot number

Manufacturer

Date of manufacture

IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use.

Should you give this device to another person, it is vital that you also pass on these instructions for use.

Safety Information

2.1 Tips for your well being

Proper use

- The system is intended for measuring the blood pressure from the upper arm of an adult. It should be used together with Apple (iOS 3.0 or higher) devices (iPhone 3G, 3GS, 4G / iPod touch 2nd, 3rd, 4th generation).

Contraindication

- The system is not suitable for measuring the blood pressure of children. Ask your doctor before using it on older children.
- The device is not suitable for persons with acute arrhythmia.

- Irregularity of pulse or arrhythmia can lead to difficulties in recording a correct reading when measurements are taken using oscillometric blood pressure devices. This device is electronically equipped to detect arrhythmia and what are known as movement artefacts, which it indicates on the display with the icon «♥». In this case, the reading may be imprecise. Inform your doctor so that he or she can assess the situation.
- If you should suffer from conditions such as arterial occlusive disease, please consult your doctor before using the unit.
- The unit may not be used to check the heart rate of patients with a pacemaker.
- Pregnant patients should take their own personal state of health into consideration and take all necessary precautions before using the unit. If in doubt, consult your doctor.
- Should you experience discomfort or other complaints during measurement, for example pain in the upper arm, please take the following action: Press the Start button (Stop) on the CardioDock or the Stop button on the display to deflate the cuff immediately. Loosen the cuff and remove it from your arm. Please contact either your supplier or MEDISANA directly.
- Do not use the cuff on open wounds, in order to prevent the spread of infection.
- Patients with a weak pulse should raise their arm and open and close their hand around 10 times before taking a reading. This exercise optimizes the pulse and the measuring process.

WARNING

**Do not take any therapeutic measures based on your own measurements!
Never change the quantity of medication prescribed by your doctor!**

2.2 Important considerations

- The CardioDock is only to be used for the specific purpose described in this instruction manual.
- The system is not intended for commercial use.
- Any misuse will void the warranty.
- On the rare occasion of a fault causing the cuff to remain fully inflated during measurement, open the cuff immediately.
- Do not operate the system in the vicinity of strong transmitters such as microwave and shortwave appliances. This may impair the correct functioning of the unit.
- To ensure CardioDock readings are accurate, keep at a minimum distance of about 30 cm from other wireless devices.
- If a call or a text message is received during a reading, the procedure is cancelled. The cuff deflates.
- This module is not designed to be used by persons (including children) with limited physical, sensory or mental abilities, or by persons with insufficient experience and/or knowledge, unless under observation by a person responsible for their safety, or unless they have been instructed in the use of the device.

- Children must be supervised to ensure that they do not play with the device.
- Do not use the module if it does not work properly, if it has been dropped or has fallen into water or become damaged.
- Do not use the module in damp environments.
- Protect the module from moisture. Should moisture enter the unit, stop using it immediately. In this case, please contact either your supplier or MEDISANA directly.
- If a fault occurs, do not try to repair the module yourself. Attempts to do so will void the warranty. Refer all servicing to authorized service personnel.
- Do not drop the module and protect it against impact and strong vibration.
- Always keep the module clean and store it in a safe place. Protect the module from direct sunlight to prolong its service life.
- For cleaning, follow the instructions in 6.2 Care and maintenance, page 31.

2.3 Information about the software

- Make sure that your Apple device is running on the latest operating system. For information about software updates, see the instructions for your Apple device.
- Make sure that your VitaDock software (app) is updated at regular intervals. For information about App updates, see the instructions for your Apple device.
- Users should set password protection in the device settings and protect their user profile in VitaDock with a password.
- The app shows a window with a warning when you change settings. Before the changes come into effect, you have to confirm the warning message.
- For the module to function correctly, the Apple device has to be sufficiently charged. Users should have a power supply unit or a charging device to hand.
- Make sure that there is always enough memory available to back up the reading data on your Apple device.
- If you receive a call or a text message during a reading, the reading is cancelled for safety reasons. If you want to prevent your reading from being cancelled, we recommend switching your iPhone to flight mode.

Thank you very much

Thank you for your confidence in us and congratulations on your purchase!

By purchasing the **CardioDock** blood pressure module, you have gained a high-quality product made by **MEDISANA**. Experience the new standard in blood and pulse measurement. Measure your blood pressure easily and accurately with your iPod touch or iPhone. To derive the most success and to enjoy your **CardioDock** blood pressure module for a very long time, we recommend that you read the following information on the use and care of the appliance carefully.

3.1 Items supplied and packaging

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use the appliance and contact your supplier or your service centre.

The following parts are included:

- 1 **CardioDock** blood pressure module
- 1 Preformed upper arm cuff
size M (22 - 32 cm)
- 1 Power supply unit with micro USB plug
- 1 Carrying case
- 1 Instruction manual

If you notice any transport damage during unpacking, please contact your supplier without delay.

WARNING

**Please ensure that the polythene packing is kept away from the reach of children!
Risk of suffocation!**

3.2 What is blood pressure?

Blood pressure is the pressure created in the blood vessels by each heart beat. When the heart contracts (= systolic) and pumps blood into the arteries, it brings about a rise in pressure. The highest value is known as the systolic pressure and is the first value to be measured. When the cardiac muscle relaxes to take in fresh blood, the pressure in the arteries also falls. Once the arteries are relaxed, a second reading is taken – the diastolic pressure.

3.3 How do you measure with CardioDock?

The **CardioDock** blood pressure module, combined with the VitaDock app, turns your iPod or iPhone into an effective blood pressure device for use on the upper arm. The pressure is read by a pressure sensor that uses a microprocessor to evaluate the vibrations created when the pressure cuff is pumped up and released over an artery.

3.4 Why measure at home with CardioDock?

MEDISANA has had many years of experience in the field of blood pressure measurement. The high precision of the devices has been confirmed by clinical tests, carried out according to international standards. One strong argument in favour of taking blood pressure at home is the fact that the readings are taken in familiar surroundings under relaxed conditions. The most important reading is the so called „baseline value“, the one taken before breakfast, directly after getting up in the morning.

With **CardioDock**, you can see for the first time what effect your daily life has on your blood pressure. VitaDock uses your readings to create individual graphs clearly showing your blood pressure values.

As far as it is possible, you should always try to take your blood pressure at the same time and under the same conditions each day. This ensures comparable results and helps in detecting the first signs of high blood pressure. If high blood pressure remains undetected, it increases the risk of further cardiovascular diseases.


THEREFORE WE RECOMMEND:

Take a blood pressure reading daily and on a regular basis, even if you are experiencing no apparent complaints.

3.5 Blood pressure classification for adults

The table on the right shows the reference values for normal and high blood pressure, discounting age and sex. The table represents the guidelines from the World Health Organisation (WHO).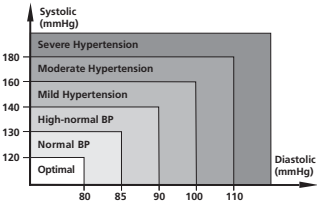

Blood pressure classification	Systolic (mmHg)	Diastolic (mmHg)	Colour code
Optimal	< 120	< 80	Green
Normal	120-129	80-84	Green
High normal	130-139	85-89	Yellow
High blood pressure degree 1	140-159	90-99	Orange
High blood pressure degree 2	160-179	100-109	Red
High blood pressure degree 3	> 180	> 110	Red

3.6 Influencing and evaluating readings

- Measure your blood pressure several times, then record and compare the results. Do not draw any conclusions from a single reading.
- Your blood pressure readings should always be evaluated by a doctor who is also familiar with your personal medical history. Take regular readings and inform your doctor from time to time.
- When taking readings, remember that the daily values are influenced by several factors. Smoking, consumption of alcohol, drugs and physical exertion influence the measured values in various ways.

- Measure your blood pressure before meals.
- Before taking readings, allow yourself at least five minutes rest.
- If the systolic and diastolic readings seem unusual (too high or too low) on several occasions, despite correct use of the unit, please inform your doctor. This also applies to the rare occasions when an irregular or very weak pulse prevents you from taking readings.

**WARNING**

Blood pressure that is too low represents just as great a health risk as blood pressure that is too high!

Fits of dizziness may lead to dangerous situations arising (e.g. on stairs or in traffic)!

3.7 CardioDock as a charger for the iPhone or iPod touch

The **CardioDock** blood pressure module is more than just a blood pressure reader. It is also a charger and docking station for your iPod or iPhone. You can also use it while looking at photos, surfing the web or reading your e-mails.

4.1 Charging the blood pressure module's battery

The CardioDock blood pressure module has an integrated lithium ion battery (3,7 V --- , 1750 mAh).

- a. The CardioDock's battery is charged using the mains adapter supplied. Insert the micro USB plug on the connecting cable into the charging socket on the reverse of the CardioDock.
- b. Avoid overcharging the battery or letting it run down completely as this shortens the battery's service life. You can see the degree of charge when the CardioDock is connected with the iPhone. We recommend charging the battery when its charge level drops below 25% and stop charging once the battery is completely charged by pulling out the USB cable.

4.2 Installing the VitaDock® (app) software

MEDISANA is setting new standards in mobile health with **VitaDock**.

VitaDock app, the central software application for displaying and evaluating the body's vital signs and health values, includes software for:

- the blood pressure and pulse module **CardioDock®**
- the blood glucose module **GlucoDock®**
- the infrared thermometer module **ThermoDock®**
- the body analysis scales **TargetScale®**

The GlucoDock, ThermoDock and TargetScale software is only fully functioning if the relevant modules or scales are used.

To operate your new **CardioDock** blood pressure module for iPod touch and iPhone, you have to install the free VitaDock app.

You can download it from Apple's app store and then install it.

To download the software, you need an Apple ID with an e-mail address and password, as well as an active Internet connection. Downloading the software may be subject to charges from your telephone provider.

- To find the VitaDock (app) in the App Store, enter the search term "VitaDock".
- Once the software has been successfully installed, the new "VitaDock" icon  appears on your device.

4.3 Settings

Before using for the first time

Check the date and time settings on your iPod touch or iPhone. The date and time are important for evaluating the readings.

Start the **CardioDock** by touching the icon . The display briefly shows the “Vita Dock” start page and then automatically switches to an entry screen for your personal data.

You can create a total of 4 user profiles in the VitaDock.


In the first user profile, you can enter your personal data. Pressing on a text field opens a keyboard field with which you can edit your data.

You have the option of activating a password for using the VitaDock app.

To do this, touch the “Password” text field and specify a password. You are asked to enter it twice, to make sure of it.

If you activate the password function, you have to enter the password each time the app restarts.

For more information, see page 28-29.

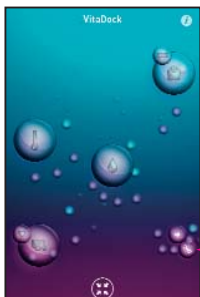"CardioDock" settings

After you save your data, you enter the VitaDock universe.

Touch the "Setting" icon .

Touch the "CardioDock" icon .

Choose:
"WHO" standard or
"custom" settings.
(WHO table on page 14)

If you chose "Custom" settings,
enter the target blood
pressure values specified by
your doctor:

Systolic
Diastolic
Pulse

Save your entries.

„Zoom“

Once you have made all the CardioDock settings, you can zoom out and slide the display to reach the other applications (CardioDock, GlucoDock, ThermoDock and TargetScale).

Touching the icons takes you directly to the programme.

„CardioDock“ display Masthead

New reading (page 22)

Result of last reading Systolic and diastolic (mmHg) display of the readings as per WHO standard (page 14).

Cardio statistics (page 26)

Cardio diary (page 27)

3MAM reading (page 25)

Result of last reading Pulse/min


„Masthead“ display

When a VitaDock module is connected, the module's serial number appears in the masthead.

The serial numbers of all the modules used with VitaDock are displayed here and saved.



5.1 Putting on the upper arm pressure cuff

- First attach the free end of the cuff's air tube to the designated connector **8** on the unit.
- Slip the preformed cuff over the bare left upper arm.
- Position the air hose in the middle of your arm in line with your middle finger (*ill. 1 a*). The lower edge of the cuff should be 1 - 2 cm above the crease of the elbow (*ill. 1 b*). Pull the cuff tight and close the Velcro fastener (*ill. 1 c*).
- Only position the cuff on the right arm if it cannot be used on the left arm. Always carry out measurements on the same arm.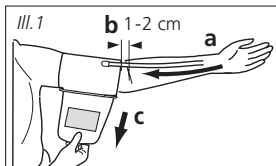

5.2 Correct position for taking readings

- If possible, take the reading while seated.
- Relax your arm and place it comfortably, on a table for instance. Keep still during the measurement: do not move and do not talk as this may alter the measurements. The Correct Measuring Position when sitting (*ill.2*).
- In special cases the measurement can also be taken lying down. Correct Position for taking measurements lying down (*ill.3*).

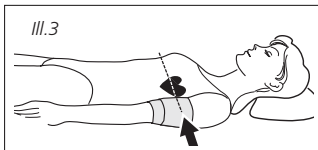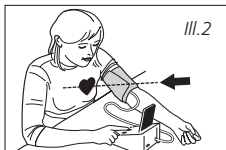The cuff should always be at the same height as your heart.

5.3 Taking blood pressure readings

To avoid disruption from an incoming call or text message during a reading, we recommend switching the iPhone to flight mode.

1. You can activate the CardioDock's blood pressure measuring mode in two ways:

a) Plug & play: Insert the Apple device, when switched on, into the CardioDock module. Select the user profile and enter the password, if necessary.

The VitaDock app starts automatically in blood pressure measuring mode.

b) Manual start: Start the VitaDock app. Select the user profile and enter the password, if necessary. The VitaDock universe appears.

Touch the "CardioDock" application and tap on the "new" icon .

The "NEW MEASUREMENT STEP 1" window opens. You are prompted to the CardioDock module (or enter the value manually). Insert the Apple device in the CardioDock module.

2. Once you have put on the upper arm cuff and are in the measuring position, start the reading.

Start the blood pressure measurement by touching "Start" or pressing the start button ⑩ on the CardioDock module.

3. The module automatically pumps up the cuff. Remain at rest during the reading: do not move and do not talk. While the cuff is pumping up, the pulse icon flashes. The rising pressure is shown as an increasing number, the pulse is shown as a wave.

Important:

- You can cancel the measurement at any time by touching the "STOP" field or pressing the Start button ⑩ on the module. The air escapes from the cuff.
- If a call or a text message is received during the reading, the measurement is cancelled. The cuff deflates.

5.4 The measurement

The unit inflates the cuff until sufficient pressure has built up for a measurement. Next, the device lets the air out of the cuff and shows the reading.

1. Display:
 - Systolic pressure
 - Diastolic pressure
 - Pulse rate

Touch the field "ADD COMMENT" to open the entry field for comments about the reading.

2. Optional comments:
 - Activity (infirm, rest, normal activity, sport)
 - Mood
 - Notes

Comments are displayed in the statistics and in the log book.

5.5 3MAM measurement

The 3MAM measurement can give you more precise readings, particularly if your blood pressure fluctuates. Three readings are taken one after the other, and a weighted average is calculated from this.

You can start the 3MAM measurement in the VitaDock app or by pressing the 3MAM **6** button on the CardioDock module.

Three readings are taken. There is a pause of 15 seconds between each reading.

The display shows the time remaining until the next reading.

The follow-on measurements start automatically; remain sitting calmly during the process.

When the last measurement is complete, the result is displayed: you can add comments and save it, as normal.

5.6 Displaying and managing the measurements

You can access the test results at any time. Touch the relevant icon to access the results.

5.6.1 Statistics


Touching the “stats” icon  loads the Cardio statistics. The display shows the readings from the last three days.

Systolic = white circle
Diastolic = green dot
3MAM = red circle

Rotate your device by 90° and touch the relevant field to view additional test results:

1 day, 1 week, 1, 3, 6 month, 1 year or overall.

1 Week

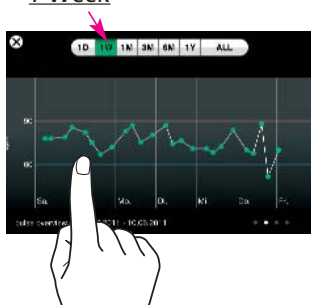The measurements for one week are displayed.

You can display the overview: mood and activity by sliding the display to the left.

5.6.2 Cardio diary

Touching the “diary” icon  loads the Cardio diary for the active user profile.


Tap the arrows to page through the log book.

5.6.3 Sending a report

You have the option of sending monthly reports for the active user profile by e-mail – to your doctor, for example. To set up the email address to send a report see your iPhone/iPod touch/iPad manual.

When the CardioDock display is shown, touch the icon  and slide the display to “Global Settings”.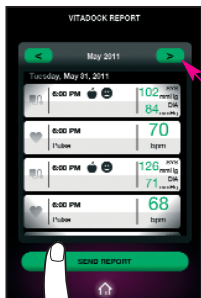

Touch the “report” icon  to display the monthly reports.

Touch the arrows to reach the previous or next month. Slide the display up or down to view the individual days in the month.

Tap “Send report” to send the data by e-mail.

5.7 Adding further user profiles

Personal Settings

Enter personal data to customize your account

Enter your name

Jack Smith

Sign out

Sex: Male Female

Date of Birth: June 15, 1971

Height unit: cm in

Height: 170 cm

Enter a password to secure your account.

Save

You have the option of creating up to three additional user profiles in the VitaDock app.

There are two ways of doing this:

- a) You can create a new user profile when you start the VitaDock app.
- b) Touch "Personal settings" icon and log out of the current user profile by touching "Sign out".

VitaDock Edit

Choose an user account to login

Jack Smith


Add another user

Once you have logged out of your user profile, you can create another user profile on the next screen.

To do this, touch "New user profile" field at the bottom.

You can then create an additional user profile.

5.8 Deleting user profiles

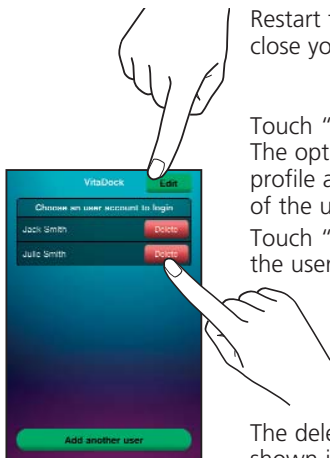Restart the VitaDock app or close your user profile.

Touch "Edit".

The option of deleting the profile appears to the right of the user profile.

Touch "Delete" to delete the user profile.

The deleted user profile is shown in grey. For a period of 14 days, you have the option of reactivating the deleted user profile by touching it. After 14 days, the user profile is automatically deleted completely from the app.

CAUTION!

The user profile, all the readings and entries are irreversibly deleted. There is no way to restore them.

6.1 Error messages

Unusual reading

- Repeat the test.
- The cuff is not positioned correctly or not put on correctly.
- You did not keep to the correct posture during the reading.
- During the reading, you spoke, moved your arm or body, or you are excited or nervous.
- If you get another unusual reading, contact your doctor.

Irregular pulse (arrhythmia)

- This device is not suitable for persons with a acute irregular pulse.

The cuff does not pump up properly

- Check that the air hose is fitted correctly.
- The air hose is pinched or is leaking.

The device does not respond

- The Apple device is not connected properly to the blood pressure module.
- The battery is too weak.
- Separate the Apple device and the module, reopen the software and reconnect the Apple device to the module.

6.2 Care and maintenance

Your **CardioDock** blood pressure module is a high-precision instrument. Handle with care to avoid damaging the electronics and prevent defects.

Observe the following instructions:

- Never use strong detergents or hard brushes.
- Clean the unit using a soft cloth lightly moistened with a mild soapy solution. Do not allow any water to get into the unit. Do not use the unit again until it is completely dry.
- Do not expose the unit to direct sunlight, and protect it from dust and moisture.
- Only inflate the cuff when it is in position around the upper arm.
- Keep the cuff clean. If it gets dirty, separate it from the device and clean it with mild soapy water. Rinse it thoroughly with cold water and let it dry in the air. Do not put the cuff in a dryer and do not iron it.
- Even if you do not use the device for a long time, charge it fully once a month.
- Put the module back in its carrying case after each use.
- *Servicing and calibration*

The unit has been calibrated by the manufacturer for a period of two years' use. **In the case of commercial use**, the unit must be serviced and recalibrated at least once every two years. This calibration will be charged for and can be carried out by an appropriate authority or an authorised service centre - in accordance with the regulations for users of medical equipment.

6.3 Guidelines / Standards

This blood pressure monitor meets the requirements of the EU standard 93/42/EEC for non-invasive blood pressure monitors. It is certified in accordance with EC Guidelines and carries the CE symbol (conformity symbol) „CE 0483“.

CE 0483

The blood pressure monitor corresponds to European standards EN 60601-1/+A1/+A2, EN 60601-1-2, EN 1060-1/+A1, EN 1060-3/+A1.

Electromagnetic compatibility:

The device complies with the EN 60601-1-2 standard for electromagnetic compatibility.

Enquire at **MEDISANA** for details on this measurement data.

6.4 Disposal

This module must not be disposed of together with domestic waste.

All users are obliged to hand in all electrical or electronic devices, regardless of whether or

 not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner.

Please remove the battery before disposing of the device / unit. Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop. Consult your local authority or your supplier for information about disposal.

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required.

6.5 Technical specifications

Product name	: CardioDock blood pressure module
Runs on	: Apple devices - iPhone 3G, 3GS, 4G - iPod touch 2nd, 3rd, 4th generation
Operating system	: Apple iOS 3.0 or higher
Power supply	: via USB (5 V $\equiv$, 1 A) or integrated battery (3,7 V $\equiv$, 1750 mAh, Li-Ion)
Measuring technique	: Oscillometric Single readings or 3MAM readings
Pressure generation	: Automatic, using micro pump
Deflation	: Automatic
Blood pressure measuring range	: 30 – 280 mmHg
Pulse measuring range	: 40 – 200 beats / minute
Maximum error tolerance for static pressure	: ± 3 mmHg
Maximum error tolerance for pulse rate	: $\pm 5\%$

Memory	: depends on memory of the connected Apple iPhone or iPod touch device
Operating conditions	: +10 °C to +40 °C, 15 % to 90 % humidity
Storage conditions	: -20 °C to +55 °C, 15 % to 90 % humidity
Dimensions (L x W x H)	: 120 x 84 x 65 mm
Cuff size	: 22 – 32 cm Adult cuff for average upper arm circumference (size M)
Weight	: approx. 450 g
Article number	: 51281
EAN number	: 40 15588 51281 0
Spare cuffs	: - Normal/Medium (size M) 22 – 32 cm 51283 4015588 51283 4 - Large (size L) 30 - 42 cm 51284 4015588 51284 1

In accordance with our policy of continual product improvement, we reserve the right to make technical and visual changes without notice.

7.1 Warranty and repair terms

Please contact your supplier or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be corrected free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. nonobservance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear (cuffs, batteries etc.).
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

MEDISANA AG

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41468 NEUSS

Germany

eMail: info@medisana.de

Internet: www.medisana.de

For service, accessories and spare parts, please contact:

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