Model: CH91

## EN English Introduction

Blood pressure measurements determined with CH91 are equivalent to those obtained by a trained observer using cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard, Electronic or Automated Sphygmomanometers. This unit is to be used by adult consumers in a home environment. Do not use this device on infants or neonates. CH91 is protected against manufacturing defects by an established International Warranty Program. For warranty information, you can contact your local distributors.

Attention: Consult the accompanying documents. Please read this manual carefully before use. For specific information on your own blood pressure, contact your physician. Please be sure to keep this manual.

### Preliminary Remarks

This Blood Pressure Monitor complies with the European regulations and bears the CE mark "CE 0120". The quality of the device has been verified and conforms to the provisions of the EC Council directive 93/42/EEC (Medical Device Directive), Annex I essential requirements and applied harmonized standards.

EN 1060-1: 1995/A2: 2009 Non-invasive sphygmomanometers - Part 1 - General requirements  
EN 1060-3: 1997/A2: 2009 Non-invasive sphygmomanometers - Part 3 - Supplementary requirements for electro-mechanical blood pressure measuring systems

EN 1060-4: 2004 Non-invasive sphygmomanometers - Part 4: Test Procedures to determine the overall system accuracy of automated non-invasive sphygmomanometers.

This blood pressure monitor was designed for long service time. To ensure accurate measurements, this monitor is recommended to be re-calibrated every two years.

### Blood Pressure Standard

The National High Blood Pressure Education Program Coordinating Committee has developed a blood pressure standard, classifying blood pressure ranges into 4 stages. (Ref. The Seventh Report of the Joint National Committee on Prevention, Detection, Evaluation, and Treatment of High Blood Pressure-Complete Report, JNC-7, 2003). This blood pressure classification are based on historical data, and may not be directly applicable to any particular patient. It is important that you consult with your physician regularly. Your physician will tell you your normal blood pressure range as well as the point at which you will be considered at risk. For reliable monitoring and reference of blood pressure, keeping long-term records is recommended. Please download the blood pressure log on our website [www.accumed.ch](http://www.accumed.ch)

### Display Explanations

**EE / Measurement Error:** Make sure the T-Plug is securely connected to the air socket and measure again quietly. Wrap the cuff correctly and return the device to your local distributor or service center. If the error keeps occurring, return the device to your local distributor or service center.

**E1 / Air Circuit Abnormality:** Make sure the T-Plug is securely connected to the air socket on the side of the unit and measure again quietly. If the error still occur, return the device to your local distributor or service center for help.

**E2 / Pressure Exceeding 300 mmHg:** Switch the unit off and measure again quietly. If the error keeps occurring, return the device to your local distributor or service center.

**E3 / Data Error:** Remove the batteries, wait for 60 seconds, and re-load. If the error keeps occurring, return the device to your local distributor or service center.

### Irregular Heartbeat (IHB) Detector

This unit is equipped with an Irregular Heartbeat (IHB) Detector which allows those who have an irregular heartbeat to obtain accurate measurements alerting the user of the presence of an irregular heart beat during the measurement.

Note: It is strongly recommended that you consult your physician if the IHB icon appears often (👤).

### Installing Batteries

- Press down and lift the battery cover in the direction of the arrow to open the battery compartment.
- Install or replace 4 "AAA" sized batteries in the battery compartment according to the indications inside the compartment.
- Replace the battery cover by clicking in the bottom hooks first, then push in the top end of the battery cover.
- Replace the batteries in pairs. Remove batteries when unit is not in use for extended periods of time.

You need to replace the batteries when:

- low battery icon appears on display.
- The ON/OFF key is pressed and nothing appears on display. Caution: Batteries are hazardous waste. Do not dispose them together with the household garbage.
- There are no user serviceable parts inside. Batteries or damage from old batteries are not covered by warranty.
- Use exclusively brand batteries. Always replace with new batteries together. Use batteries of the same brand and same type.

### Applying the Cuff

- Unwrap the arm cuff, leaving the end of the cuff through the D-ring of the cuff.
- Put your left arm through the cuff loop. The color strip indication should be positioned closer to you with the tube pointing in the direction of your arm (Fig. ❷). Turn your left palm upward and place the edge of the arm cuff at approximately 1.5 to 2.5 cm above the inner side of the elbow joint (Fig. ❸). Tighten the cuff by pulling the end of the cuff.

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[www.accumed.ch](http://www.accumed.ch)

- Center the tube over the middle of the arm. Press the hook and loop material together securely. Allow room for 2 fingers to fit between the cuff and your arm. Position the artery mark (❷) over the main artery (on the inside of your arm) (Fig. ❷, ❸). Note: Locate the main artery (on the inside of your left arm). Identify where the pulse can be felt the strongest. This is your main artery.
- Plug in the cuff connecting tube into the unit (Fig. ❸).
- Lay your arm on a table (palm upward) so the cuff is at the same height as your heart. Make sure the tube is not kinked (Fig. ❸).
- This cuff is suitable for the right if the arrow falls within the solid color line as shown on the right (Fig. ❸). If the arrow falls outside the solid color line, you will need a cuff with other circumferences. Contact your local dealer for additional size cuffs.

### Measurement Procedures

- Here are a few helpful tips to help you obtain more accurate readings:
- Blood pressure changes with every heartbeat and is in constant fluctuation throughout the day.
  - Blood pressure recording can be affected by the position of the user, his or her physiological condition and other factors. For greatest accuracy, wait one hour after exercising, bathing, eating, drinking beverages with alcohol or caffeine, or smoking to measure blood pressure.

- Before measurement, it's suggested that you sit quietly for at least 5 minutes as measurement taken during a relaxed state will have greater accuracy. You should not be physically tired or exhausted while taking a measurement.
- Do not take measurements if you are under stress or tension.
- During measurement, do not talk or move your arm or hand muscles.
- Take your blood pressure at normal body temperature. If you are feeling cold or hot, wait a while before taking a measurement.
- If the monitor is stored at very low temperature (near freezing), have it placed at a warm location for at least one hour before using it.
- Wait 5 minutes before taking the next measurement.

- Press the ON/OFF key. All displays will appear for approximately two seconds before returning to "0". The unit is ready for measurement at this point.
- Press the bulb to increase pressure in the arm cuff. The display will show the pressure reading within the cuff. Continue pumping until the pressure reaches approximately 50-60 mmHg above your normal systolic pressure. If your blood pressure is higher than 180 mmHg and the pressure in cuff is not enough, an upward arrow (▲) will display to remind you that you should continue pumping the pressure to more than 200 mmHg.
- Sit still and wait for the monitor to deflate and measure your blood pressure. It is important to remain still and quiet during measurement. Any significant movement may affect measurement results.
- Upon completion of the measurement cycle, the monitor will display systolic pressure, diastolic pressure and pulse rate readings at once. Measurement is now completed, and the reading is automatically stored in the monitor.
- Press the ON/OFF key to rapidly deflate and turn off the power. If no key is pressed, the unit will shut off automatically in 1 minute. Note: This monitor automatically switches off approximately 1 minute after last key operation. To interrupt the measurement, simply press the Memory key or ON/OFF key; the cuff will deflate immediately. During the measurement, do not talk or move your arm or hand muscles.

### Recalling Values from Memory

- To recall stored blood pressure readings from memory, simply press the Memory key, and then the last set of memorized readings will be displayed.

- Another press of the Memory key will recall the previous set of readings.
- All readings stored in memory will be displayed with its sequence number. Every measurement comes with an assigned memory sequence number.

### Clearing Values from Memory

Press and hold the Memory key for approximately 5 seconds, then the data in the memory zone can be erased automatically.

### Troubleshooting

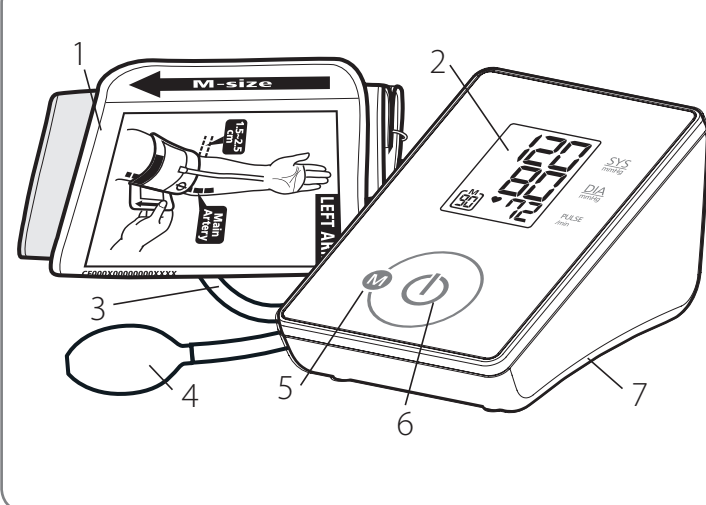
If any abnormality will arise during use, please check the following points.

| Symptoms                                                                                 | Check Points                                                                                                                                                                       | Correction                                                                                                                                               |
|------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------|
| No display when the ON/OFF key is pressed                                                | Have the batteries run down?<br>Have the batteries' polarities been positioned incorrectly?<br>Is the cuff placed correctly?                                                       | Replace them with four new batteries.<br>Re-insert the batteries in the correct positions.<br>Wrap the cuff properly so that it is positioned correctly. |
| EE mark shown on display or the blood pressure value is displayed excessively low (high) | Is the cuff pressure pumped enough? (50-60 mmHg above normal systolic value)?<br>Did you talk or move during measurement?<br>Did you vigorously shake the cuff during measurement? | Measure it again and pump up 50-60 mmHg more than normal systolic value.<br>Measure it again.<br>Keep arm steady during measurement.                     |

Note: If the unit still does not work, return it to your dealer. Under no circumstance should you disassemble and repair the unit by yourself.

### Cautionary Notes


- The unit contains high-pressure assemblies. Therefore, avoid extreme temperatures, humidity, and direct sunlight. Avoid dropping or strongly shocking the main unit, and protect it from dust.
- Clean the blood pressure monitor body and the cuff carefully with a slightly damp soft cloth. Do not press. Do not wash the cuff or the chemical cleaner on it. Never use thinner, alcohol or petrol (gasoline) as cleaner.
- Leaky batteries can damage the unit. Remove the batteries when the unit is not used for a long time.

- EN
- Arm Cuff
  - LCD Display
  - Air Tube and Connector,
  - Inflating Bulb
  - Memory Key
  - ON/OFF/START Key
  - Battery Cover

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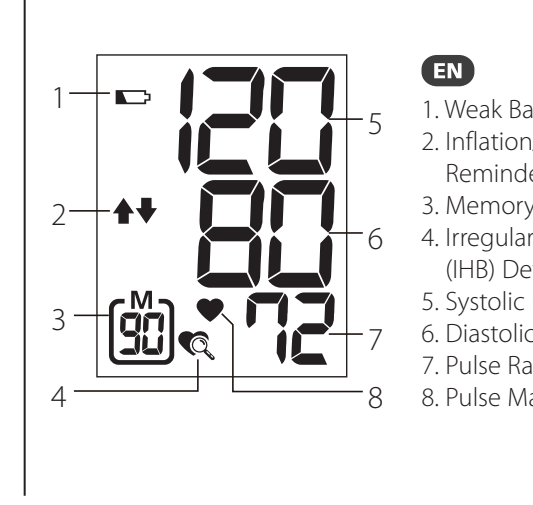
- EN
- Weak Battery Mark
  - Inflation/deflation Reminder
  - Memory Mark
  - Irregular Heartbeat (IHB) Detector
  - Systolic Pressure
  - Diastolic Pressure
  - Pulse Rate
  - Pulse Mark

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| Blood Pressure Standard (JNC7: 2003, unit: mmHg) |                   |     |                    |
|--------------------------------------------------|-------------------|-----|--------------------|
|                                                  | Systolic Pressure | and | Diastolic Pressure |
| Normal                                           | <120              |     | <80                |
| Suspected Hypertension                           | 120~139           | or  | 80~89              |
| Suspected Stage 1 Hypertension                   | 140~159           | or  | 90~99              |
| Suspected Stage 2 Hypertension                   | ≥160              | or  | ≥100               |

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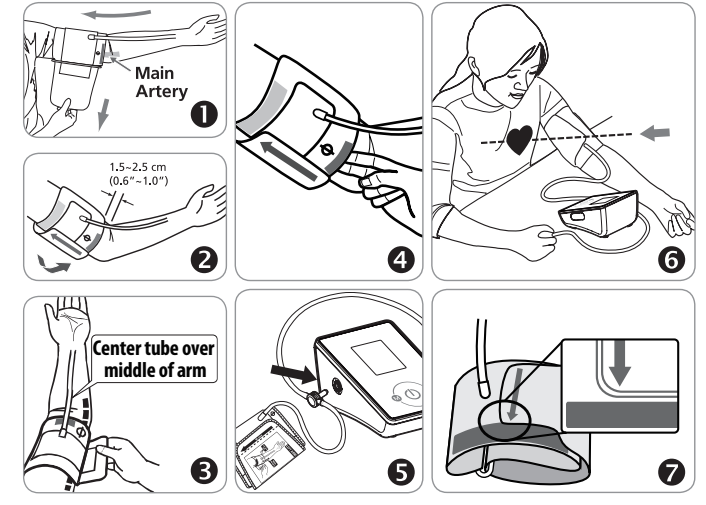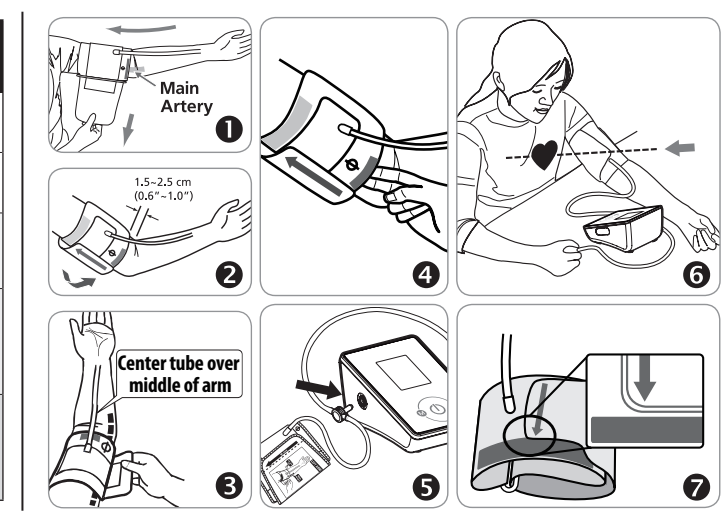
|                                                                              |                        |  |  |
|------------------------------------------------------------------------------|------------------------|--|--|
| <b>Warranty Certificate (Must be completed within 10 days of purchase)</b>   |                        |  |  |
| <b>Customer Name:</b>                                                        | <b>City:</b>           |  |  |
| <b>Street Address:</b>                                                       | <b>ZIP code:</b>       |  |  |
| <b>State:</b>                                                                | <b>E-mail address:</b> |  |  |
| <b>Telephone:</b>                                                            |                        |  |  |
| <b>Gender: Male <input type="checkbox"/> Female <input type="checkbox"/></b> | <b>Age:</b>            |  |  |
| <b>Product Information:</b>                                                  |                        |  |  |
| <b>Date of purchase:</b>                                                     |                        |  |  |
| <b>Store where purchased:</b>                                                |                        |  |  |
| <b>Price Paid (Tax):</b>                                                     |                        |  |  |

This instrument is covered by a 3 year guarantee from the date of purchase. The guarantee is valid only on presentation of the guarantee card completed by the dealer confirming date of purchase or the receipt. Batteries, cuff and wearing parts are not included. Opening or altering the instrument invalidates the guarantee. The guarantee does not cover damage, accidents or non-compliance with the instruction manual. Please contact your distributor.

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