TOPCOM®

BPM WRIST 2511

**USER GUIDE
HANDLEIDING
MODE D'EMPLOI
BEDIENUNGSANLEITUNG
ANVÄNDARHANDBOK
BRUGSANVISNING
ANVÄNDARHANDBOK
KÄYTTÖOHJE
MANUAL DE USUARIO
V 1.1**

CE 0366

Exclusively produced for Topcom Europe NV,
Grauwmeer 17, 3001 Heverlee, Belgium
by Rossmax Int'l Ltd.
N° 6018 Huyi Rd., Waigang, Jiading District,
Shanghai China 201806

This product is in compliance with the essential requirements and other relevant provisions of the directive 93/42/EEC and 89/336/EEC.
The Declaration of conformity can be found on :

<http://www.topcom.net/support/cedclarations.php>

- UK** The features described in this manual are published with reservation to modifications.
- NL** De in deze handleiding beschreven mogelijkheden worden gepubliceerd onder voorbehoud van wijzigingen.
- FR** Les possibilités décrites dans ce manuel sont publiées sous réserve de modifications.
- DE** Die in dieser Bedienungsanleitung umschriebenen Möglichkeiten, werden vorbehaltlich Änderungen publiziert.
- ES** Las características descritas en este manual pueden ser objeto de futuras modificaciones.
- SE** Funktionerna i denna bruksanvisning publiceras med reservation för ändringar.
- DK** Vi forbeholder os retten til ændringer af de specifikationer, der er beskrevet i denne brugsanvisning.
- NO** Funksjoner beskrevet i denne manualen kan endres uten nærmere informasjon.
- FI** Tässä ohjekirjassa julkaissut tiedot voivat muuttua ilman ennakkoilmoitusta.

1 Introduction

Congratulations with your purchase of the Topcom BPM Wrist 2511. This fully automatic, wrist mounted blood pressure monitor is both easy to use and ideally suited for daily measurements. The large display includes systolic, diastolic and pulse, all clearly shown at the completion of each reading.

In addition, you can store up to 50 measurements per Memory Zone, ideal for users who want to monitor and track their blood pressure on a regular basis. The BPM wrist 2511 is compact and portable, making it ideal for home use and travel.

Please read this manual carefully before use. For specific information on your own blood pressure, contact your physician. Put the user guide away in a safe place for future reference.

2 Safety instructions

1. This product is for household use only. It is not intended as a substitute for consultation with your doctor.
2. For users diagnosed with common arrhythmia (atrial or ventricular premature beats or atrial fibrillation), diabetes, poor circulation of blood, kidney problems, or for users suffered from stroke, or for unconscious users, the device might not be suitable for use. Consult your doctor in case of any doubt.
3. The unit should not be operated by children so to avoid hazardous situations.
4. The unit contains high-precision assemblies. Therefore, avoid extreme temperatures, humidity, and direct sunlight. Avoid dropping or strongly shocking the main unit, and protect it from dust.
5. Leaky batteries can damage the unit. Remove the batteries when the unit is not used for a long time.
6. Do not press the START key when the cuff is not placed around the wrist.
7. Do not disassemble the main unit or cuff.
8. If the unit is stored in a cold place, allow it to acclimate at room temperature before use.
9. Clean the blood pressure monitor body and the cuff carefully with a slightly damp, soft cloth. Do not press. Do not bend the pre-formed cuff inside out. Do not wash the cuff or use chemical cleaner on it. Never use thinner, alcohol or petrol (gasoline) as cleaner.
10. To ensure accurate measurements, it is recommended that calibration be completed every two years.

3 Helpfull tips

Here are a few helpful tips to help you obtain more accurate readings:

- Do not measure your blood pressure immediately after consuming a large meal. To obtain more accurate readings, please wait one hour before measuring.
- Do not smoke or drink alcohol before measuring your blood pressure.
- You should not be physically tired or exhausted while taking a measurement.
- It is important that you relax during measurement. Try to take 15-minutes rest before a reading.
- Do not take measurements if you are under stress or under tension.
- Take your blood pressure at normal body temperature. If you are feeling cold or hot, wait a while before taking a measurement.
- If the monitor is stored at very low temperature (near freezing), have it placed at a warm location for at least one hour before using it.
- Wait about 5 minutes before taking the next pressure measurement.

4 Disposal of the device

At the end of the product lifecycle, you should not throw this product into the normal household garbage but bring the product to a collection point for the recycling of electrical and electronic equipments. The symbol on the product, user guide and/or box indicate this.

Some of the product materials can be re-used if you bring them to a recycling point. By re-using some parts or raw materials from used products you make an important contribution to the protection of the environment. Please contact your local authorities in case you need more information on the collection points in your area.

5 Blood pressure

5.1 What is Blood Pressure?

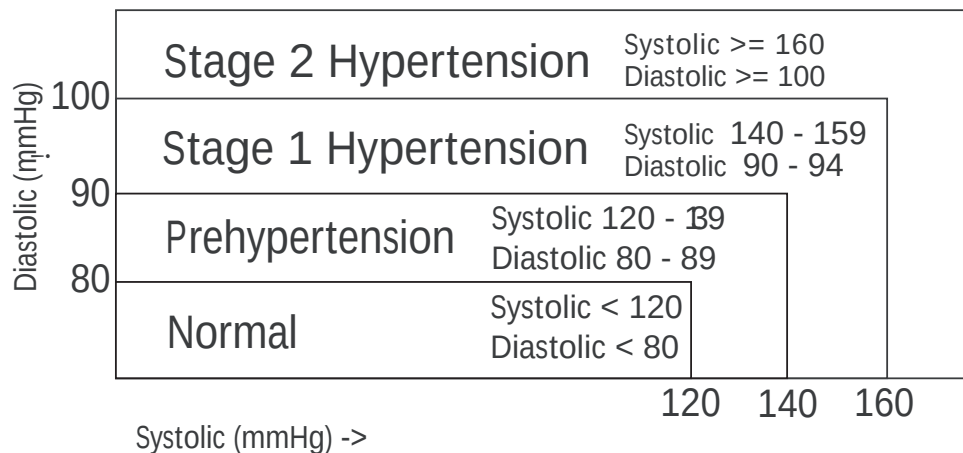
Blood pressure is the pressure exerted on the artery tube while blood flows through the arteries. The pressure measured when the heart contracts and sends blood out of the heart is systolic (highest). The pressure measured when the heart dilates with blood flowing back into the heart is called diastolic (lowest) blood pressure.

5.2 Why Measure Your Blood Pressure?

Among the various health problems afflicting modern people, problems associated with high blood pressure are by far the most common. High blood pressure's dangerously strong correlation with cardiovascular diseases and high morbidity has made measuring blood pressures a necessity of identifying those at risk.

5.3 Blood Pressure Standard

The World Health Organization (WHO) and National High Blood Pressure Education Program has developed a Blood pressure standard, according to which areas of low and high-risk blood pressure are identified. This standard however, is a general guideline as individual's blood pressure varies among different people and different groups...etc.


It is important that you consult with your physician regularly. Your physician will tell you your normal blood pressure range as well as the point at which you will be considered at risk.

5.4 Blood Pressure Fluctuation


Blood pressure fluctuates all the time!

You should not be worried if you encountered two or three measurements at high levels. Blood pressure changes over the month and even throughout the day depending on circumstances (state of mind, temperature, ...)

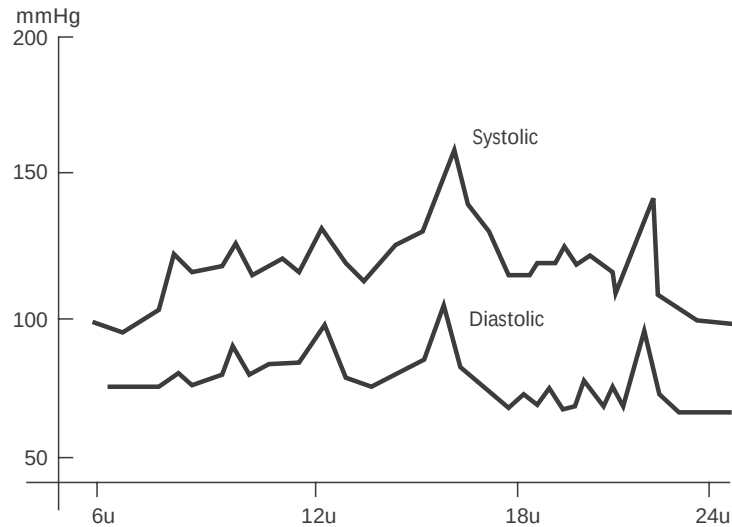Ex. Fluctuation within the day for a Male, 35 years old.

6 Battery installation

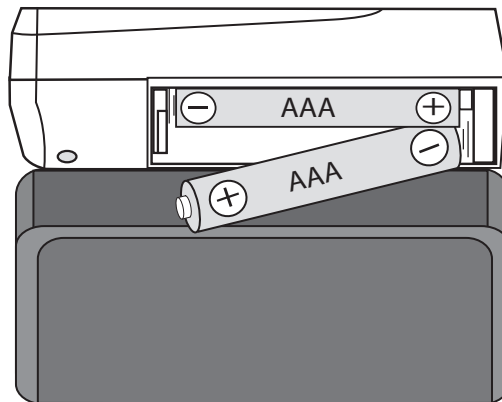
- Open the battery compartment by pulling the cover away
- Insert 2 AAA non - rechargeable batteries. Follow the polarity as indicated in the battery compartment.


Do not shortcircuit & dispose of in fire. Remove the battery if this device is not going to be used for a long period.



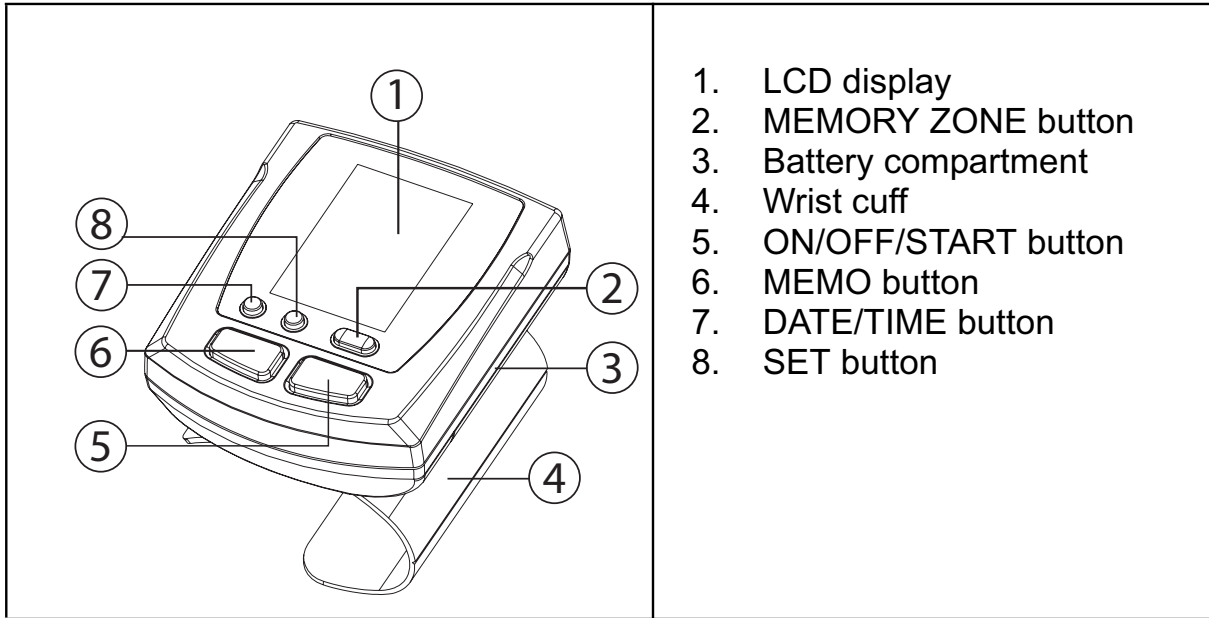
7 Buttons

8 Display



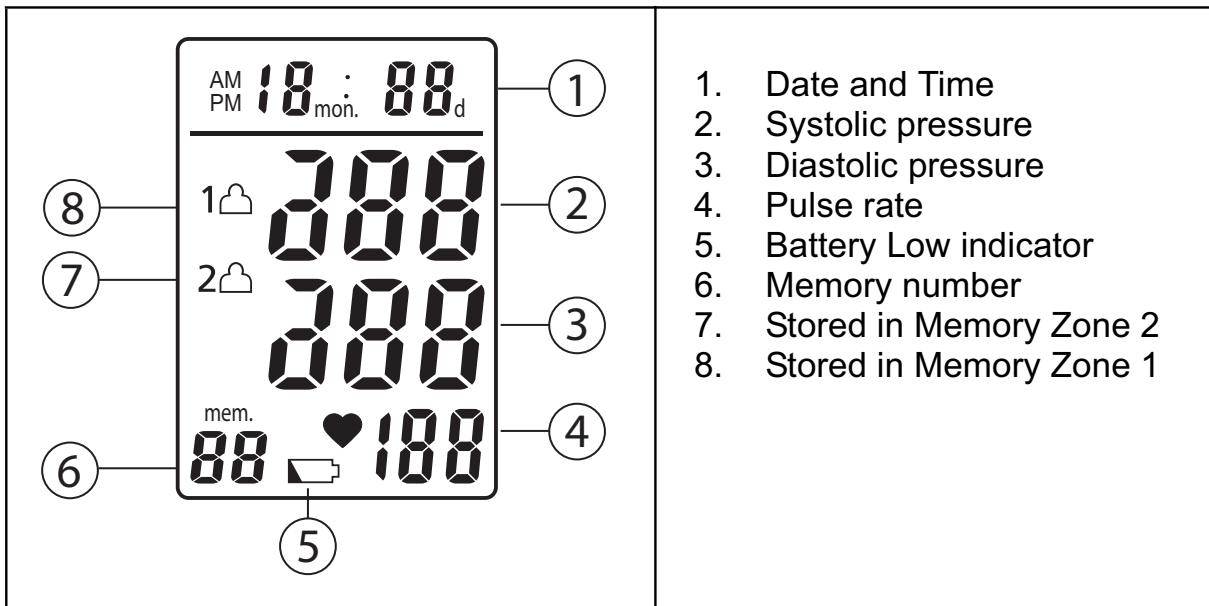
9 Applying the cuff

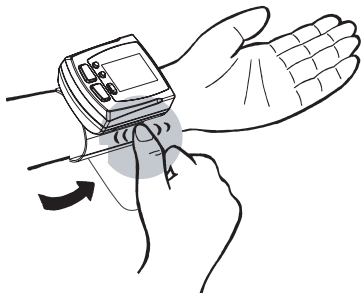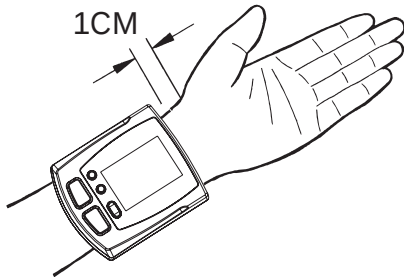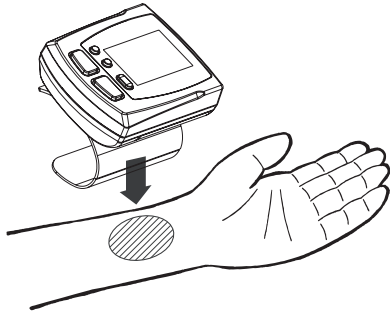1. Remove all watches, jewelry, etc. prior to attaching the wrist monitor. Clothing sleeves should be rolled up and the cuff should be wrapped on bare skin for correct measurements. Apply cuff to left wrist with palm facing up.
2. Make sure the edge of the cuff is about 1 cm from the palm. In order to insure accurate measurements.
3. Fasten the velcro strap securely around your wrist so there is no extra space between the cuff and the wrist. If the cuff is not wrapped tight enough, the measurement values will be false.
4. If your physician has diagnosed you with poor circulation on your left arm, place the cuff around your right wrist.

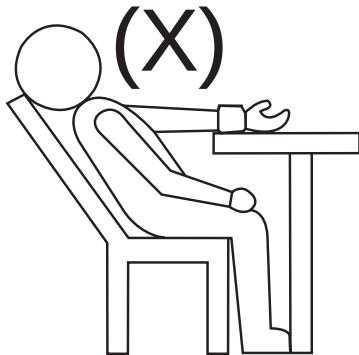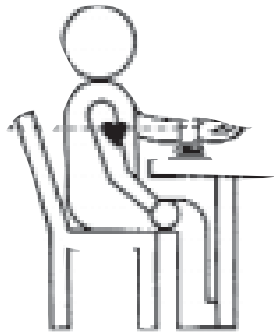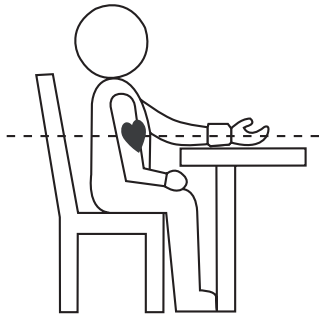10 Measuring posture

1. Relax your entire body, especially the area between your elbow and fingers. Place your elbow on a table so that the cuff is at the same level as your heart.

Your heart is located slightly below your arm pit.

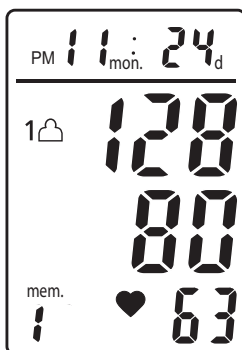
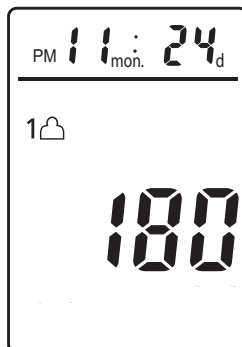
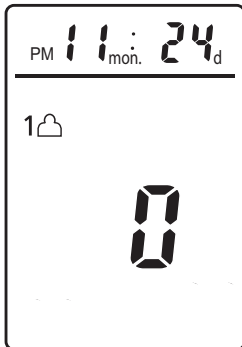
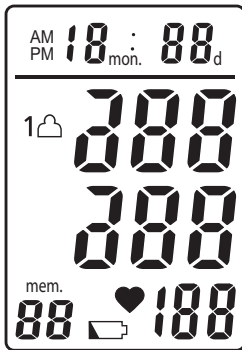
2. If the cuff is not at the same level as your heart or if you can not keep your arm completely still throughout the reading, use a soft object such as folded towel to support your arm.
3. Turn your palm upwards. Sit upright in a chair, and take 5-6 deep breaths. Avoid leaning back while the measurement is being taken.

11 Measurement

11.1 Important

1. This monitor automatically switches off 1 minute after the last key operation.
2. To interrupt the measurement, simply press any key. The cuff will deflate immediately after a key is pressed.
3. During the measurement, do not talk or move your arm or hand muscles.
4. The result will be stored automatically in the selected Memory Zone. Select the desired Memory Zone before measurement (see §12)

11.2 Procedure

81

1. Press the POWER button. All digits will light up, checking the display functions. The checking procedure will complete after about 2 seconds.
2. After all symbols appear, the display will show "0" with the designated Memory Zone.
3. After initialisation the monitor automatically inflates the cuff to approximately 180 mmHg. The measurement will begin.

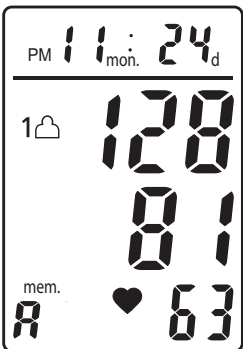
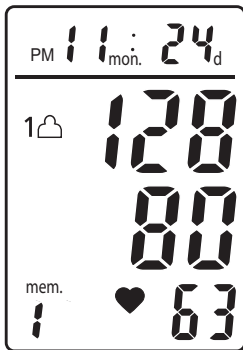
This monitor will re-inflate automatically to approximately 220mmHg if the system detects that your body needs more pressure to measure your blood pressure.

4. When the measurement is completed, systolic, diastolic and pulse will be shown simultaneously on the LCD screen.

When a mistake was detected during measurement the Error mark appears on the display

12 Memory

The BPM 2511 has two Memory zones. Each can store up to 50 measurements including the date and time

The result will be stored automatically in the selected Memory Zone after the measurement.

Select the desired Memory Zone before measurement.

- Press the MEMORY ZONE button to activate the display. The actual selected Memory Zone is displayed.
- Press the Memory Zone button again to select the memory zone 1 or 2 in which you want to store the measurement result.
- Press the POWER button to switch the unit off.

To recall measurement results:

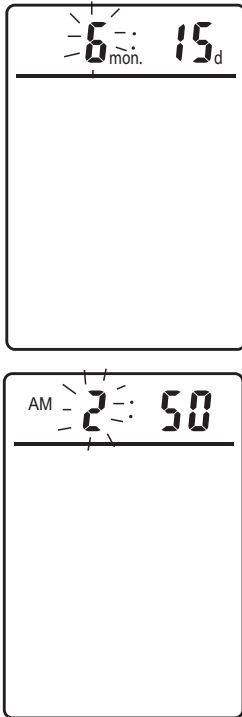
- Press the MEMO button to activate the display.
- Press the MEMORY ZONE button to select the memory zone 1 or 2 from which you want to recall the measurement result.
- Press the MEMO button to confirm. The calculated average of all measurement together is displayed.
- Press the MEMO button again to see the last measurement result.
- Press the MEMO successively for prior values stored.

When the batteries are installed, the data in the memories will not be lost even when the Blood Pressure Monitor is switched off.

Remove the batteries for more than 10 seconds to erase the memories.

13 Time adjustment

To adjust the date and time:

- Press the DATE/TIME button, the month will blink on the display.
- Press the SET button successively to change the month.
- Press the DATE/TIME button to confirm the month. The day will blink on the display.
- Change the day, hours and minutes as described above, using the SET button to change and the DATE/TIME button to confirm every setting.
- After setting the minutes, "0" will be displayed.
- Press the POWER button to switch the unit off.

14 Technical specifications

Power source	3V DC: Two LR03 (AAA) Non rechargeable batteries
Measurement method	Oscillometric
Measurement zone	Pressure: 40~250 mmHg Pulse: 40~199 beats/minute
Accuracy	Pressure: ± 3 mmHg Pulse: within $\pm 5\%$ of reading
Pressure sensor	Semi-conductor
Inflation	Pump driven
Deflation	Automatic Pressure release valve
Memory capacity	50 memories for each zone
Auto-shut-off	1 minute after last key operation
Operation environment	10°C~40°C (50°F~104°F) 40~85% RH max.
Storage environment	-10°C~60°C (14°F~140°F) 10~95% RH max.
Weight	120g (without batteries)
Wrist circumference	13.5~22 cm (5.3"~7.7")
Limited patient	Over 18 years old
Dimensions	82(L) x 69(W) x 66(H) mm
K.B.	Short time operation 2 minutes
Classification	Classification of type BF

*Specifications are subject to change without notice.

15 Topcom Warranty

15.1 Warranty period

The Topcom units have a 24-month warranty period. The warranty period starts on the day the new unit is purchased. Consumables or defects causing a negligible effect on operation or value of the equipment are not covered. The warranty has to be proven by presentation of the original purchase receipt, on which the date of purchase and the unit-model are indicated.

15.2 Warranty handling

A faulty unit needs to be returned to a Topcom service centre including a valid purchase note.

If the unit develops a fault during the warranty period, Topcom or its officially appointed service centre will repair any defects caused by material or manufacturing faults free of charge.

Topcom will at its discretion fulfil its warranty obligations by either repairing or exchanging the faulty units or parts of the faulty units. In case of replacement, colour and model can be different from the original purchased unit.

The initial purchase date shall determine the start of the warranty period. The warranty period is not extended if the unit is exchanged or repaired by Topcom or its appointed service centres.

15.3 Warranty exclusions

Damage or defects caused by incorrect treatment or operation and damage resulting from use of non-original parts or accessories not recommended by Topcom are not covered by the warranty.

The warranty does not cover damage caused by outside factors, such as lightning, water and fire, nor any damage caused during transportation.

No warranty can be claimed if the serial number on the units has been changed, removed or rendered illegible.

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