SMARTHEART™

Automatic Digital Blood Pressure Arm Monitor

Model # 01-554


INSTRUCTION MANUAL **ENGLISH AND SPANISH**

Please read this instruction manual
completely before operating this unit.

STOP!
PLEASE ENSURE YOU HAVE ALL OF THE
FOLLOWING COMPONENTS BEFORE USING
YOUR DIGITAL BLOOD PRESSURE MONITOR


Digital Monitor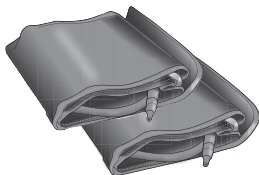

Adult Cuff
Large Adult Cuff

Storage Case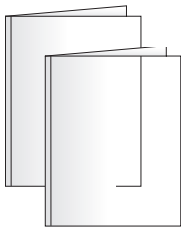

Instruction Manual
Quick Start Guide
Blood Pressure Reading Tracking Log
FAQ Information Sheet
Replacement Parts Order Form

IF YOU ARE MISSING ANY PARTS, INCLUDING INSERTS OR INSTRUCTION MANUALS, DO NOT RETURN TO PLACE OF PURCHASE. CONTACT CUSTOMER CARE AT 866-326-1313.

INDEX

Care & Safety Information	4-5
Introduction & Indications for Use	6
Digital Blood Pressure Monitor Features	7
Getting Started	8
Battery Installation	9
Setting the Date and Time	10-11
Applying Your Blood Pressure Cuff	12-13
Taking Your Blood Pressure Reading	14-15
Interpreting Your Results	16-17
Memory Recall	18-19
Care and Maintenance	20-21
Display Symbols	22-23
Troubleshooting and Error Codes	24-25
FCC Statement	26
Electromagnetic Compatibility	27-29
Specifications	30
Warranty	31
Instrucciones en Español	33-63

Toll-Free Customer Care Help Line: 1-866-326-1313
Monday – Friday 8:30 a.m. – 4:30 p.m. CST

SmartHeart™
Manufactured for
Veridian Healthcare
1175 Lakeside Drive
Gurnee, IL 60031

Made in China

#93-1305 09/16
©2016 Veridian Healthcare, LLC

CARE & SAFETY INFORMATION

NOTE: Read all instructions carefully before use. The following basic precautions are needed when using an electrical product.

CAUTION: Failure to read and observe all precautions could result in personal injury or equipment damage.

Improper care or use of your blood pressure monitor may result in injury, damage to the unit or ineffective treatment. Following these instructions will ensure the blood pressure monitor's efficacy and long life.

GENERAL CAUTIONS AND WARNINGS

- Measurement results should be discussed with your physician or healthcare professional; never self-diagnose or attempt treatment as this may be dangerous
- Intended for adult use only; this device is not approved for infant or child use
- Keep out of reach of children
- Do not use the unit if it has any damaged parts (including power cord or plug and cuff), if it has been submersed in water or dropped
- If any abnormality occurs, discontinue use until the unit has been examined and repaired
- Only use included attachments and parts; do not use attachments from other brands or models as these may not be properly calibrated for use with this device and may result in measurement error
- Do not disassemble the unit or attempt to repair it; substitution of a component different from that supplied may result in measurement error and will void manufacturer warranty
- Always follow local regulations for proper disposal of the monitor, cuff and batteries
- Do not use this device with other diagnostic monitoring equipment
- Take Care to note the blood circulation of the user and discontinue use if prolonged discomfort occurs

CARE & SAFETY INFORMATION

- It is inappropriate for people with serious arrhythmia to use this Electronic Sphygmomanometer
- Do not share this device with infected persons where there may be a risk of cross-contamination
- Consult with your physician before use if any of the following conditions are present: arrhythmia, inflammatory diseases, intravascular therapy, arteriovenous shunts or if a mastectomy has been performed.

OPERATING CAUTIONS AND WARNINGS

- Prolonged over-inflation may cause congestion, swelling or bruising in some people
- If you experience any discomfort or an abnormality occurs, stop using the device immediately; to stop the device during operation, press the **POWER** button and the cuff will automatically deflate
- Operation outside of stated operating temperature may result in measurement error or device malfunction; operation environment temperature is: 50°F – 104°F (10°C – 40°C); Humidity: ≤85%RH max
- Never use this unit while operating a vehicle or in the bathtub or shower

STORAGE CAUTIONS AND WARNINGS

- Storage outside of stated storage temperature may result in measurement error or device malfunction; storage environment temperature is: -4°F – 122°F (-20°C – 50°C); Humidity: 85%RH max
- Keep the unit out of reach of small children
- Always keep the unit unplugged while not in use; remove the batteries if the unit will not be used for an extended period of time

CLEANING CAUTIONS AND WARNINGS

- Never immerse the unit in water to clean as it may damage the unit
- Follow the 'Cleaning and Maintenance' portion of this manual for instruction on how to clean and care for your monitor

INTRODUCTION & INDICATIONS FOR USE

It is recommended that you first seek the advice and recommendation of your physician or healthcare professional when using home diagnostic devices, including blood pressure monitors.


Automatic digital blood pressure monitors use the oscillometric method to electronically measure your blood pressure. The monitor detects your blood's movement through the artery in your arm and converts the movements into a digital reading. The oscillometric method does not require a stethoscope, making the monitor ideal for home use.

Blood pressure readings determined with this device are equivalent to measurements obtained by a trained healthcare professional using the cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard for Electronic or Automated Sphygmomanometers.

Thank you for purchasing a SmartHeart Automatic Digital Blood Pressure Monitor. With proper care and use, your monitor will provide you with many years of reliable readings.

**PLEASE READ THIS INSTRUCTION MANUAL
COMPLETELY BEFORE OPERATING THIS UNIT.**

BLOOD PRESSURE MONITOR FEATURES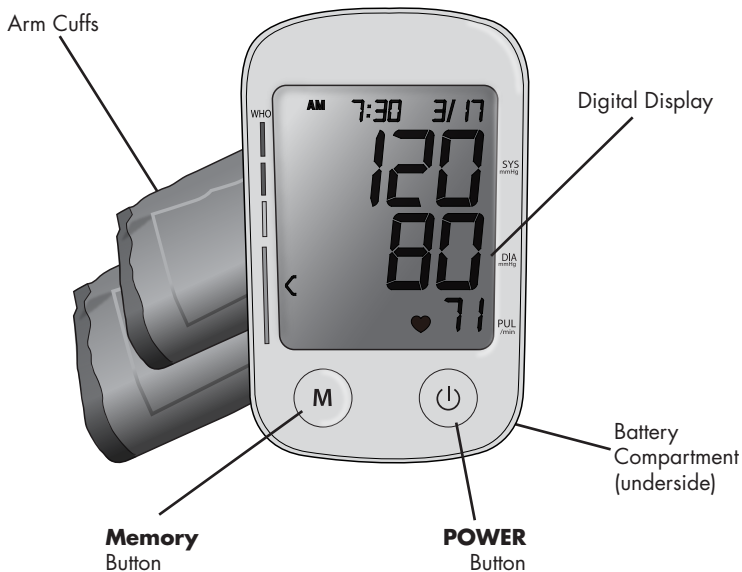

DETAILS OF THE DIGITAL DISPLAY SCREEN AND INTERPRETATION OF THE SYMBOLS ARE INCLUDED THROUGHOUT THIS MANUAL.

GETTING STARTED

Before Taking a Measurement

1. Your readings should only be interpreted by your physician or healthcare professional with access to your individual medical history. Regular use of a home blood pressure monitor will allow you to track and record your readings for discussion with your physician.
2. Conduct your measurement in a quiet place while seated in a relaxed position. Rest for 15 minutes before taking your reading. If an error occurs or you desire to take a second reading, allow 15 minutes between readings for your blood vessels to return to normal.
3. Avoid smoking, eating, taking medication, alcohol, physical activity or any other stressful activity for 30 minutes prior to taking a reading.
4. Always remove any jewelry or constrictive clothing that may interfere with the cuff placement.
5. Keep yourself and the monitor still during measuring; do not talk during the reading.
6. It is recommended that you take your readings at the same time each day to better monitor any indications in your results.
7. Record your daily measurement on the included chart or some other written document to share with your physician.

BATTERY INSTALLATION

This unit requires 4 AAA batteries.

It is necessary to replace the batteries when the “Low Battery” symbol appears on the display or when the display does not turn on after the **POWER** button is pressed.


The ‘Low Battery’ symbol will appear on the bottom of the display.

The battery compartment is located on the underside of this monitor.

1. Depress the clip on the battery compartment cover and remove the cover.
2. Insert or replace 4 AAA batteries into the battery compartment, ensuring to match the indicated polarity symbols. Always use new batteries.
3. Replace the battery cover.
4. Dispose of batteries according to local disposal and recycling regulations.

It is recommended to remove the batteries if the unit will not be used for an extended period of time.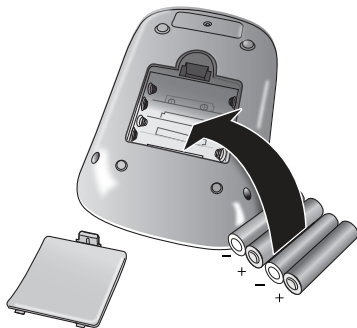

SETTING DATE/ TIME

IT IS NECESSARY TO SET THE DATE AND TIME FOR THE UNIT EVERY TIME BATTERIES ARE INITIALLY INSTALLED OR REPLACED

The date/time feature is on the top of the display screen. This unit stores the date and time, including year allowing for a date/time stamp on all readings in the memory recall function.

The **M** and **POWER** buttons are located below the display.

M -Memory button

-POWER button


1. When the batteries are first installed, the device will automatically prompt you to set the Date/Time feature.*

NOTE: If the batteries have already been installed—when the unit is off (a blank display screen), press and release the **MEMORY & POWER button for 3-5 seconds to make the unit go into clock setting mode.*

2. **Military/ Standard Clock-** When you first enter into the clock setting mode you have the option to choose between a 24 or 12 hour clock. Press the **M** button to toggle between the two settings. Once a selection is made press the **POWER** button to advance to the next step.
3. **YEAR**—next the year will flash on the display. Press the **M** button to advance the display to the desired year, press the **POWER** button to confirm your selection and advance to the date settings.

SETTING DATE/ TIME

4. **MONTH/DATE**—repeat the same sequence of step 2 to confirm the MONTH and DATE setting.
5. **HOURS/MINUTES**—repeat the same sequence of step 2 to confirm the HOUR and MINUTE setting.
6. After the Date/Time setting function is complete, the monitor is ready for use.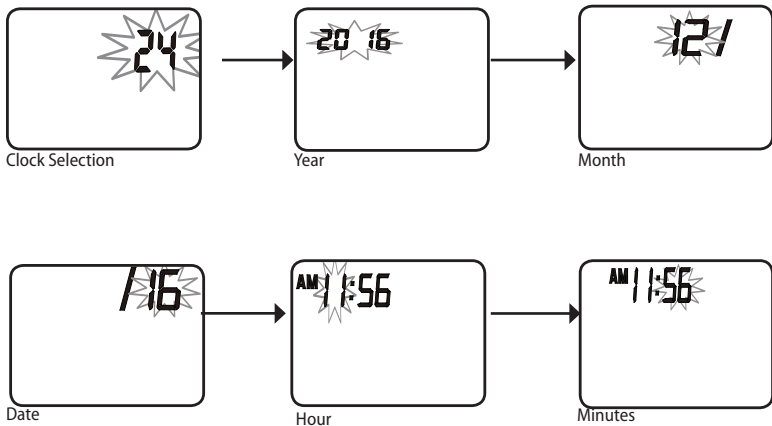


FITTING AND APPLYING YOUR CUFF

PROPER CUFF FIT AND PLACEMENT IS CRITICAL IN OBTAINING ACCURATE BLOOD PRESSURE MEASUREMENTS. PLEASE READ THESE INSTRUCTIONS CAREFULLY AND CONSIDER CONSULTING WITH YOUR PHYSICIAN FOR FURTHER DIRECTION.

Before applying your blood pressure cuff, be sure you have selected the appropriate size cuff.

To determine the proper cuff size, measure the circumference of your left upper arm at the point midway between inside bend of your elbow and your shoulder.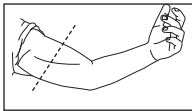

Adult Cuff fits arm circumference 8.6" – 11.8" (22–30 cm)

Large Adult Cuff fits arm circumference 11.8" – 16.5" (30–42 cm)

If for any reason you are unable to or should not use your left arm, please modify the instructions for cuff application to your right arm. Your physician can help you identify which arm is best for you to take measurements from.

Assembling Your Cuff

Your cuff may need to be 'threaded' through the metal bar to form the cuff cylinder for your arm.

1. Lay the open cuff hook and loop side down, size/instruction wording side up on a flat surface.
2. Keeping the words on the inside of the cuff, thread the end of the cuff through the metal bar, Fig 1. The end of the cuff should be able to fold over onto itself for securing via the hook and loop closure, Fig 2.

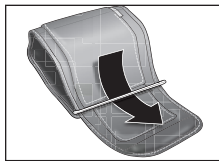
Fig. 1

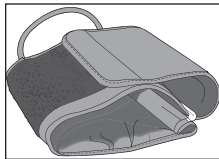
Fig. 2

FITTING AND APPLYING YOUR CUFF

Applying Your Cuff

Be seated at a table or desk with your feet flat on the floor. To avoid accidentally pulling on or dropping your monitor, the cuff should not be plugged into the monitor until after the cuff is applied to your arm.

The cuffs are printed with a useful guide for applying the cuff; please consult the following steps for additional direction.

1. Remove any constrictive clothing or jewelry that may interfere with cuff placement.
2. Slide the cuff cylinder onto your left arm and align the artery mark over the brachial artery on the inside of your arm, Fig 3.

The tube should follow the inside length of the arm, Fig 4.

3. The bottom edge of the cuff should be positioned approximately half an inch (0.5") above the elbow joint, Fig 5.
4. When the cuff is closed around the arm, the triangle guide imprint should be within the OK range bar marking; if the triangle is outside of the range, a different sized cuff is needed
5. The cuff should fit comfortably, yet snugly around your arm, Fig 6. You should be able to insert one finger easily between your arm and the cuff.

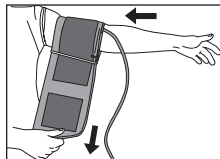
Fig. 3

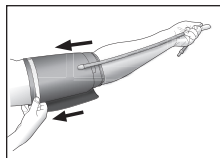
Fig. 4

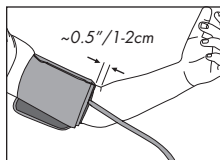
Fig. 5

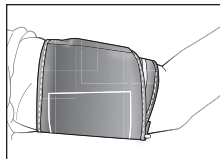
Fig. 6

TAKING YOUR BLOOD PRESSURE READING

Please read the preceding portions of this manual prior to taking your first reading.

Reminder, it is important to avoid smoking, eating, taking medication, alcohol consumption or physical activity 30 minutes prior to taking a reading.

1. Position the monitor on a flat, stable surface with the digital display in view. Be seated with your feet flat on the floor and legs uncrossed.
2. Insert the cuff tubing connector into the port on the left side of your monitor.
3. Rest your elbow on a solid surface with your palm facing upward. Elevate your arm so that the cuff is at the same level as your heart, Fig 1. Relax your left hand.
4. Press and release the **POWER** button to turn the unit on; the unit will briefly flash a self-test, Fig 2.
5. The display will show the current memory bank in use, U1 or U2, Fig 3. Press the **M** button to select a different memory bank if desired. If no selection is made, the unit will default to the memory bank displayed.
6. The display will briefly show a '0' before the measurement begins.
7. As the inflation process begins, the display may briefly show a downward arrow ∇ indicating the cuff is releasing any air remaining in the cuff prior to measurement inflation.

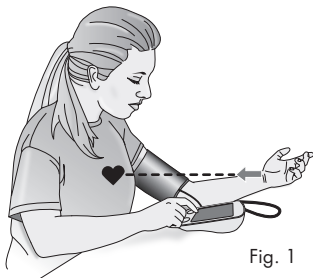
Fig. 1

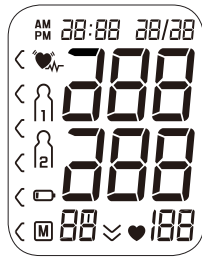
Fig. 2

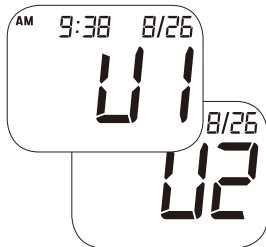
Fig. 3

TAKING YOUR BLOOD PRESSURE READING

YOU CAN STOP THE INFLATION OR DEFLATION PROCESS AT ANY TIME BY PRESSING THE POWER BUTTON.

8. The cuff will automatically inflate; you will see the numbers increase on the display screen and feel the cuff inflate.
9. The cuff will automatically deflate when the measurement is complete; then your blood pressure measurement and pulse reading results will show on the display screen at the same time, Fig 4.
10. The Hypertension Indicator will indicate your reading range on the left display in comparison to the color bars on the face of the monitor, Fig 5. See page 17 for information regarding interpreting the Hypertension Indicator results.
11. If an irregular heartbeat was detected during the reading, the Irregular Heartbeat Detector indicator will appear on the display, Fig 6. See page 16 for more information on the Irregular Heartbeat Detector feature.
12. The reading will automatically be stored in memory, up to 60 readings per user.
13. Select **POWER** to turn the unit off and conserve energy and battery life.
14. The unit will automatically shut-off approximately 1 minute after use.
15. Disconnect the cuff and tubing prior to storage.

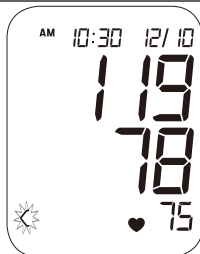
Fig. 4

Fig. 5



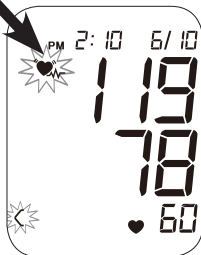
Fig. 6

INTERPRETING YOUR BLOOD PRESSURE READING

Irregular Heartbeat

An irregular heartbeat is defined as a heartbeat rhythm that is more than 25% slower or 25% faster than the average measurement detected while the monitor is conducting the measurement. When this device detects an irregular rhythm two or more times during the reading, the IHB symbol will appear on the display.

This monitor will operate if the icon appears, but the results may be compromised, especially if this symbol appears often. Please consult with your physician or trained healthcare professional for further information regarding an irregular heartbeat and if this symbol appears frequently.

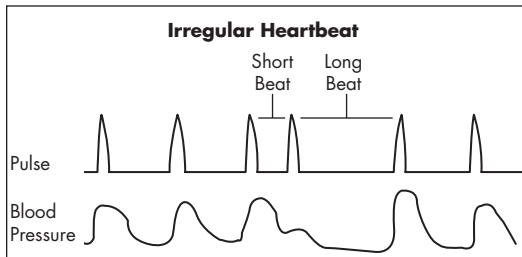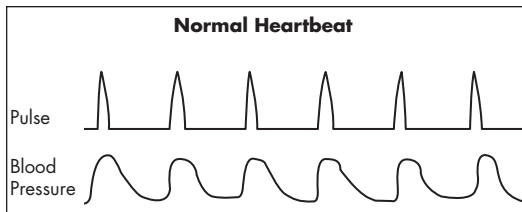INTERPRETING YOUR BLOOD PRESSURE READING

Hypertension Indicator - WHO Classifications

This unit features our unique Hypertension Indicator. The color bars on the left side of the monitor display correspond with an icon on the digital display, indicating where the measurement results fall within the World Health Organization standards.

This chart can help you identify where your blood pressure measurements fall according to WHO classification standards. This chart should only be considered as a guideline; always consult with your physician to interpret your individual results. Never adjust your medication(s) dosage, or severely alter your diet or exercise routine without consulting with your physician.

Stage 3 Hypertension

Systolic pressure: ≥ 180 mmHg
or
Diastolic pressure: ≥ 110 mmHg

Stage 2 Hypertension

Systolic pressure: $\geq 160 \leq 179$ mmHg
or
Diastolic pressure: $\geq 100 \leq 109$ mmHg

Stage 1 Hypertension

Systolic pressure: $\geq 140 \leq 159$ mmHg
or
Diastolic pressure: $\geq 90 \leq 99$ mmHg

High Normal

Systolic pressure: $\geq 130 \leq 139$ mmHg
or
Diastolic pressure: $\geq 85 \leq 89$ mmHg

Normal

Systolic pressure: $\geq 120 \leq 129$ mmHg
or
Diastolic pressure: $\geq 80 \leq 84$ mmHg

Optimal

Systolic pressure: < 120 mmHg
or
Diastolic pressure: < 80 mmHg

MEMORY FUNCTION

Recalling Measurements in Memory

This monitor stores and recalls up to 60 measurements per memory bank/user, and features 3 averaging modes—

- All readings in memory
- A.M. (morning) measurements
- P.M. (evening) measurements

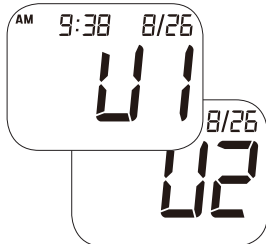
Fig. 1

1. When the unit is OFF, press and release the **M** button to get the memory bank. The display will show the current memory bank in use, U1 or U2. Press the **POWER** button to alternate banks. If no selection is made, the unit will default to the memory bank displayed
2. Press the **M** button again; next display will show the average of ALL results in the memory bank, an A will appear on the bottom left of the display, Fig 2.

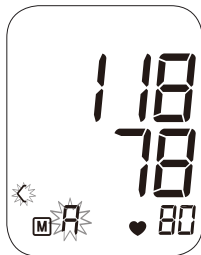
Fig. 2

3. Press **M** again to display the A.M. average, then the P.M. average, Fig 3.
 - A.M. is defined as measurements taken between 5:00 a.m. and 9 a.m.
 - P.M. is defined as measurements taken between 6:00 p.m. and 8 p.m.

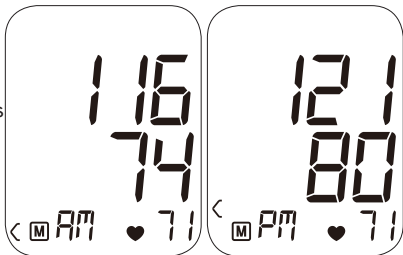
Fig. 3

MEMORY FUNCTION

- Continue to press the **M** button to scroll through the measurements in the bank from newest to oldest.
- Each memory bank stores up to 60 readings; when the number of readings exceeds 60, the oldest data will be replaced with the newest record.
- All results for a given measurement will display, including measurement results, pulse rate, Hypertension Indicator and Irregular Heartbeat alert (if applicable). The date/time stamp is on the top of the display.
- Press **POWER** to turn the monitor off at any time during review of the stored measurements.


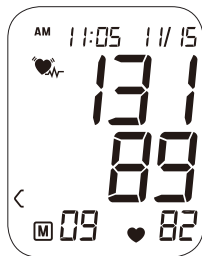
Fig. 4

Clearing Measurements From Memory

Each memory bank must be cleared separately. To delete all results in a given bank:

- When the monitor is off, press and release the **M** button to access the memory bank feature; the first display is the 'Average' display.
- Press and hold the **M** button for 3-seconds; the display will show CL and 00
- The memory bank has been cleared and will show 0 for all areas until a new measurement is taken.

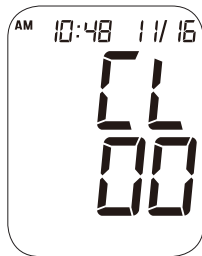CARE AND MAINTENANCE

Proper care and maintenance of your blood pressure monitor will help ensure long life for the product; improper use of the monitor may void the included warranty.

Always disconnect the cuff and tubing from the monitor prior to cleaning and storage.

Monitor Care and Maintenance

- Do not use any liquids on the monitor; use a soft, dry cloth to clean the monitor as needed
- Storage outside of stated storage temperature may result in measurement error or device malfunction; storage environment temperature is: -4°F – 122°F (-20°C – 50°C); Humidity: ≤85%RH (non- condensing)
- Avoid high temperatures and direct sunlight
- Always keep the unit unplugged while not in use; remove the batteries if the unit will not be used for an extended period of time
- Do not attempt to disassemble your monitor; disassembling your monitor may void the manufacturers warranty
- Do not subject the monitor to strong shocks; take care not to drop the monitor
- No component can be maintained by user in the monitor. The circuit diagrams, component part lists, descriptions, calibrations instructions, or other information which will assist the user's appropriately qualified technical personnel to repair those parts of equipment which are designated repairable can be supplied

CARE AND MAINTENANCE

Cuff Care and Maintenance

- The cuff may be spot cleaned with a mild-detergent as needed
- To disinfect the cuff, wipe the inside (skin contact side) of the cuff using a cloth lightly moistened with 70% - 90% ethyl rubbing alcohol and allow to air dry; it is suggested to disinfect the cuff frequently when used in clinical settings or if the cuff is contaminated in any way
- Never attempt to iron or dry the cuff; if the cuff becomes wet, allow the cuff to air dry completely before use or storage
- Do not attempt to remove the tube and bladder from the nylon cuff as this may damage the cuff
- Take care not to forcefully bend or kink the tubing during storage
- It is recommended the performance should be checked every 2 years or after a repair. Please contact customer care at 866-326-1313.
- It is recommended to clean the cuff every 200 uses.
- The monitor can maintain the safety and performance characteristics for minimum of 10,000 measurements of three years
- The cuff integrity is maintained after 1,000 open- close cycles of the closure

DEVICE & LABEL SYMBOLS

These symbols may appear on your device, instructions or packaging and may vary by make and model.

Symbol	Meaning
	Read This Manual—All included manuals should be read prior to first use
	Warning—Symbol indicates a warning, prohibition or mandatory action that mitigates a risk that is not necessarily obvious to the device operator
	Type BF Applied Parts—Indicates that a part of this unit comes in contact with the patient in order to carry out it's intended function; in the case of this device the cuff is the Type BF Applied Part
	Environment Protection—dispose of this product properly; consult with your local recycling ordinances for proper recycling and disposal
	Keep Dry—This device should be kept dry; never submerge the unit or cuff. Consult with the Care and Maintenance section of this manual for information on cleaning your monitor

DESCRIPTION OF DISPLAY SYMBOLS

Symbol	Meaning
SYS	Systolic Blood Pressure—The ‘top’ blood pressure result
DIA	Diastolic Blood Pressure—The ‘bottom’ blood pressure result
PUL	Pulse Rate—Pulse rate per minute during the measurement
mmHg	Millimeter of Mercury—Unit of blood pressure measurement for this device
	Low Battery—Battery life indicator
	Pulse Reading—Appears when pulse reading is displayed.
	Irregular Heartbeat Symbol—An irregular heartbeat was detected during the measurement
U1 U2	User 1/User 2—Indicates which user is active
A	Average—The display shows the average of all measurements in memory
AM PM	AM Average / PM Average—The display shows the average of all measurements taken within AM or PM time constraints
M	Memory—The display shows the measurements stored in memory

TROUBLESHOOTING

Problem	Possible Cause	Solution
Display shows an abnormal result	Cuff position was not correct or was not tight enough	Apply the cuff properly and attempt a new measurement
	Body posture was not correct during measurement	Review the 'Taking a Measurement' portion of this manual and attempt a new measurement
	Speaking, excessive movement, angry, excited or nervous anxiety during measurement	Wait a period of time and attempt a new measurement after calm has returned; do not speak or move during measurement
Display is blank when power is on	Batteries may be expired or installed improperly	Check the polarity of the batteries and reinstall if necessary; replace batteries with new batteries
No response when POWER is pressed or when new batteries have been loaded	Incorrect operation; possible strong electromagnetic interference	Remove batteries and wait a minimum of five minutes; reinstall batteries and attempt a new measurement
	The battery is low.	Replace the batteries
Monitor keeps re-inflating	Circuit is locked.	Remove and reinsert the batteries; attempt new measurement

ERROR CODES

Code	Meaning	Corrective Action
E0	Pressure system is unstable before measurement	Re-apply the cuff and attempt new measurement.
E1/E2	Failed to detect systolic/diastolic pressure	
E3/ E4	Pneumatic system leakage or cuff is too loose/ too tight during inflation	Re-apply the cuff and attempt new measurement.
E5	The cuff pressure is above 300mmHg	Wait 5 minutes. Sit comfortable and remain still; attempt new measurement. If the error persists, please contact Customer Care
E6	More Than 3 minutes with cuff pressure above 15 mmHg	
E7	EEPROM accessing error	
E8	Device Parameter checking error	
ER A	Pressure Sensor parameter error	

TECHNICAL ALARM DESCRIPTION

The monitor will show 'HI' or 'Lo' as technical alarm on LCD with no delay if the determined blood pressure (systolic or diastolic) is outside the rated range specified in part SPECIFICATIONS. In this case, you should consult a physician or check if your operation violated the instructions.

The technical alarm condition (outside the rated range) is preset in the factory and cannot be adjusted or inactivated. This alarm condition is assigned as low priority according to IEC 60601-1-8.

The technical alarm is non-latching and need no reset. The signal displayed on LCD will disappear automatically after about 8 seconds.

FCC STATEMENT

NOTE:

POTENTIAL FOR RADIO/TELEVISION INTERFERENCE (for U.S.A. only)

This product has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation. The product generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If the product does cause harmful interference to radio or television reception, which can be determined by turning the product on and off, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna
- Increase the separation between the product and the receiver.
- Connect the product into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

POTENTIAL FOR RADIO/TELEVISION INTERFERENCE (for Canada only)

This digital apparatus does not exceed the Class B limits for radio noise emissions from digital apparatus as set out in the interference-causing equipment standard entitled "Digital Apparatus", ICES-003 of the Canadian Department of Communications.

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Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

ELECTROMAGNETIC COMPATIBILITY

Table 1 EMISSIONS

For all ME EQUIPMENT and ME SYSTEMS

Guidance and manufacture's declaration - electromagnetic emissions

This monitor is intended for use in the electromagnetic environment specified below.

The customer or the user of this monitor should assure that it is used in such an environment.

Phenomenon	Compliance	Electromagnetic Environment Guidance
RF emissions	CISPR 11 Group 1, Class B	Home healthcare environment
Harmonic distortion	IEC 61000-3-2 Class A	Home healthcare environment
Voltage fluctuations and flicker	IEC 61000-3-3 Compliance	Home healthcare environment

ELECTROMAGNETIC COMPATIBILITY

Table 2 — ENCLOSURE PORT

For all ME EQUIPMENT and ME SYSTEMS

Guidance and manufacture's declaration - electromagnetic immunity

This monitor is intended for use in the electromagnetic environment specified below.

The customer or the user of this monitor should assure that it is used in such an environment.

Phenomenon	Basic EMC Standard	Immunity test levels
		Home healthcare environment
Electrostatic Discharge	IEC 61000-4-2	± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air
Radiated RF EM field	IEC 61000-4-3	10V/m 80MHz-2.7GHz 80% AM at 1kHz
Proximity fields from RF wireless communications equipment	IEC 61000-4-3	Refer to table 3
Rated power frequency magnetic fields	IEC 61000-4-8	30A/m 50Hz or 60Hz

ELECTROMAGNETIC COMPATIBILITY

Table 3 — PROXIMITY FIELDS FROM RF WIRELESS COMMUNICATIONS EQUIPMENT

For ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING Guidance and manufacturer's declaration - electromagnetic immunity

This monitor is intended for use in the electromagnetic environment specified below.

The customer or the user of this monitor should assure that it is used in such an environment.

Test frequency (MHz)	Band (MHz)	Immunity test levels
		Professional healthcare facility environment
385	380-390	Pulse modulation 18Hz, 27V/m
450	430-470	FM, ± 5 kHz deviation, 1kHz sine, 28V/m
710	704-787	Pulse modulation 217Hz, 9V/m
745		
780		
810		
870	800-960	Pulse modulation 18Hz, 28V/m
930		
1720		
1845		
1970	1700-1990	Pulse modulation 217Hz, 28V/m
2450		
5240		
5500		
5785	2400-2570	Pulse modulation 217Hz, 28V/m
	5100-5800	Pulse modulation 217Hz, 9V/m

PRODUCT SPECIFICATIONS

Name	Automatic Digital Blood Pressure Arm Monitor
Model Number	01-554
Display System	Digital display/LCD
Measuring Method	Oscillometric Method, Automatic Air Inflation and Measurement
Power Source	4 x 1.5V  AAA Batteries
Measuring Range	Cuff Pressure: 0-300mmHg, Systolic pressure:60-260mmHg, Diastolic Pressure: 40-199mmHg; Pulse: 40-180 beats/minute
Accuracy	Pressure: ± 3 mmHg; Pulse: $\pm 5\%$
Deflation	Automatic pressure release valve
Memory	Built-in memory with 2 banks, 60 measurements per bank
Automatic Power-Off	Approximately 1 minute after last button operation
Battery Life	Approximately 100 measurements
Operation Environment	Temperature 50°F – 104°F (10°C – 40°C); Humidity $\leq 85\%$ RH max
Storage Environment	Temperature -4°F – 122°F (-20°C – 50°C); Humidity, 85% RH max
Monitor Dimensions	5-29/32" x 3-3/4" x 1-5/8"
Monitor Weight	8.0 oz. (with batteries); 6.6 oz. (without batteries)
Arm Circumference Range	Adult size cuff fits arm circumference: 8.6" – 11.8" (22 – 30 cm) Large adult cuff fits arm circumference: 11.8" – 16.5" (30 – 42 cm)
Accessories	Instruction Manual, Quick Start Guide, Storage Case, 4 AAA Batteries Replacement Cuff Order Form, Blood Pressure Log

Specifications are subject to change without notice

LIFETIME LIMITED WARRANTY

Congratulations on your purchase of a Digital Blood Pressure Monitor. Your Digital Blood Pressure Monitor is covered by the following limited warranty commencing upon the date of purchase, and subject to the following terms and conditions:

The warrantor warrants that its Digital Blood Pressure Monitor will be free from defects in materials and workmanship under normal consumer usage for a period of one year for the original purchaser of the product.

Periodic maintenance, repair and replacement of parts due to normal wear and tear are excluded from coverage. Defects or damage that result from: (a) improper operation, storage, misuse or abuse, accident or neglect, such as physical damage (cracks, scratches, etc.) to the surface of the product resulting from misuse; (b) contact with liquid, water, rain, extreme humidity or heavy perspiration, sand, dirt or the like, extreme heat, or food; (c) use of the Digital Blood Pressure Monitor for commercial purposes or subjecting the Digital Blood Pressure Monitor to abnormal usage or conditions; or (d) other acts which are not the fault of the Warrantor, are excluded from coverage. This warranty does not cover batteries or other power sources that may be provided with, or used with the Digital Blood Pressure Monitor.

If the Digital Blood Pressure Monitor fails to conform to this limited warranty, return the Digital Blood Pressure Monitor postage prepaid to: **Attn: Repair Department, 1175 Lakeside Drive, Gurnee, IL, 60031**. When returning a product, please also include: (i) a copy of your receipt, bill of sale or other comparable proof of purchase; (ii) a written description of the problem; and (iii) your name, address and telephone number. Carefully package the product to avoid any damage that may occur while in transit; shipping insurance with returned receipt is recommended. At our option, the Warrantor will repair or replace the unit found to be defective in materials or workmanship under normal consumer usage. The purchaser will be notified of any additional repairs required prior to completing the repair, and will be responsible for parts charges, if any, and repair charges not covered by this limited warranty.

EXCEPT AS PROVIDED FOR IN THIS LIMITED WARRANTY, ALL EXPRESS AND IMPLIED WARRANTIES AND CONDITIONS ARE DISCLAIMED, INCLUDING WITHOUT LIMITATION THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. THE REPAIR OR REPLACEMENT AS PROVIDED UNDER THIS LIMITED WARRANTY IS THE EXCLUSIVE REMEDY OF THE CONSUMER, AND IS PROVIDED IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. IN NO EVENT SHALL THE WARRANTOR BE LIABLE, WHETHER IN CONTRACT OR TORT (INCLUDING NEGLIGENCE) FOR DAMAGES IN EXCESS OF THE PURCHASE PRICE OF THE PRODUCT, OR FOR ANY INDIRECT, INCIDENTAL, SPECIAL OR CONSEQUENTIAL DAMAGES OF ANY KIND, OR FOR DAMAGES TO, OR LOSS OF, OTHER PROPERTY OR EQUIPMENT OR PERSONAL INJURIES TO THE FULL EXTENT THESE DAMAGES MAY BE DISCLAIMED BY LAW.

Some states and jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitation on the length of an implied warranty, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that vary from state to state or from one jurisdiction to another.

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