ESPAÑOL

Welch Allyn tensiómètros

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ENGLISH

Welch Allyn Sphygmomanometers

Dear customer, We are delighted that you have decided to purchase a precision blood pressure meter from Welch Allyn. In this blood pressure meter, the manometer is directly connected to the pump valve. It is a typical modern one-hand unit.

Disytest-Home

This unit has a modern patented red pushbutton valve. This valve is easy to use and provides smooth operation. By lightly pressing on the pushbutton, the air can be released at a rate of 2-3 mmHg/sec. This rate can be finely adjusted. When the pushbutton is pressed down into its final position and snaps into place, the quick release unit is activated, allowing the cuff to be completely deflated. Lightly pressing on the upper part of the pushbutton causes it to pop out of its holder and back into the final position. The valve is then closed again and ready for the next measurement. A diaphragm case, which has a connecting piece for easy connection of the stethoscope tube and earpieces, is built into the cuff.

Important:

All units are equipped with an excess pressure prevention device (Pressostabil®).

Fitting the cuff

After choosing the appropriate cuff size, the arm cuff is put in position, usually on the left upper arm. The arm circumference range is indicated on the cuff (also see table). The cuff should be tightly placed against the soft parts of the arm, but should not exert any pressure on the artery when deflated. The cuff should be placed approximately in the middle of the upper arm, in such a way that the tube points downward (in a distal direction) and is about in the middle of the bicep. If possible, the diaphragm case should be positioned on the artery so that optimum auscultation is possible. The stethoscope tube is then connected to the diaphragm case of the cuff by turning the snap closure sightly to the right. The earpieces of the stethoscope are then attached. The arm should be almost fully extended at the elbow and the muscles should be completely relaxed when the cuff is being put in place and during measurement.

Measurement

After putting the cuff in place, air is pumped into the entire system with the valve closed. After pumping up the cuff, the patient must remain completely motionless and one should avoid pressing on the cuff, as this affects the measurement. By lightly pressing on the finely adjustable red pushbutton valve with the index finger, the air can be released from the cuff. The speed of this release can be very finely adjusted using the pushbutton valve. This device is a so called one-hand model, leaving one hand free for auscultation of systolic and diastolic blood pressure with the stethoscope. After noting the upper and lower blood pressure values, the screw valve should be completely opened. This guarantees that when the cuff is put back in place, the air will have completely escaped. The valve should again be closed before beginning another measurement.

Specifications

Measurement range	0-300 mmHg
Precision	± 3 mmHg
Scale graduation	2 mmHg
Excess pressure prevention device	Pressostabil® (prevents pressure of more than 350 mmHg)

Cuff sizes for Welch Allyn	Arm size for velcro D-ring cuffs	Arm size for velcro cuffs
Adults,large upper arm		34 - 52 cm
Adults, upper arm	24 - 33 cm	29 - 42 cm

The Welch Allyn dimensions for arm size were developed over many years of cooperation with our customers and according to their wishes. Welch Allyn cuffs offer optimum measurement reliability and user-friendliness. Our One-Piece-Cuffs apply to the AAMI standards for suitable fitting.

Care instructions

Reusable Welch Allyn cuffs can be washed in a washing machine at 60°C, but must not be spin-dried. The blood pressure meter can be disinfected by wiping clean with conventional agents.

Important note

As a rule, blood pressure meters should be technically checked every 2 years. In such cases, return your unit to Welch Allyn or to your dealer.

Calibration

Should the manometer fail to be accurate to ± 3 mmHg, Welch Allyn will reset the manometer free of charge within a period of 1 year from the purchase date.

Accessories

We provide a 1 year guarantee against defects in materials or workmanship defects for cuffs, rubber bags, and tubes.

Warranty

We provide against defect in materials and workmanship for your Welch Allyn manometer movement the following guarantee:

Disytest-Home: 1 year guarantee. Should the manometer be defective or deviate from the manufacturer's specifications, Welch Allyn will repair or replace the unit or its parts free of charge.This guarantee does not apply in the case of damage caused by incorrect handling, improper use, accidents, modifications, or during transportation. Guarantee claims shall not apply if the device is not used in accordance with the manufacturer's recommendations or is opened by unauthorized persons.

Welch Allyn®

Disytest-Home Blood Pressure Meter

Directions for Use

WelchAllyn®

Advancing Frontline Care™

Bestell-Nr. 10-11-213 Rev. B

Welch Allyn USA (Headquarters)
Tel: 1-315-685-4560 or 1-800-535-6663
Fax: 1-315-685-0054

Welch Allyn Ireland (European Customer Service)
Tel: (353) 46 90 67700
Fax: (353) 46 90 67756

Welch Allyn Canada
Tel: 1-905-890-0004
Fax: 1-905-890-0008

Welch Allyn U.K.
Tel: 0-207-365-6780
Fax: 0-207-365-9694

Welch Allyn Italy
Tel: (+39) 02-6968-2425
Fax: (+39) 02-6963-3697

Welch Allyn Australia
Tel: (+61) 2-9638-3000
Fax: (+61) 2-9638-3500

Welch Allyn China
Tel: (+86) 21 6327 9631
Fax: (+86) 21 6327 9632

Welch Allyn Japan
Tel: (+81) 3-219-0071
Fax: (+81) 3-219-0074

Welch Allyn Singapore
Tel: (+65) 291-0882
Fax: (+65) 291-5780

Welch Allyn Germany
Tel: (+49) 7477 9271 70
Fax: (+49) 7477 9271 90

Welch Allyn France
Tel: (+33) 60 09 33 66 or (+33) 55 69 58 49
Fax: (+33) 60 09 67 97

Welch Allyn Spain
Tel: (+34) 917499357
Fax: (+34) 917499361

Welch Allyn South Africa
Tel: (+27) 11 777 7509
Fax: (+27) 11 777 7510

 Welch Allyn GmbH & Co. KG
Zollerstrasse 2-4
D-72417 Jungingen, Germany
Tel. (+49) 7477-9271-0 Fax (+49) 7477-9271-90 e-mail: info@welchallyn.de

 The CE mark on this product indicates that it is in conformity with the provisions of Council Directive 93/42/EEC.

 European Regulatory Manager, Medical Division, Welch Allyn Ltd. Navan
Business Park, Dublin Road, Navan, County Meath, Republic of Ireland
Tel: +353 46 90 67700 Fax: +353 46 90 67756

© 2006 Welch Allyn, Inc. All rights reserved. No one is permitted to reproduce or duplicate, in any form, this manual or any part thereof without permission from Welch Allyn. Welch Allyn assumes no responsibility for any injury to anyone, or for any illegal or improper use of the product that may result from failure to use the product in accordance with the instructions, cautions, warnings, or statement of intended use published in this manual. Welch Allyn® is a trademark of Welch Allyn, Inc.

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